
BESPOKE
STAIRLIFTS LTD.

INFINITY

CURVED STAIRLIFT INSTALLATION MANUAL

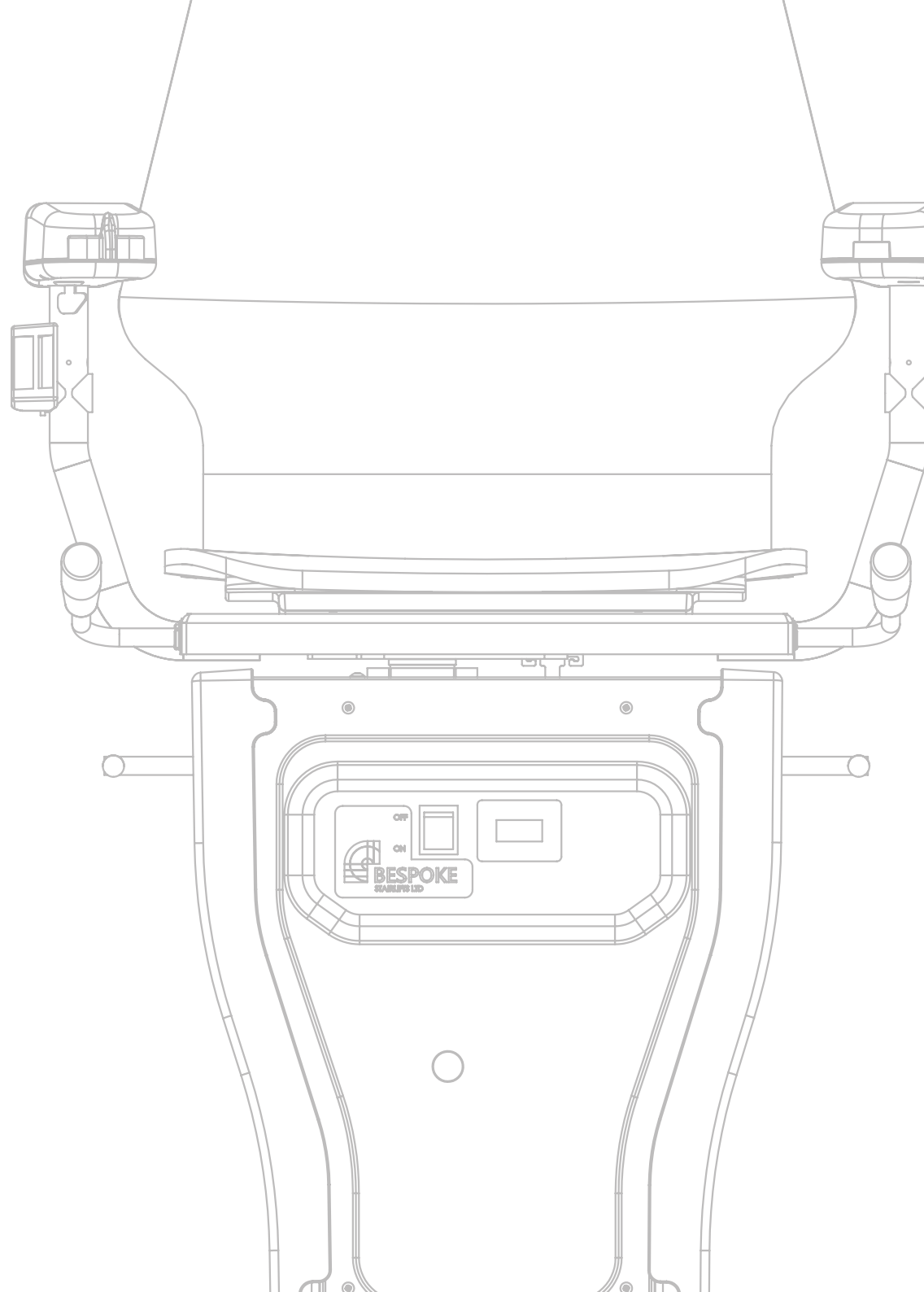December 2024

CONTENTS

1.0	Pre-Installation 5	5.0	Fitting Rail Furniture 55	9.0	Wiring Diagram 101
	Pré-installation FR		Installation des accessoires du rail FR		Schémas de câblage FR
	Preinstalación ES		Instalación de los accesorios del riel ES		Diagramas de cableado ES
	Vorinstallation DE		Montage der Schienenmöbel DE		Verdrahtungspläne DE
	Pre-installazione IT		Montaggio degli accessori del binario IT		Schemi elettrici IT
	Pré-instalação PT		Instalação dos acessórios do trilho PT		Diagramas de fiação PT
2.0	Assembling the Rail 15	6.0	Programming 71		
	Assemblage du rail FR		Programmation FR		
	Montaje del riel ES		Programación ES		
	Montage der Schiene DE		Programmierung DE		
	Assemblaggio della rotaia IT		Programmazione IT		
	Montagem do trilho PT		Programação PT		
3.0	Mounting the Charger 31	7.0	Safety Checks 87		
	Montage du chargeur FR		Vérifications de sécurité FR		
	Montaje del cargador ES		Comprobaciones de seguridad ES		
	Montage des Ladegeräts DE		Sicherheitsprüfungen DE		
	Montaggio del caricatore IT		Controlli di sicurezza IT		
	Montagem do carregador PT		Verificações de segurança PT		
4.0	Carriage Preparation 35	8.0	Diagnostics 91		
	Préparation de la Carrosse FR		Diagnostics FR		
	Preparación del Carruaje ES		Diagnósticos ES		
	Vorbereitung der Kutsche DE		Diagnostik DE		
	Preparazione della Carrozza IT		Diagnostica IT		
	Preparação da Carruagem PT		Diagnósticos PT		

The information contained within this manual has been designed for use by Bespoke approved engineers who have received the appropriate product training in the following categories:

1. Product installation

2. Testing and commissioning

It is also assumed that the person using this manual will have the basic knowledge and skills required in both the mechanical and electrical aspects to apply consideration and awareness for the safety of themselves and others involving their work.

The contents of this manual are in the recommended order of placement, which forms the full installation and commissioning of this standard model.

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1.1 Site Assessment

The guidance given in this document is to assist in the installation of a stairlift. It is designed to ensure that surveyors, designers and installers have necessary information that may adhere to requirements of international standards, and is intended to ensure the safety of stairlift users.

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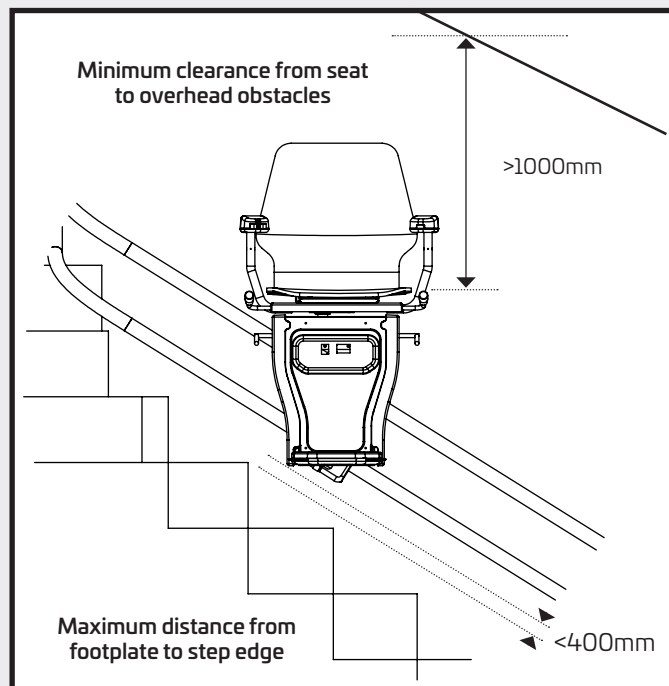
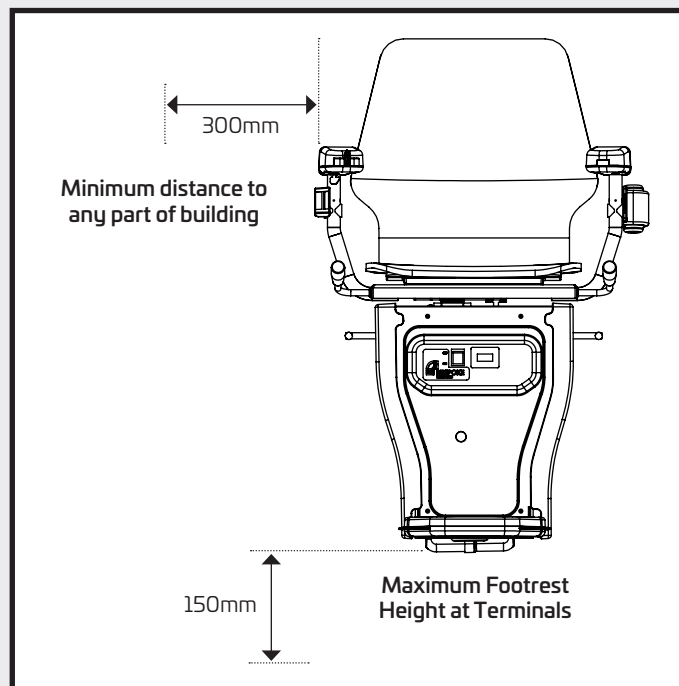
1.1 Site Assessment

These diagrams are designed to illustrate the safe locating of the Infinity Curved Stairlift to advise both surveyors, designers and installers. If these guidelines cannot be achieved, a detailed risk assessment must be carried out and agreed by both the customer and experienced surveyor.

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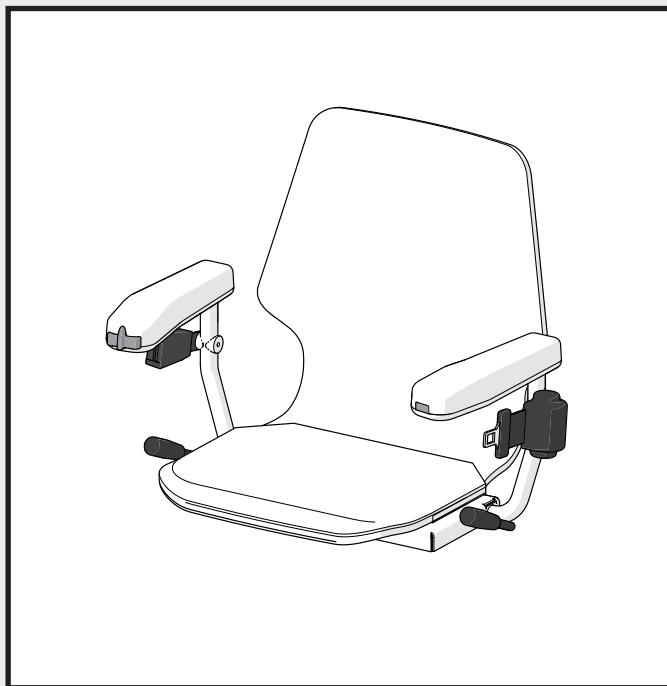
1.2 Box Contents

1 x Infinity Carriage Unit
1 x Infinity Carriage Kit Box

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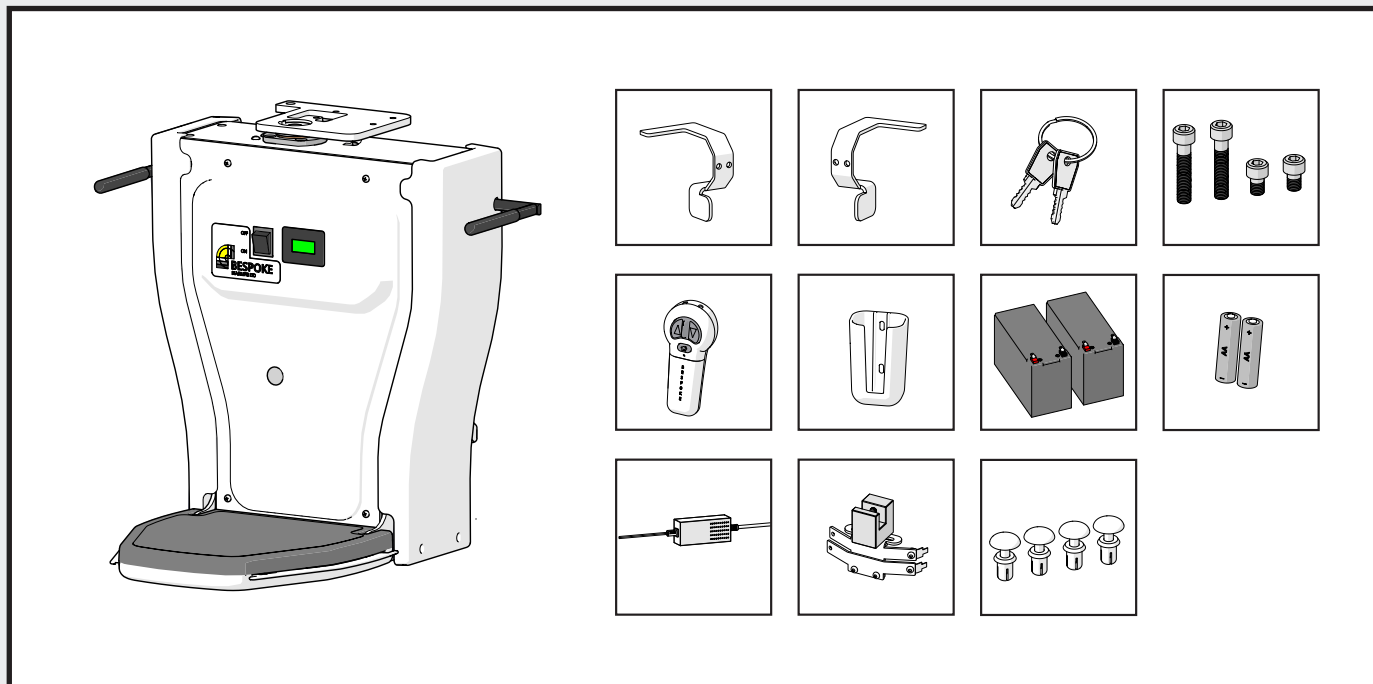
1.2 Box Contents

Charger
Charge points (x2 Min)
Endstop A
Endstop B
Remote control (x2Min)
Remote Holster (x2 Min)
2x 12V Batteries

4x AA Batteries
Keys
2x M8x40mm bolts
2x M8x12mm bolts
4x 4mm set cover fixings

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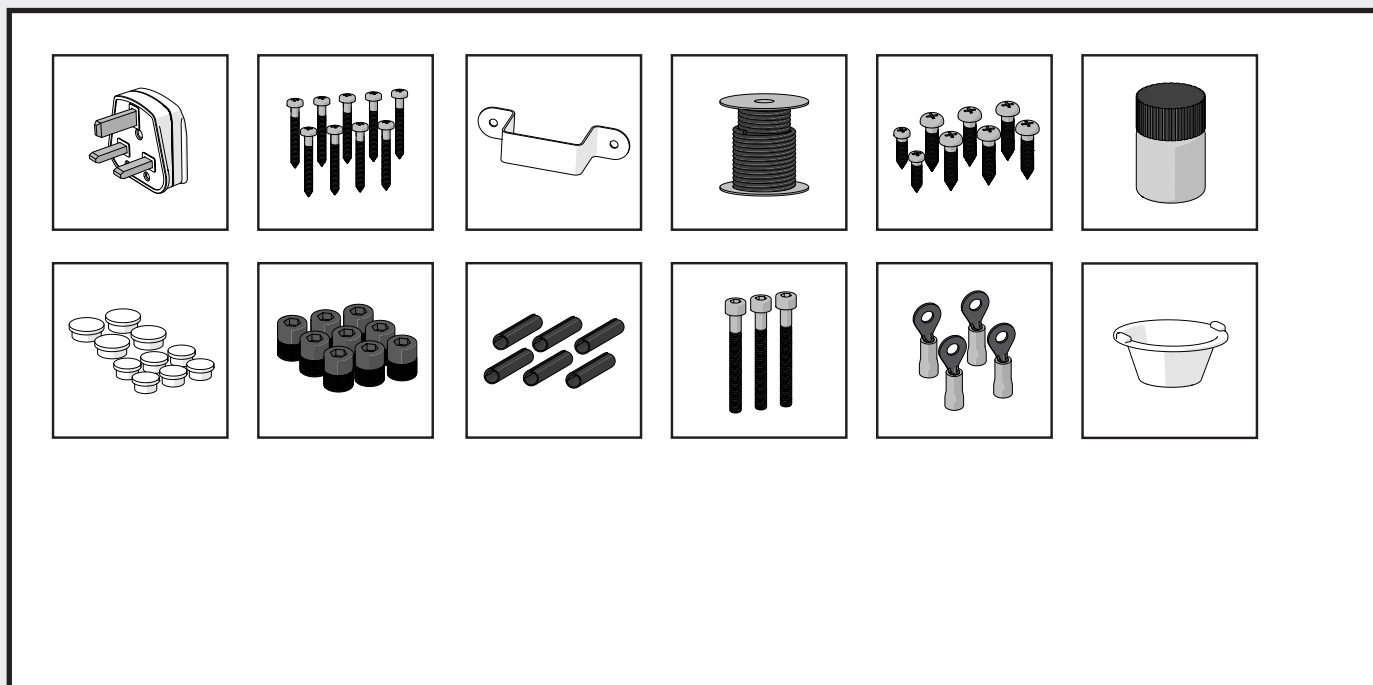
1.2 Box Contents

Charger Plug (Regional)
Base Screws (Quantity Varies)
Charge Bracket
2-Core cable
Charge Point Touch Up Paint
Rail & Leg bungs
Base Grub Screws M8x10mm

Spiral Pins M6x45mm
Racking Joint Bolt M5x45mm
Eyelet Connector
Grease
Female Spade Connector

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1.3 Tools Required

- Spirit Level
- Tape measure (5m)
- Magnetic Angle finder
- Digital multimeter (Voltage, Amps, Milliamps and Continuity)
- Battery Tester
- Socket Tester
- Stanley Knife/Utility Knife
- 6mm Twisted drill bit
- Hammer (Claw or Ball pein)
- Hexagon Keys Sizes 2-10mm (T handles or long series)
- Nut Runners sizes 5mm, 5.5mm, 7mm and 8mm
- Torque wrench/s with drive range of 12- 100
- Bit/Socket set 2.5mm – 10mm
- Square drive sockets 8-19mm
- Socket
- Extension Bar
- Ratchet
- Wire stripper
- Crimping tool
- Electrical or combination pliers
- Electrical side cutters
- External Circlip pliers
- Cross head screw driver set (variable sizes)
- Flat screwdriver set (variable sizes)
- Terminal screwdriver
- Electric Combi drill
- Lube Grease
- Dust sheet/ Floor protector
- Vacuum Cleaner
- Cable puller
- Scriber
- Finger File

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- 1-3. Staircase elevations, staircase dimensions and leg heights.
4. Bases required.
5. Order details

- a. Rail Angle
b. Knee clearance
c. Total leg height
d. Distance from walls

FR Le schéma d'installation
ES El dibujo de instalación
DE Die Installationszeichnung
IT Il disegno di installazione
PT O Desenho da Instalação


SECTION 2.0

Assembling the Rail

Assemblage du rail **FR**
Montaje del riel **ES**
Montage der Schiene **DE**
Assemblaggio della rotaia **IT**
Montagem do trilho **PT**

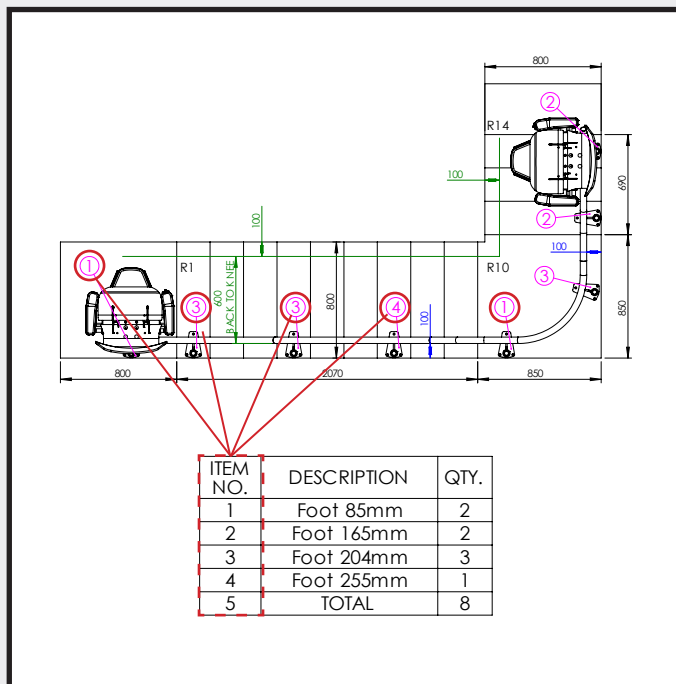2.1 Positioning the Bases

The number on the plan view of the installation drawing relates to the item number on the leg table. Item numbers are different sized bases.

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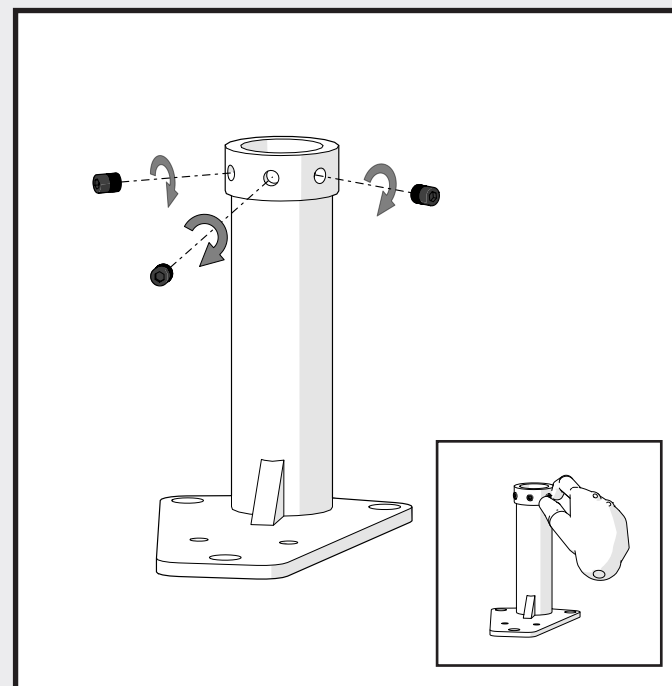
Before placing on the stairs, insert 3x M6 grub screws into each base.

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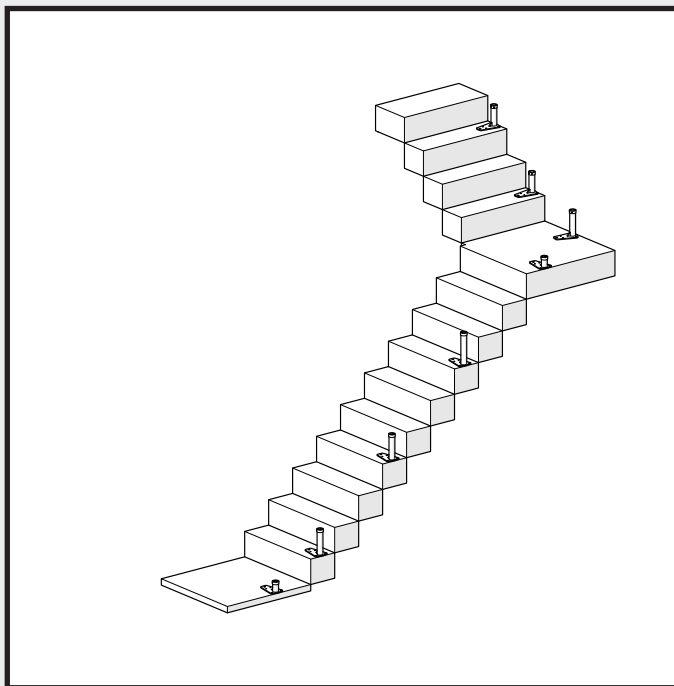
2.2 Rail Positioning

Place the correct size base as illustrated on the drawing on the correct step.

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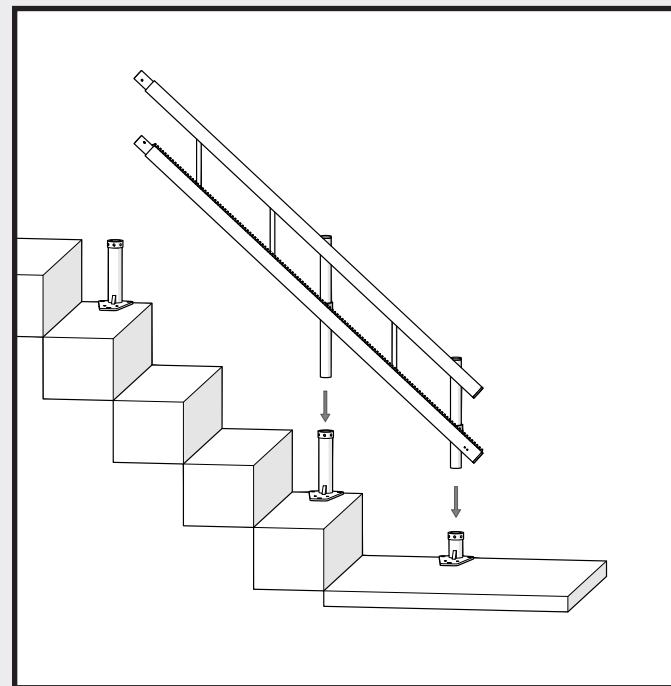
Starting with the bottom section, place the rail stanchions in the bases and gently rest the sections against the wall. Be careful not to damage the decor.

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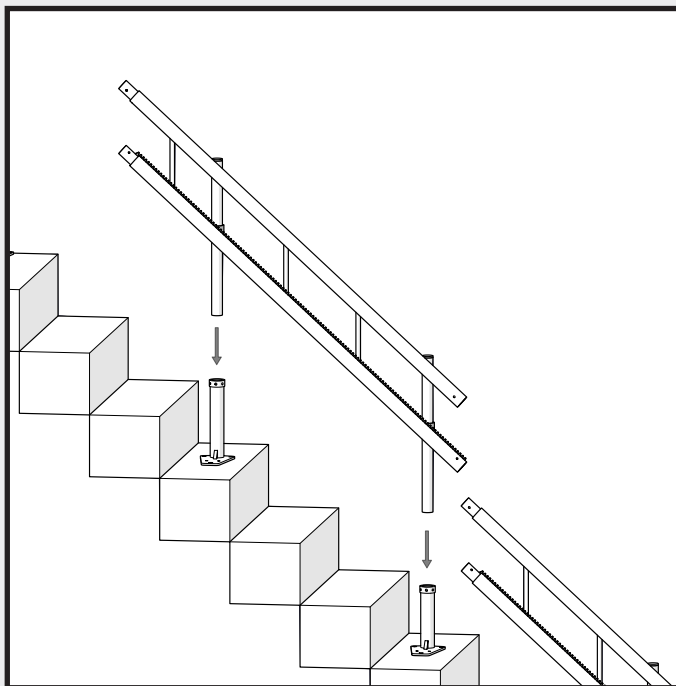
2.2 Rail Positioning

Do the same for the next section. Do not join the rails together at this point.

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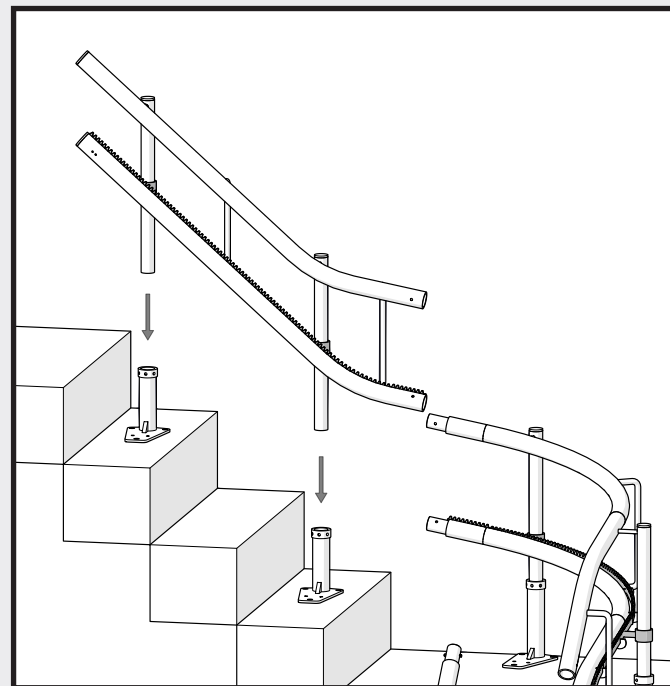
Continue to do the same for the remaining sections until all the sections are in the correct bases and in their correct position on the stairs.

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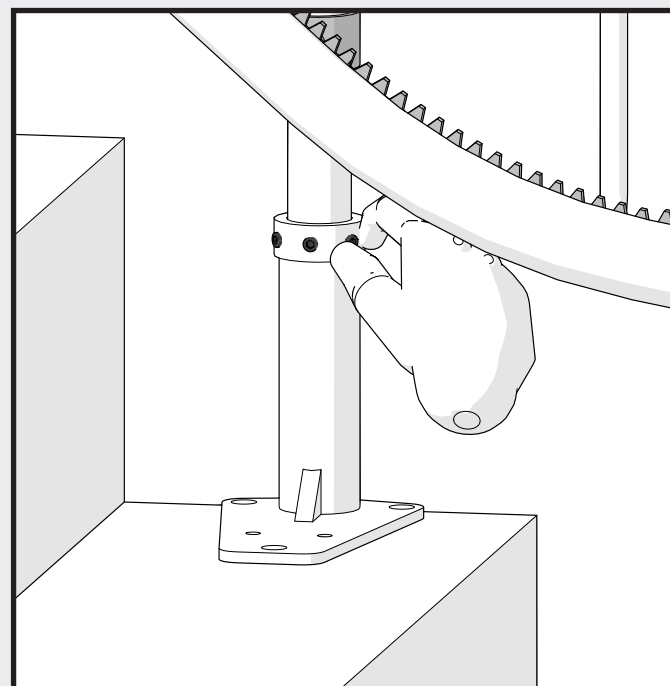
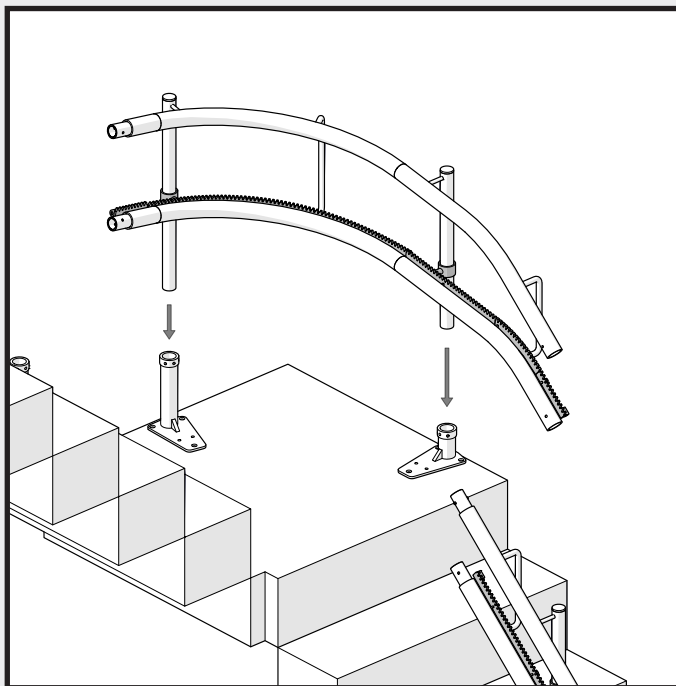
Place the stanchion into the correct base. Insert the 3x M6 Grub screws into the base. The grub screws need to be lightly tightened to support the rail. These will need to be adjusted before fully tightening.

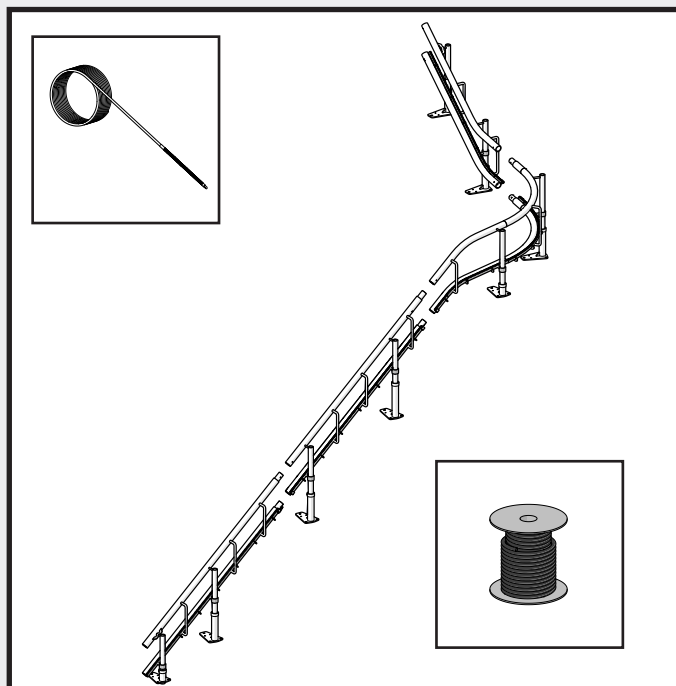
FR Positionnement du rail dans les bases
ES Colocación del riel en las bases
DE Positionierung der Schiene in den Basen
IT Posizionamento del binario nelle basi
PT Posicionamento do trilho nas bases

2.3.1

19

2.4 Inserting the wiring

Start with positioning the cable supplied with your curved stairlift near the bottom rail section of the stairlift.

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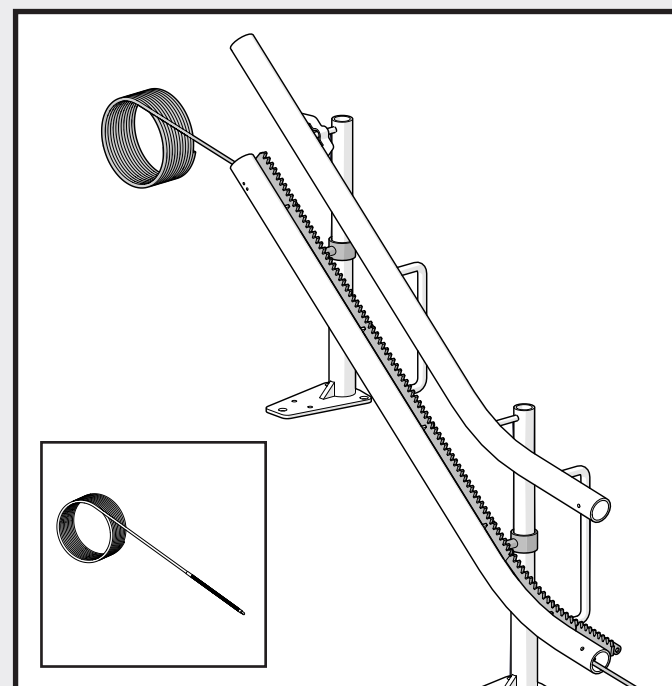
Feed your cable puller into the bottom tube of the top section of rail and feed down through all the sections of the rail until the bottom is reached. (For longer rails this may be done in sections from the top down.)

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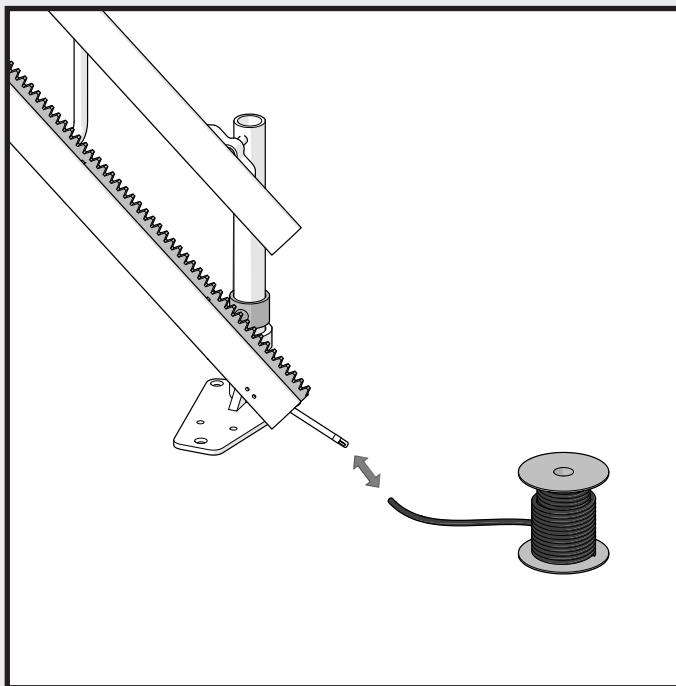
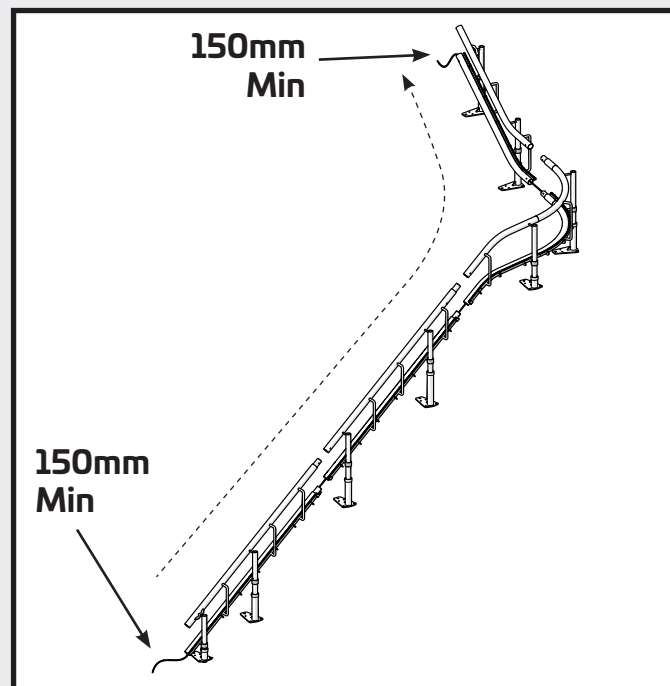
Securely tape together the cable pulley and to the 2 core cable. Ensure the cable is connected so when pulling through the rail the cable does not become detached.

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Now carefully feed the 2 core cable up from the bottom rail section until the 2 core cable reaches the top rail section. Ensure there is 150mm minimum at both ends of the rail.

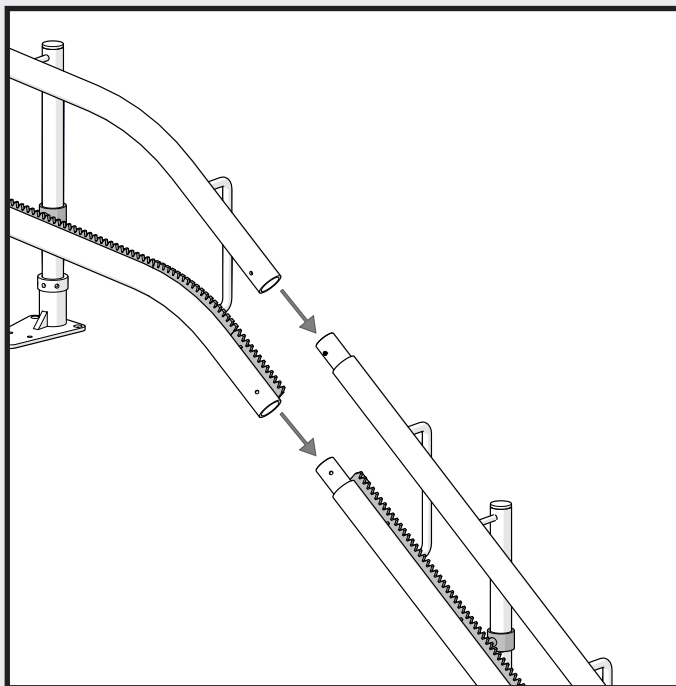
FR Insertion du câblage
ES Inserción del cableado
DE Einfügen der Verkabelung
IT Inserimento del cablaggio
PT Inserção da fiação

2.4.2

21

2.5 Assembling the Rail

Apply grease to male part of joint. Assemble rail joints together working from bottom to top of stairs, working with top tubes first.

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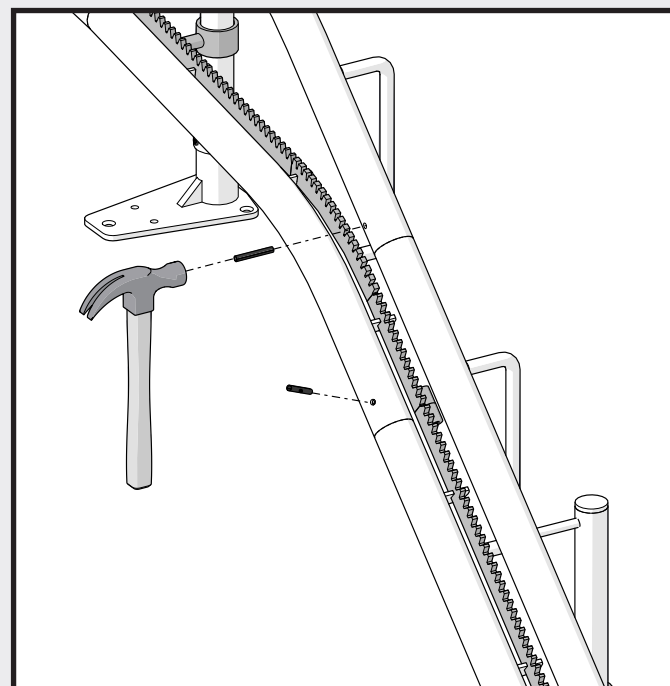
Pins should be flush with stair side of rail. Take care not to damage cable when inserting pins by looking through the hole to see if the cable is not in the way.

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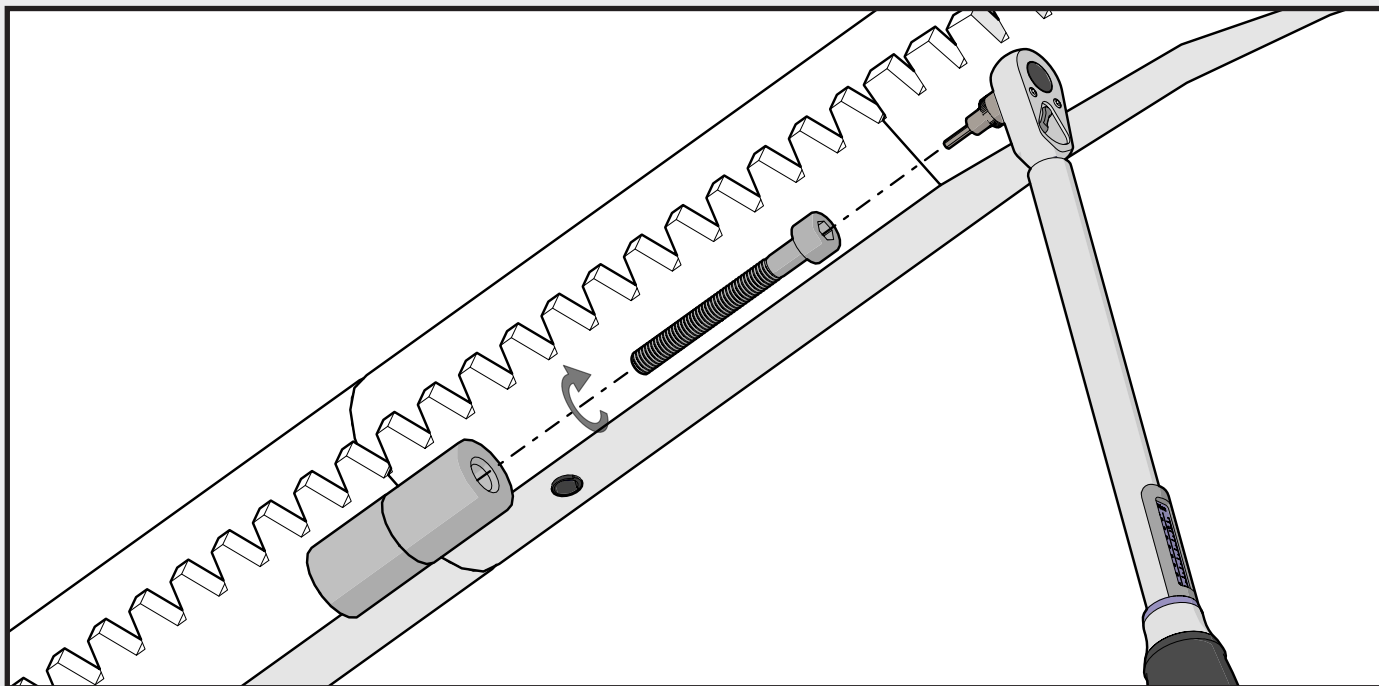
 **BESPOKE**
 STAIRLIFTS

2.5 Assembling the Rail

Finally, tighten the jointing bolts to 17 N-m.
Repeat the process for each of the section joints to
complete the rail connection assembly.

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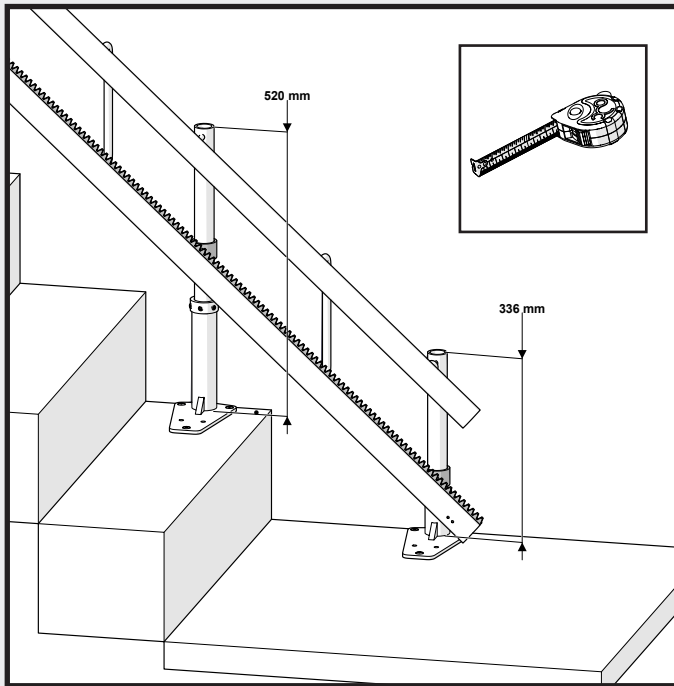
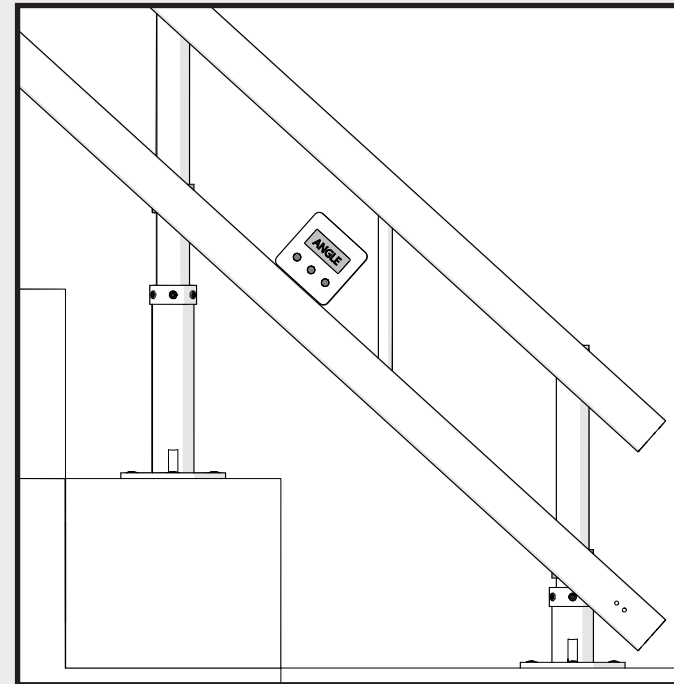
Now the rail is assembled check the heights of the stanchions and bases. These measurements are a guide, due to uneven stairs the numbers may not be exact.

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Check the angle of the stairlift rail using a digital angle finder. The angle of the rail should be used as a guide and closely correspond with the installation drawing.

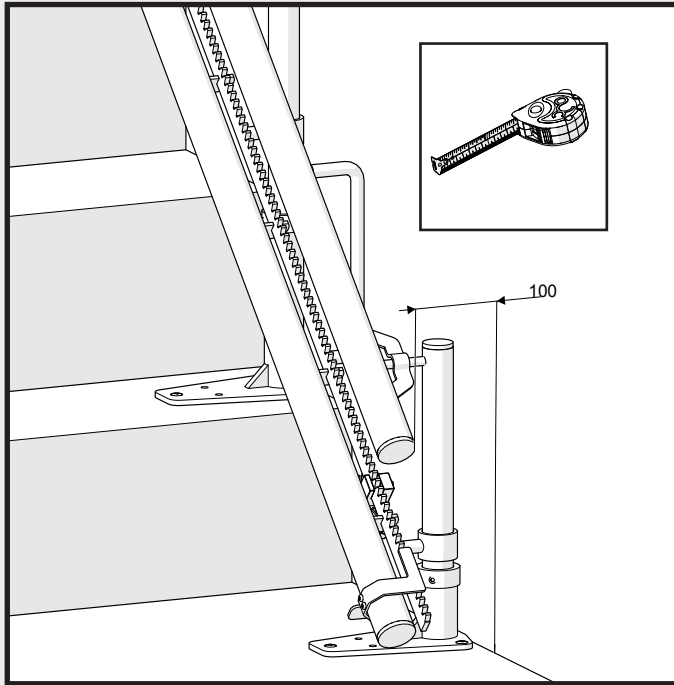
2.6.1

24

FR Hauteurs et angles
ES Alturas y ángulos
DE Höhen und Winkel
IT Altezze e angoli
PT Alturas e ângulos

2.7 Measurements to Walls

Check the clearance measurement of the rail at the bottom against the wall or closest obstruction. This should be no less than 100mm.

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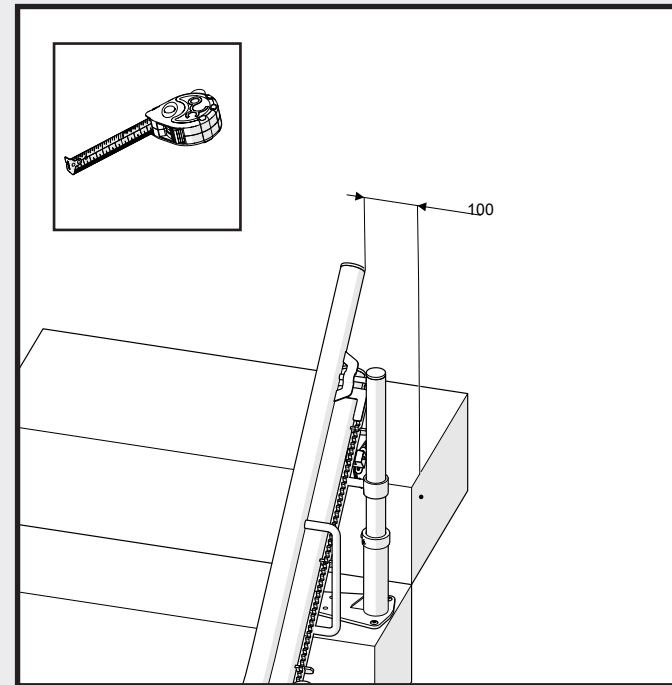
Work your way up the rail and position the straight sections the same 100mm from any wall or obstruction. Finally position the rail 100mm from any wall or obstruction at the top stanchion.

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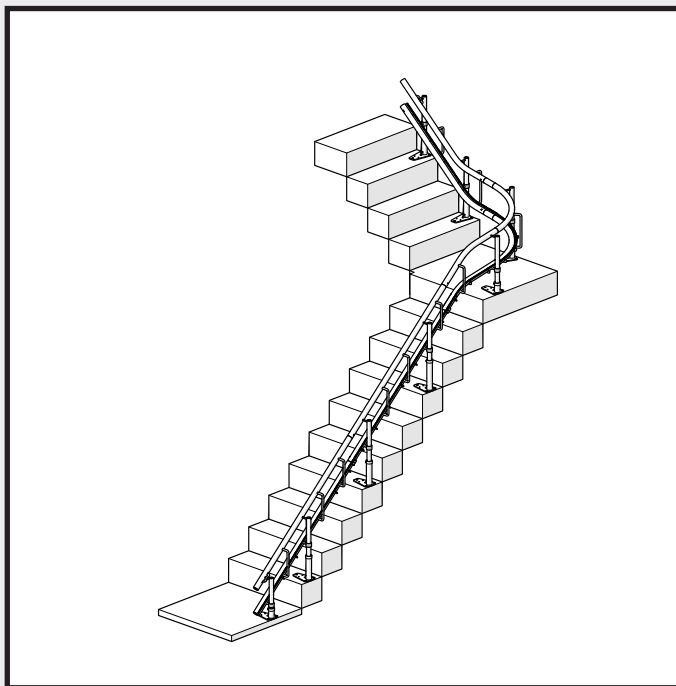
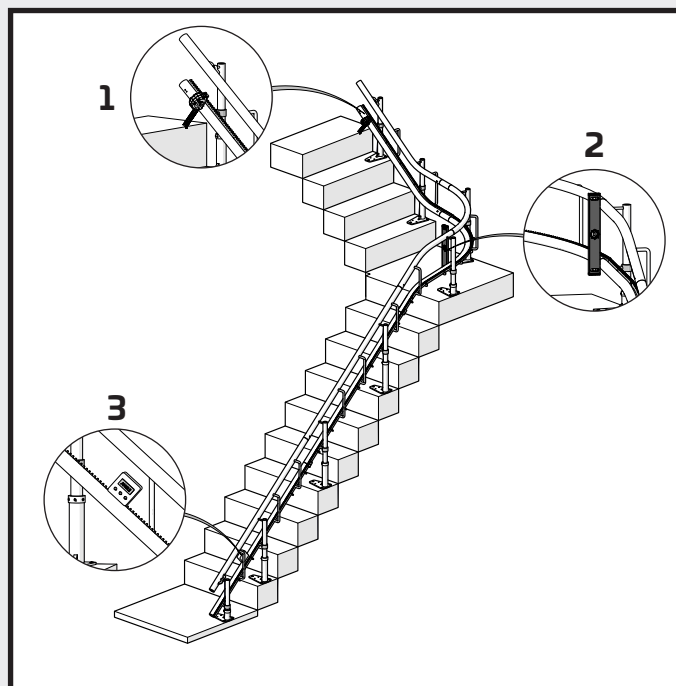
The rail should now be in an approximate position, before the rail is fixed to the stairs the clearances and angles must be checked to ensure the carriage runs correctly.

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1. The Rail should have a minimum clearance from the top nose of 100mm
2. The rail should be level.
3. The angles of the rail should correspond closely to the install drawing.

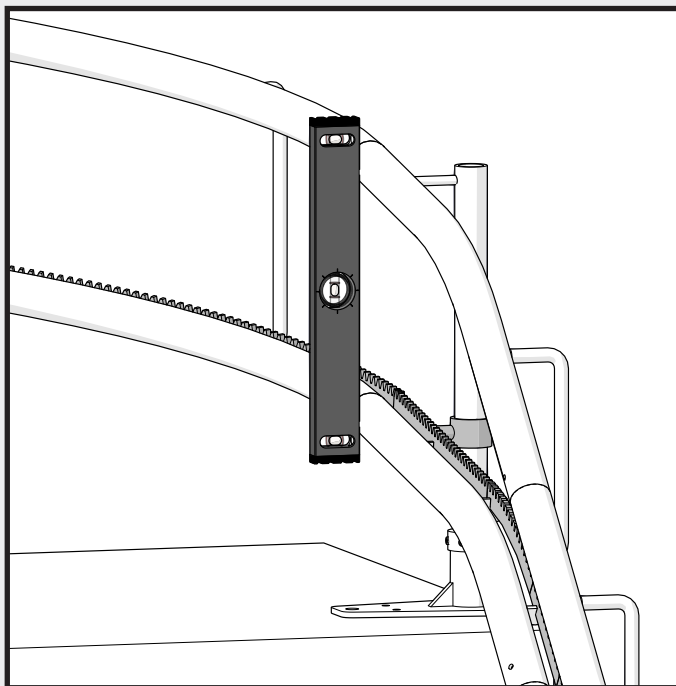
2.8.1

26

FR Vérifiez les dimensions du rail
ES Verifique las dimensiones del riel
DE Überprüfen Sie die Abmessungen der Schiene
IT Controlla le dimensioni del binario
PT Verifique as dimensões do trilho

2.8 Adjusting the rail.

If the rail is out of level it can be adjusted by raising or lowering the base opposite to correct.

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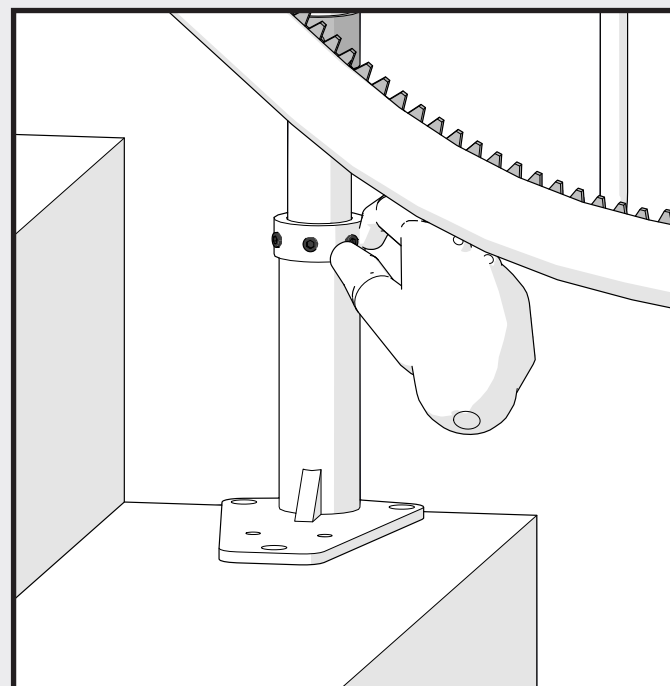
These should be hand tight as there may be further movement once the final fixing is completed.

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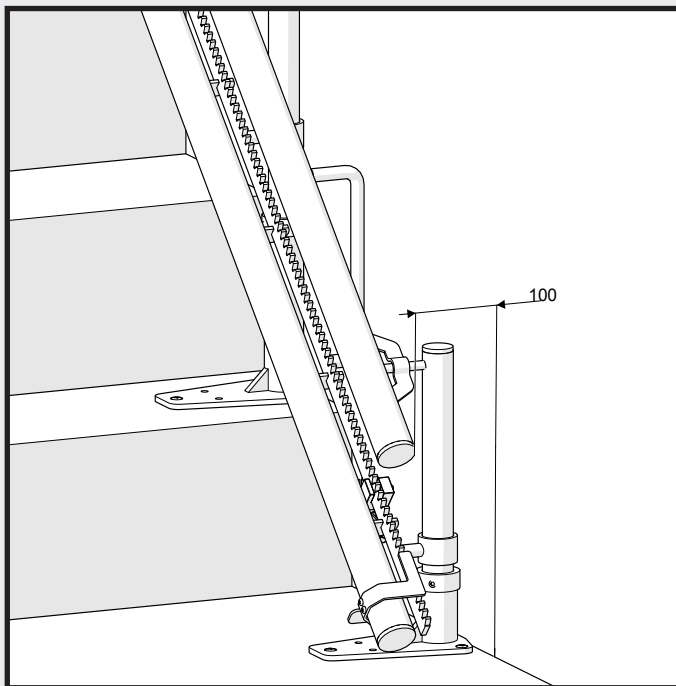
2.8 Adjusting the rail.

Once the rail is level carry out final checks on the position of the rail as in section 2.6 - 2.8.

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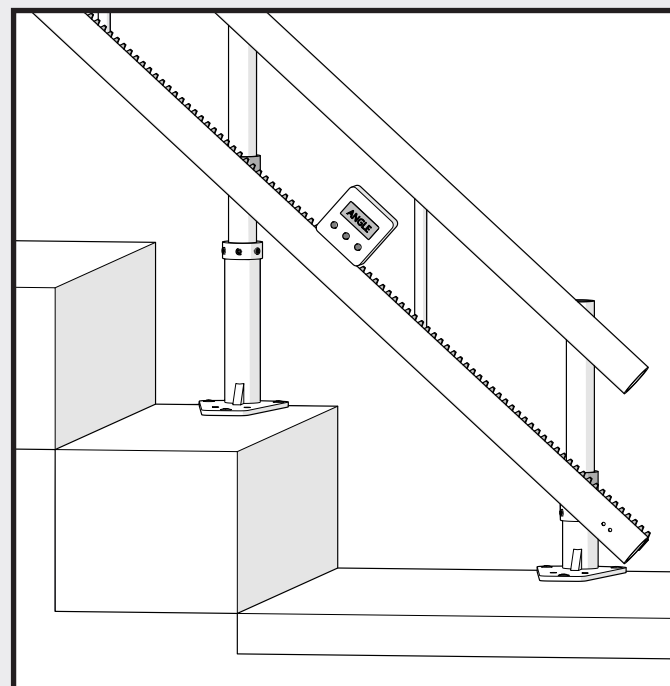
Check the angle corresponds closely with the install drawing, once complete we can fix the rail to the stairs.

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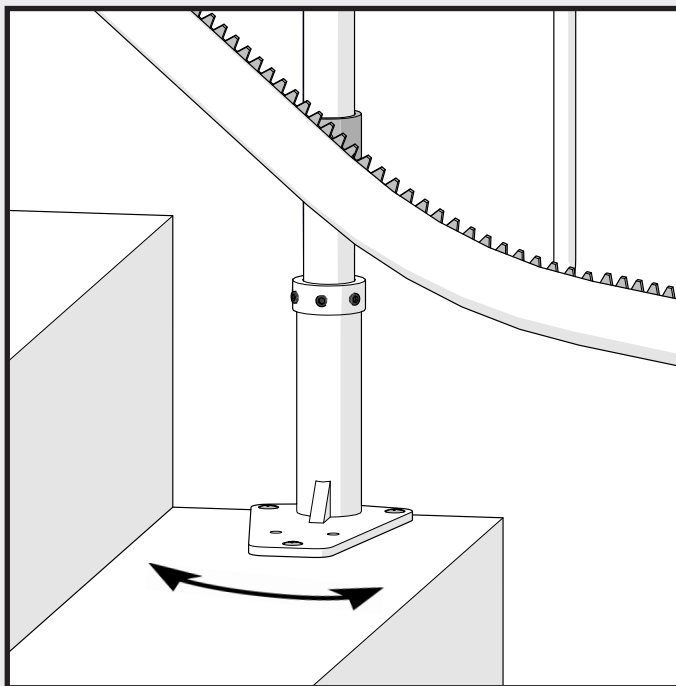
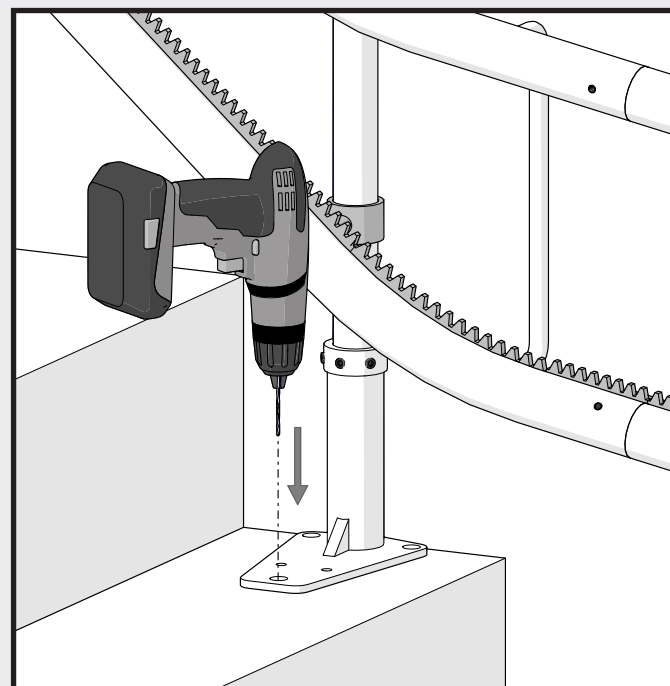
Once the final rail position checks are complete the rail can now be fixed in position. Straighten up the footplates on the step ready for fixing.

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Start fixing at the bottom base. If the stairs are made of a material other than wood drill each of the 3 base holes using a 5.5mm drill bit and insert red rawl plugs. Wood stairs require no drilling.

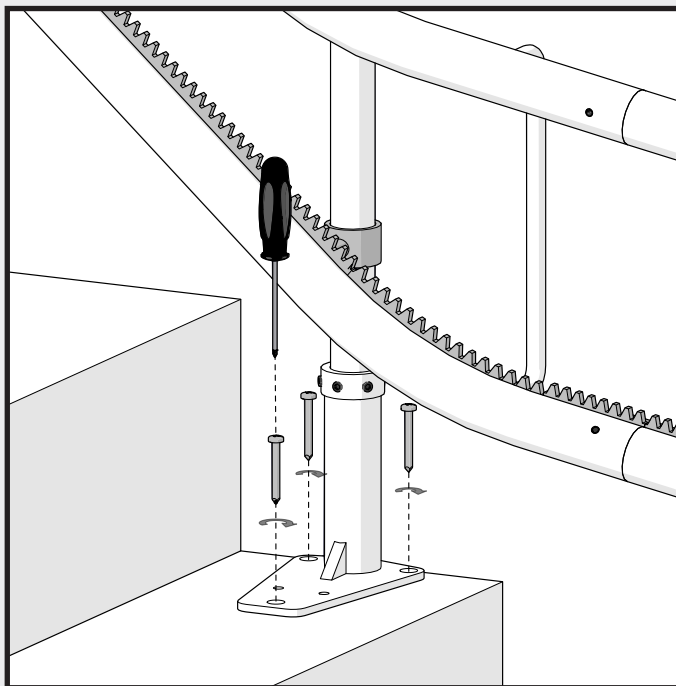
FR Fixation des bases
ES Asegurando las bases
DE Sicherung der Basen
IT Fissaggio delle basi
PT Fixação das bases

2.9.1

29

2.9 Securing the Bases

Fix the base to the stair tread using (3x) M10x50mm screws, ensuring they are fixed neatly.

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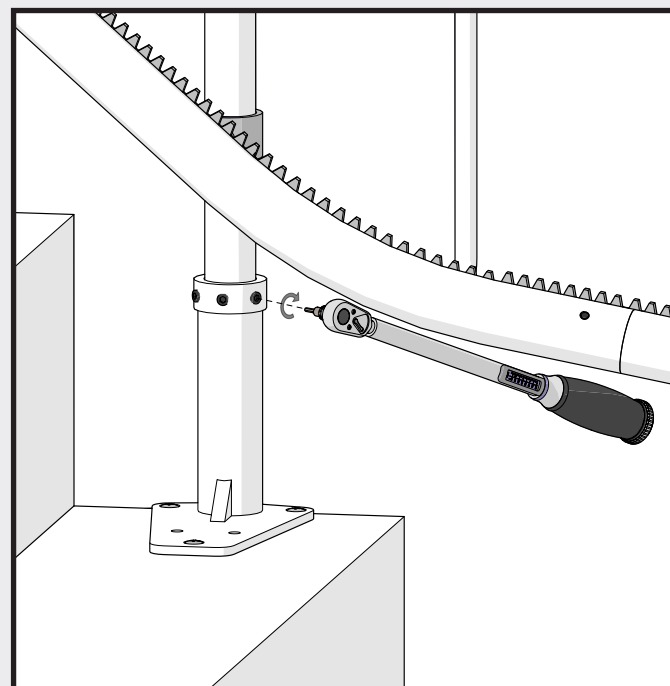
Once the bases are fixed tighten the grub screws to 20Nm. Repeat the process for each base moving up the rail, checking the rail position, until all the bases are complete.

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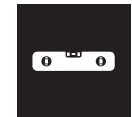
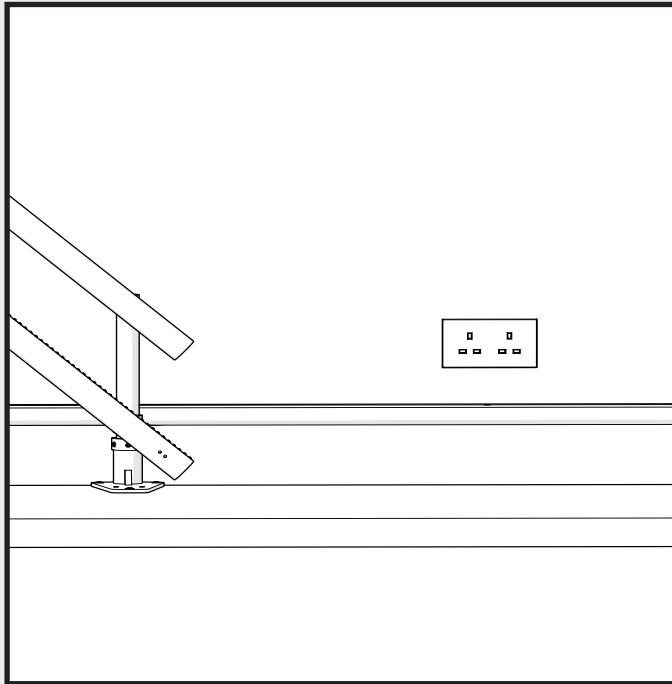
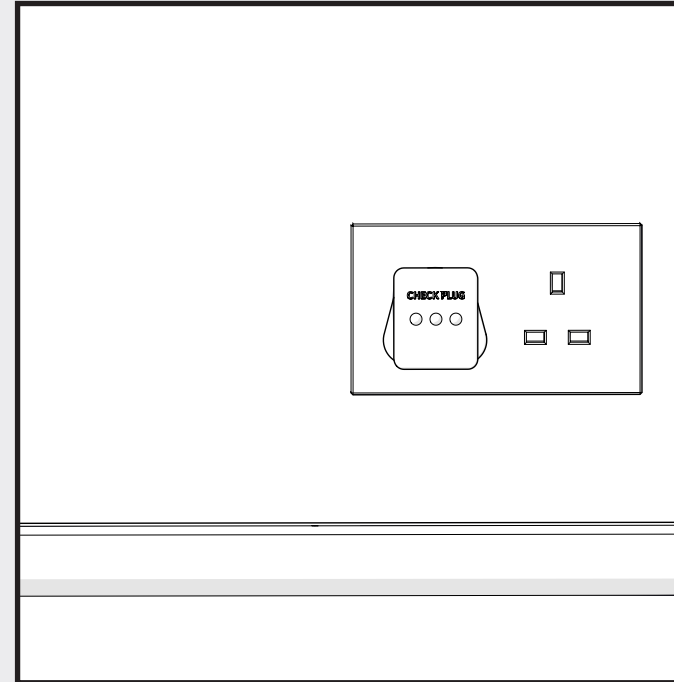
Identify a suitable location for the charger. This needs to be at either the top or the bottom terminations of the rail. Cable must not be run under the carpet or across walkways.

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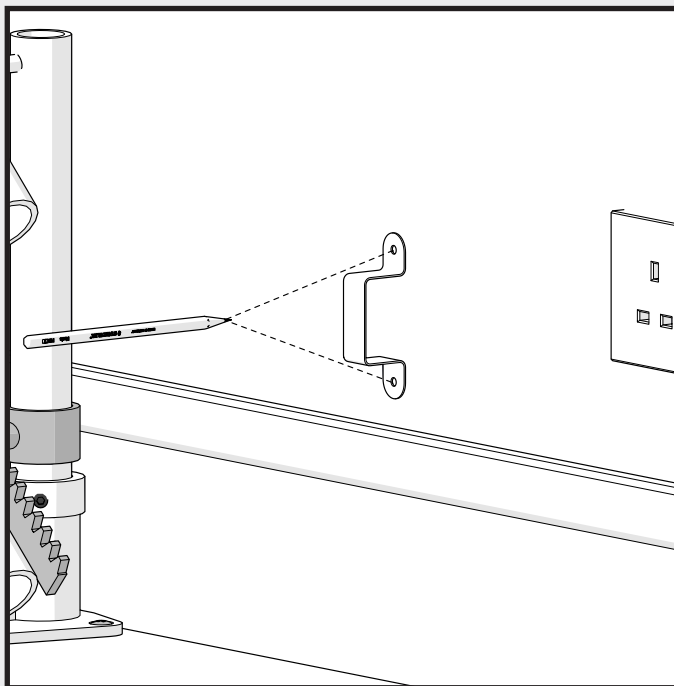
Using a mains socket tester to ensure the site wiring is safe and free from faults. If not suitable please find an alternative power source.

3.2 Mounting the Charger

Once a suitable location is found, take the charger bracket and mark 2 holes with a pencil.

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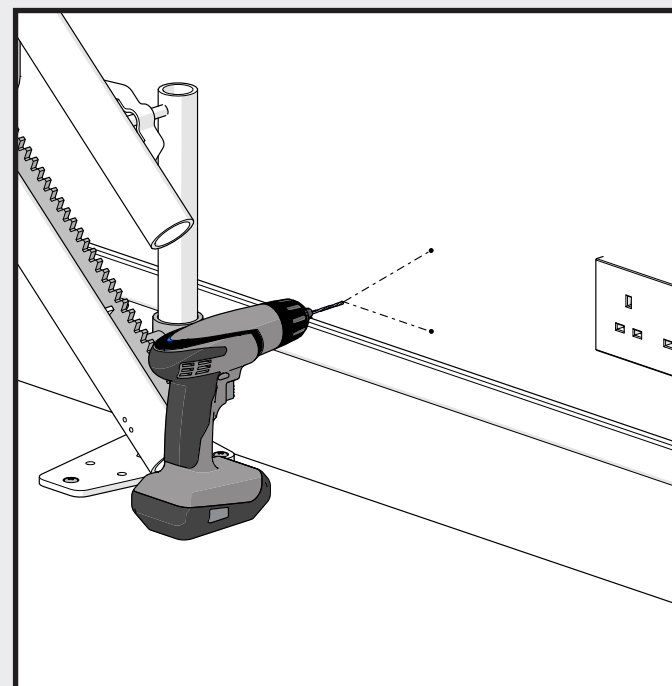
Using a 5.5mm drill bit, drill 2x holes for the wall plugs should they be required. Please use correct fixing methods for the material the charger will be fixed to.

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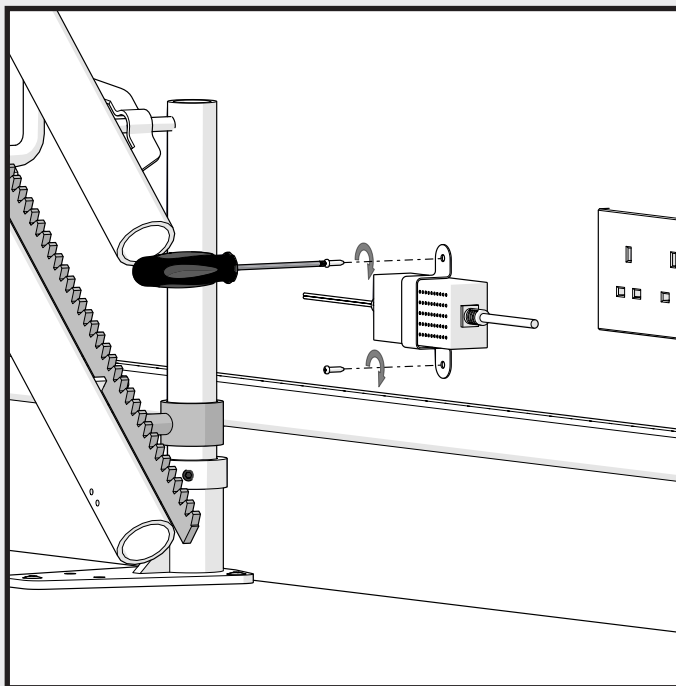
33

3.2 Mounting the Charger

Once the plugs (if required) are inserted. Attach the charger, using the charger bracket, to the wall.

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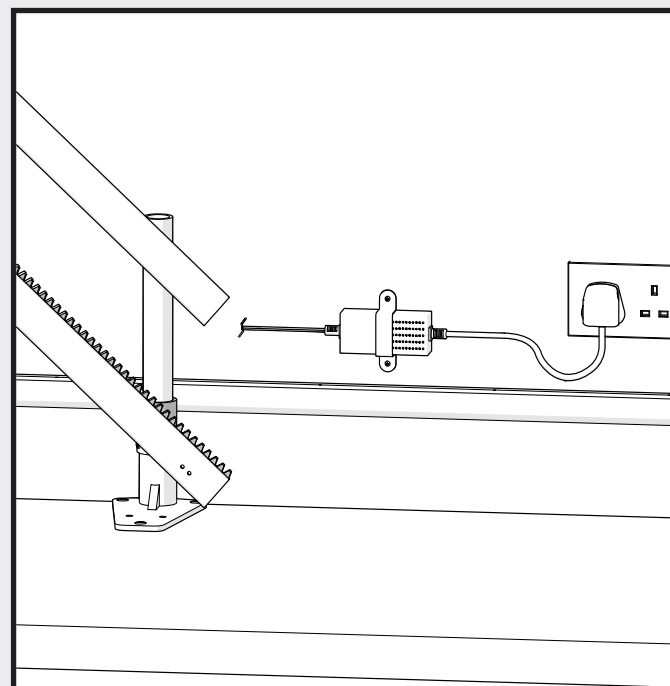
Once attached to the wall DO NOT switch the power on.

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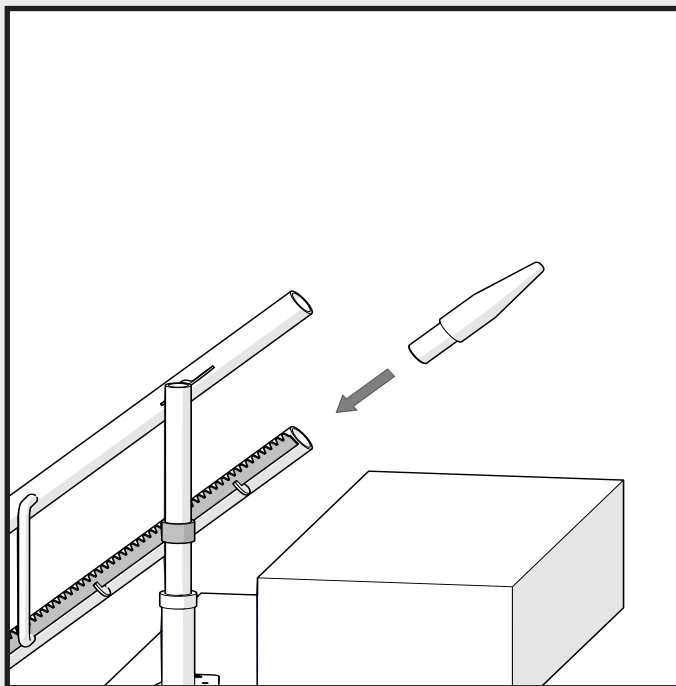
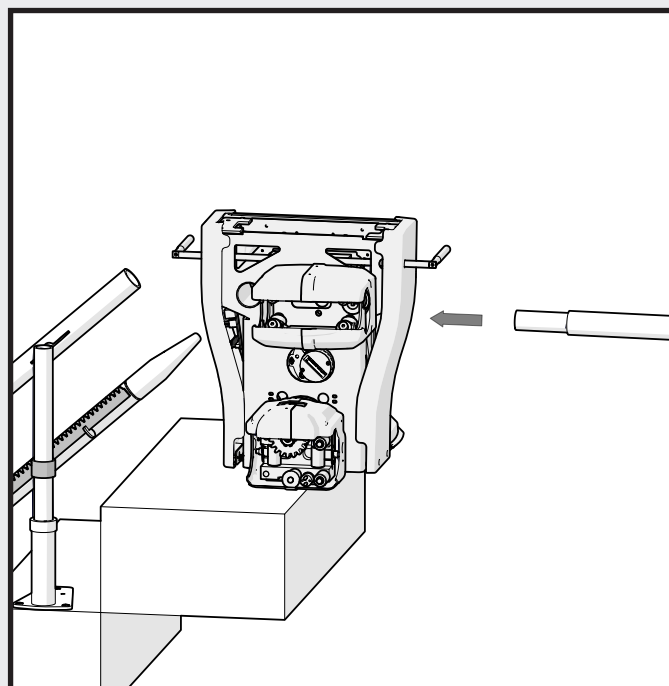
To begin loading, take the carriage to the top of the stairlift along with the loading cone and loading bar. Insert the cone in the bottom tube, this helps load the carriage safely.

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Place the loading bar in the top skate of the carriage. This is to to simplify loading the carriage and to avoid any damage.

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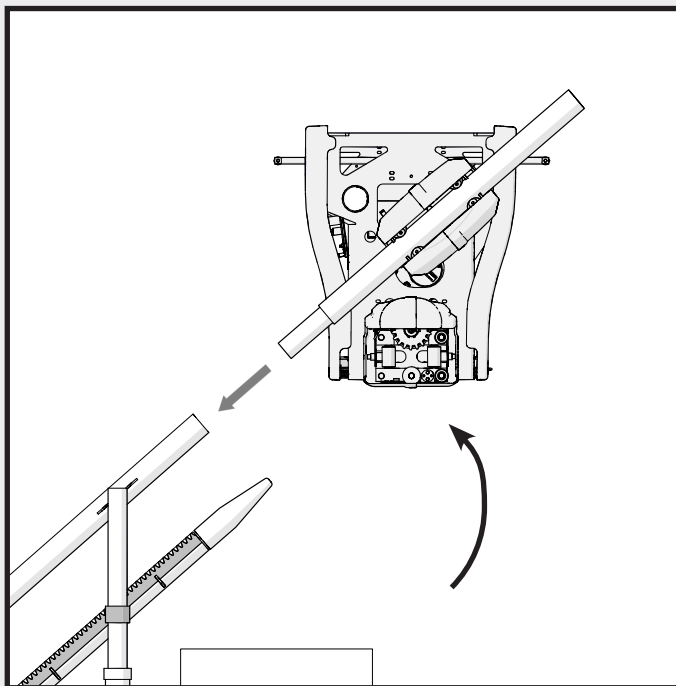
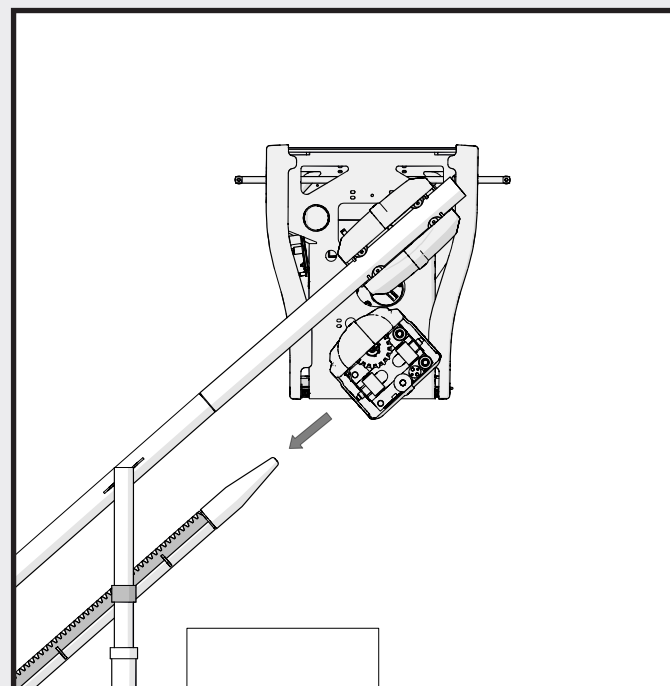
Support the carriage by the loading bar and insert the smaller end into the top tube of the rail. Be careful to avoid the carriage slipping down the loading bar. Pivot the loading bar and locate fully into the top tube.

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With the loading bar inserted, gently guide the carriage and bottom skate onto the loading cone, ensuring correct alignment to avoid damage to the bottom skate.

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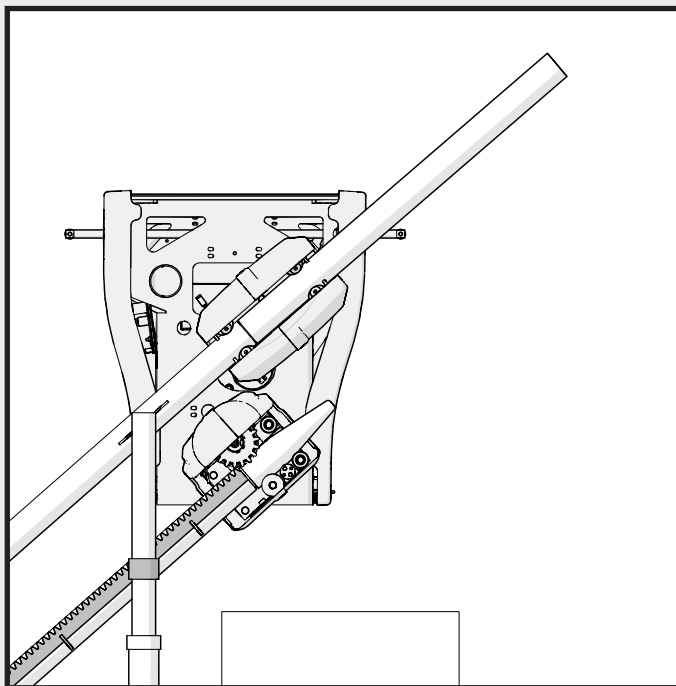
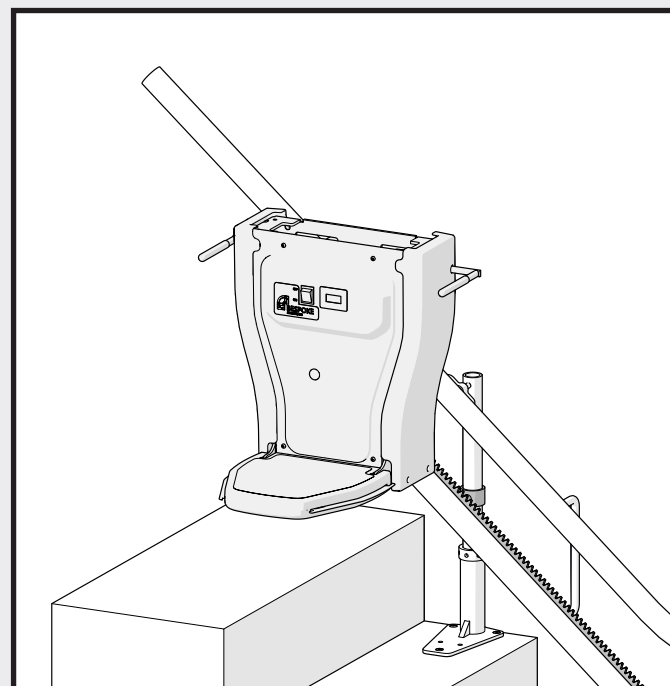
Gently lower the carriage to rest the pinion onto the rack. Keep the loading bar and cone in place.

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The carriage is now loaded onto the rail. To move the carriage the batteries must be inserted.

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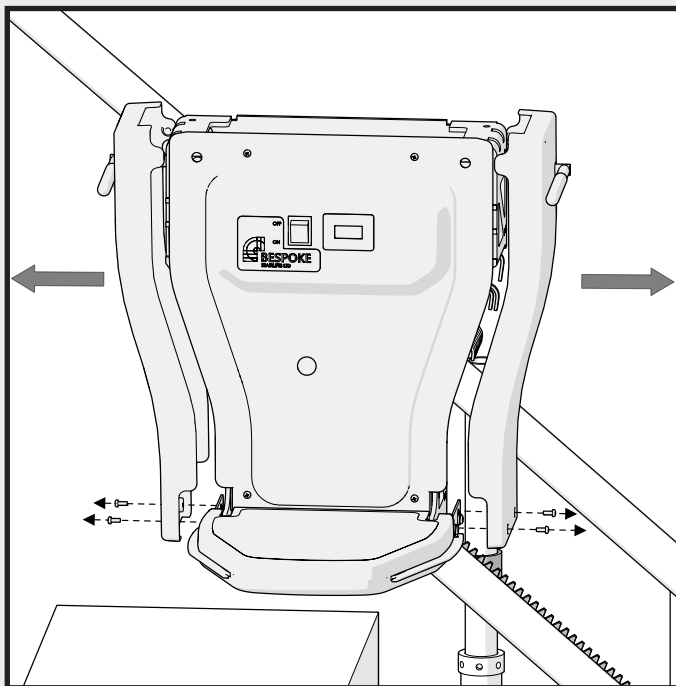
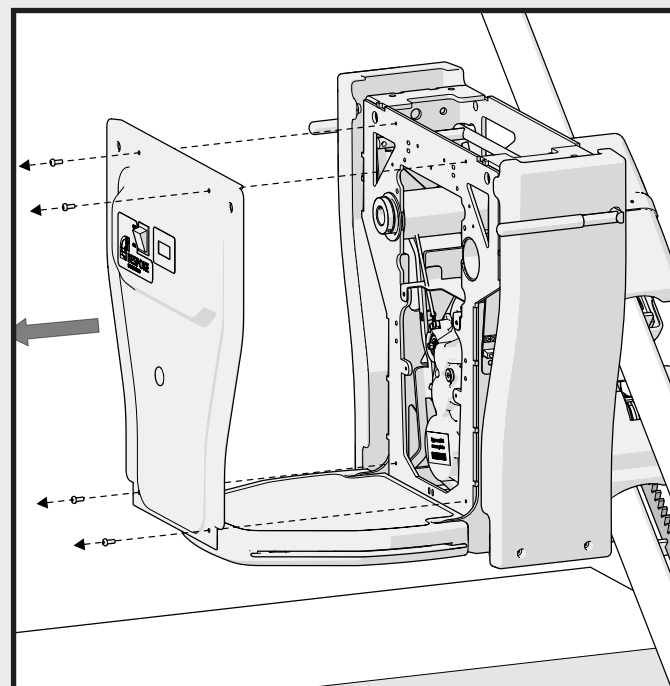
Carefully remove the screws from the side panels and remove. Store carefully to one side

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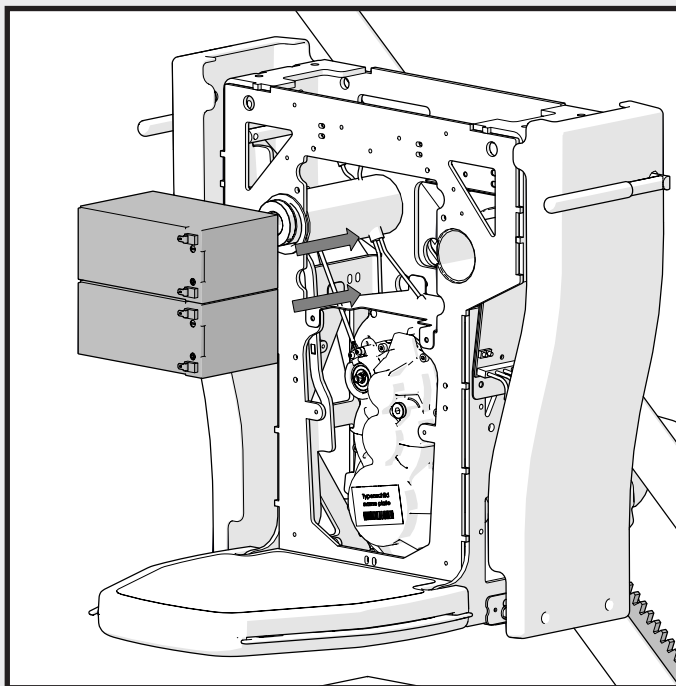
Carefully remove the screws from the front panel to enable the installation of the batteries.

FR Insertion des Piles
ES Insertando las Baterías
DE Einlegen der Batterien
IT Inserimento delle Batterie
PT Inserindo as Baterias

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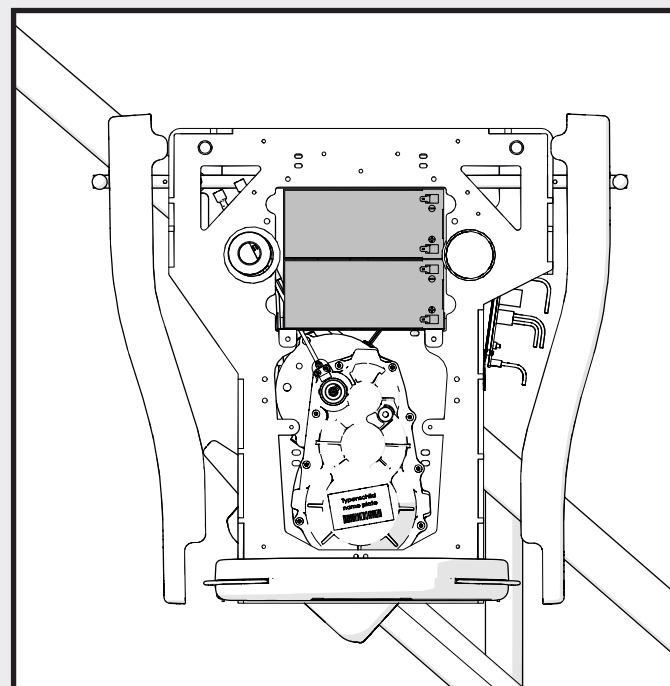
Carefully insert the batteries as shown into the battery tray, being careful not to damage any cables.

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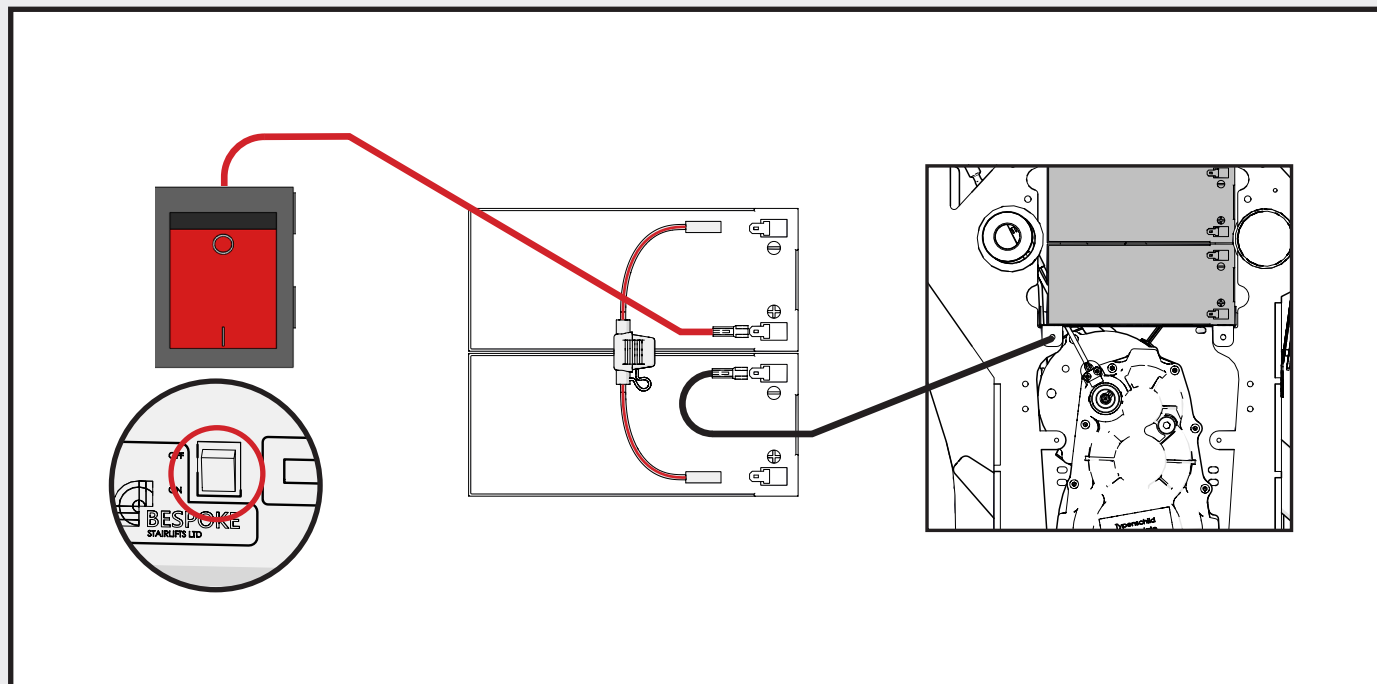
Once the batteries are in the correct position, they can now be connected to the carriage.

4.2 Connecting the Batteries

Connect the red cable from the main power switch to the '+' terminal on the top battery. Connect the 'Earth' cable from the chassis to '-' of the bottom battery. Then connect the fused link to the remaining '+' and '-' terminals on the batteries.

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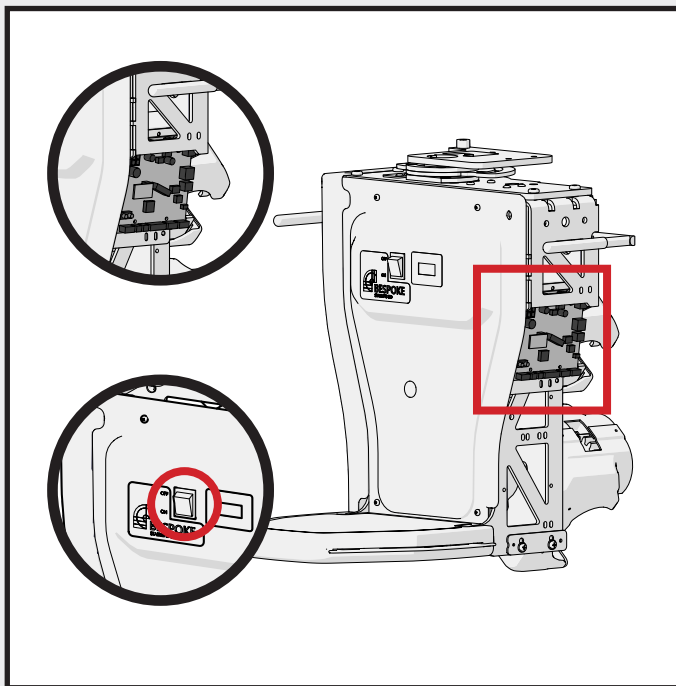
41

4.3 Moving the Carriage

Switch the carriage on and locate the main PCB board.

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The carriage can now be moved using the remote controls.
The remote controls are paired to the lift in the factory and you can now use them to move the carriage up and down the rail.

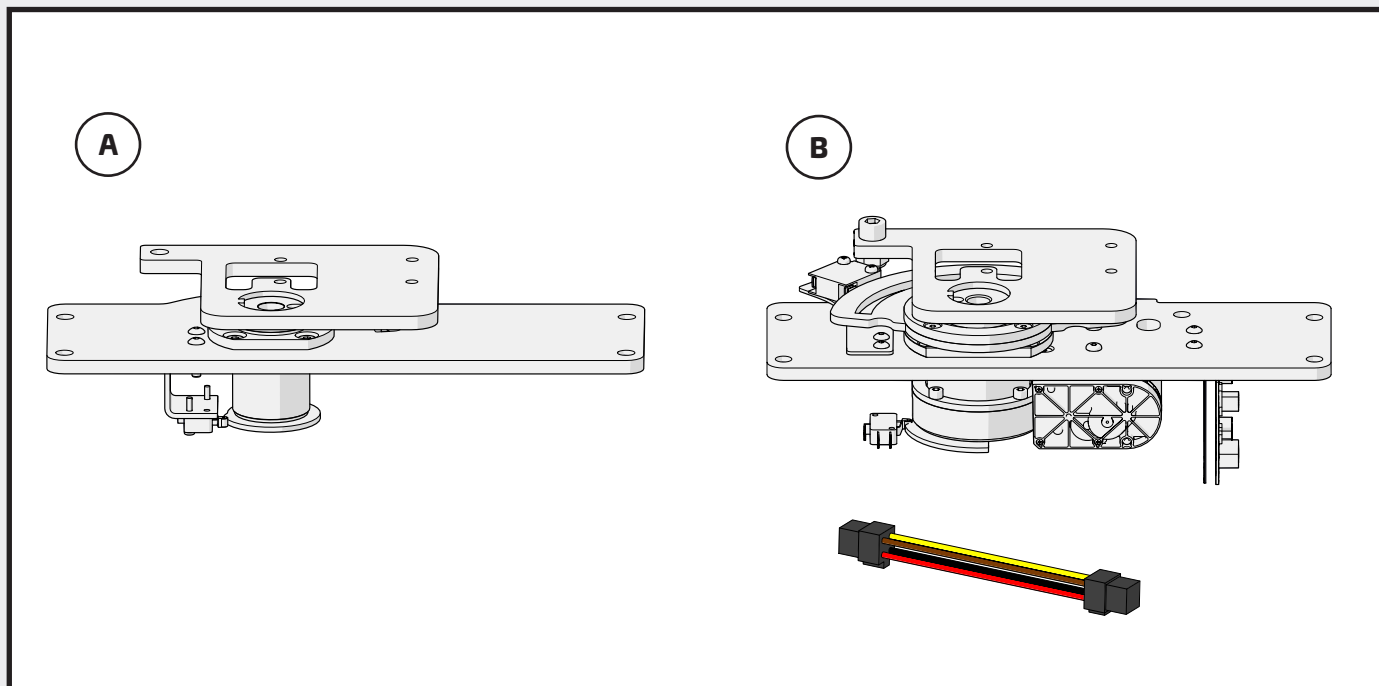
4.3.3

44

FR Déplacement de la Carrosse
ES Moviendo el Carruaje
DE Bewegen der Kutsche
IT Spostamento della Carrozza
PT Movendo a Carruagem

 **BESPOKE**
STAIRLIFTS

4.4 Assembling the Swivel

The next step is to mount the swivel. Select which swivel type and move onto the correct section for installation details.

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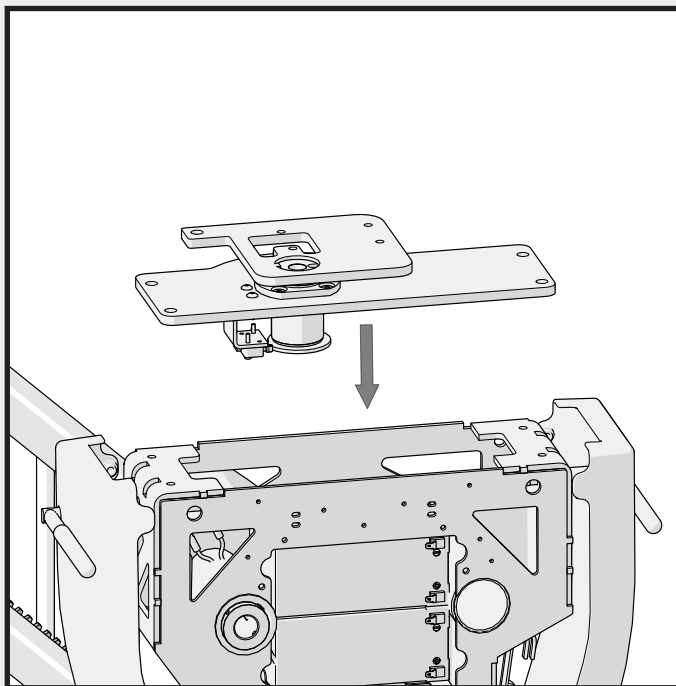
4.4 Assembling the Manual Swivel

Position the Swivel Assembly onto the Carriage

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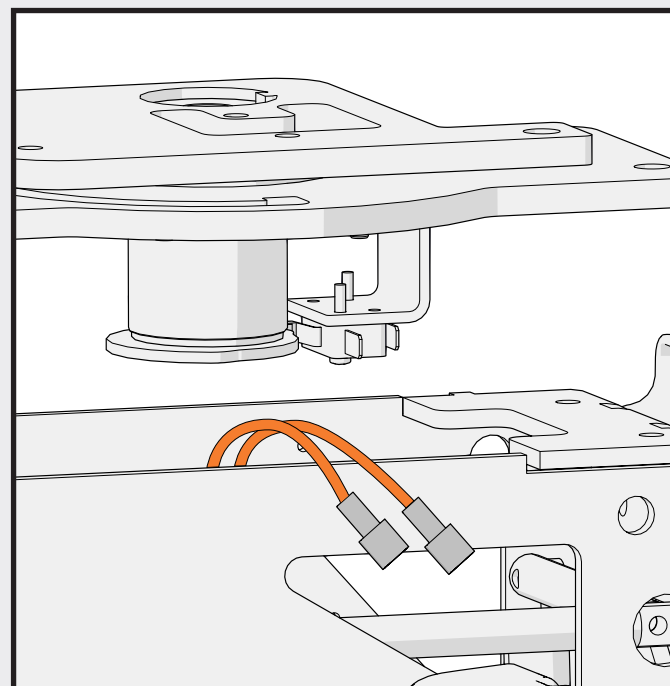
PT Posicione o conjunto de pivô sobre a carruagem.

Identify the two orange connecting cables inside the Carriage

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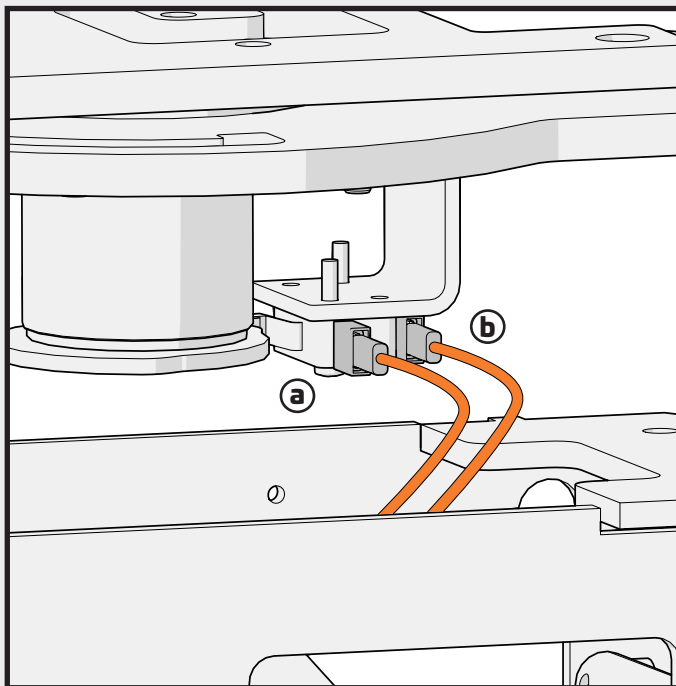
4.4 Assembling the Manual Swivel

Connect the two cable terminals to terminals A and B on the Swivel Assembly Switch.

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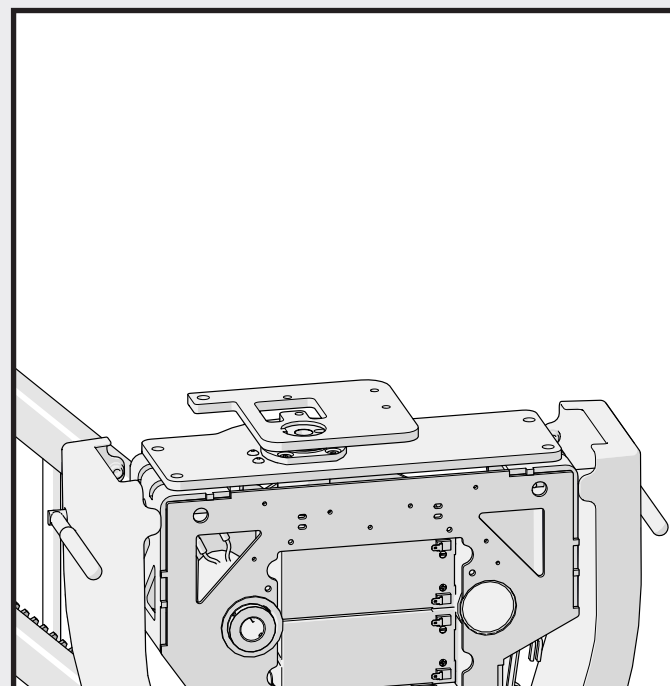
Carefully place the swivel onto the carriage body ensuring the cables are not damaged.

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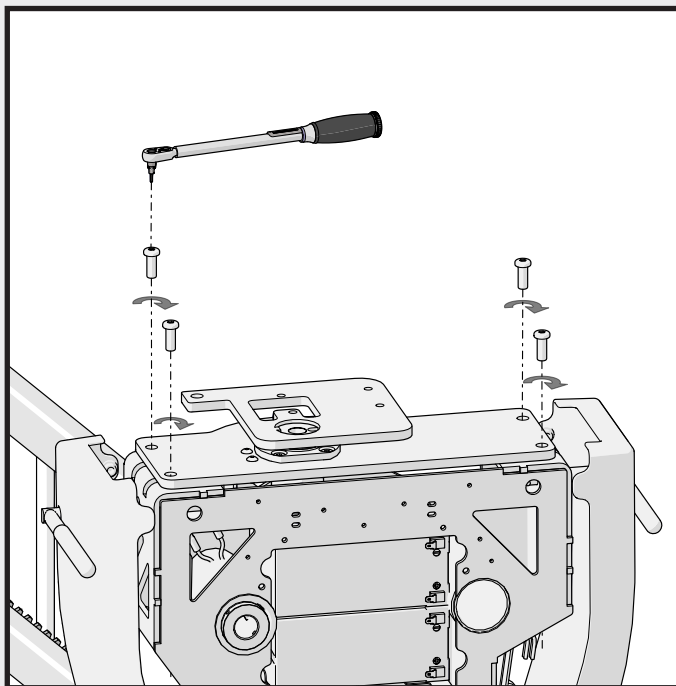
4.4 Assembling the Manual Swivel

Secure the Swivel Assembly to the Carriage with the 4xM8 screws and tighten to 30Nm.

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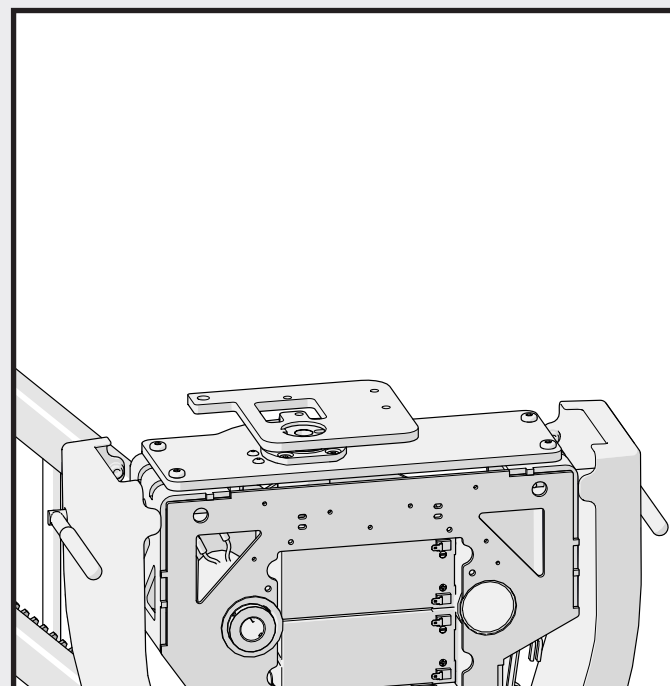
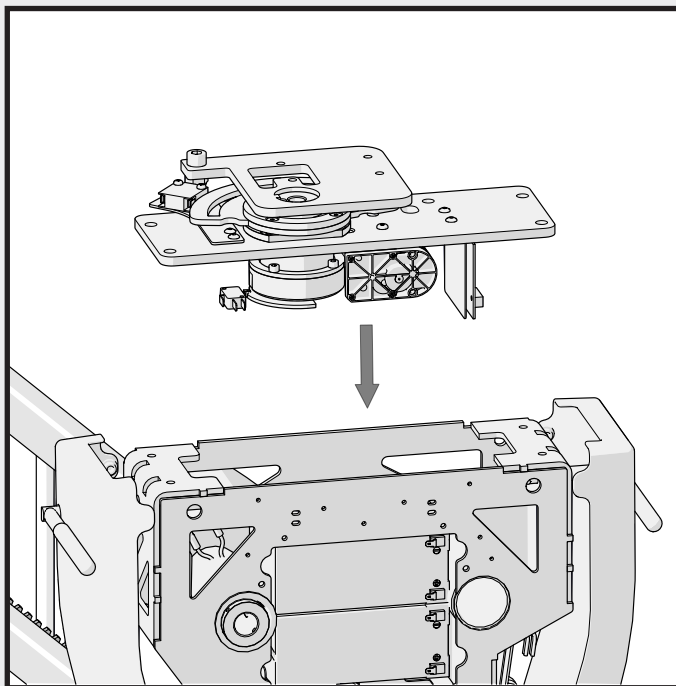
4.5 Assembling the Powered Swivel

Position the Powered Swivel assembly on the carriage.

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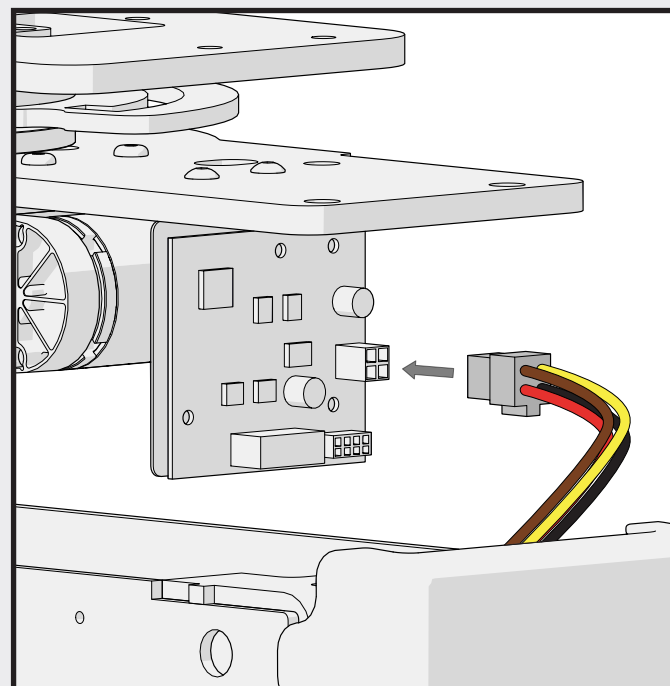
Using the auxiliary cable provided, connect to the powered swivel board.

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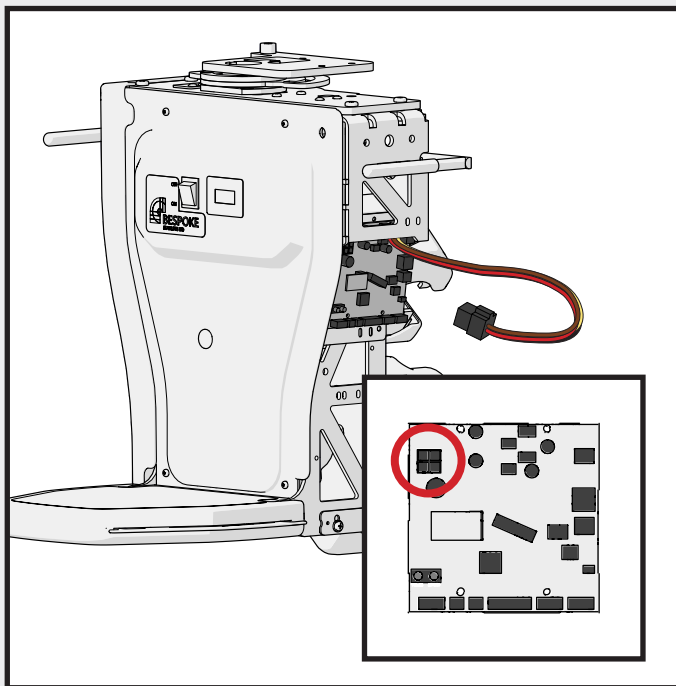
49

4.5 Assembling the Powered Swivel

Connect the other end of the auxiliary cable to the main PCB AUX connection.

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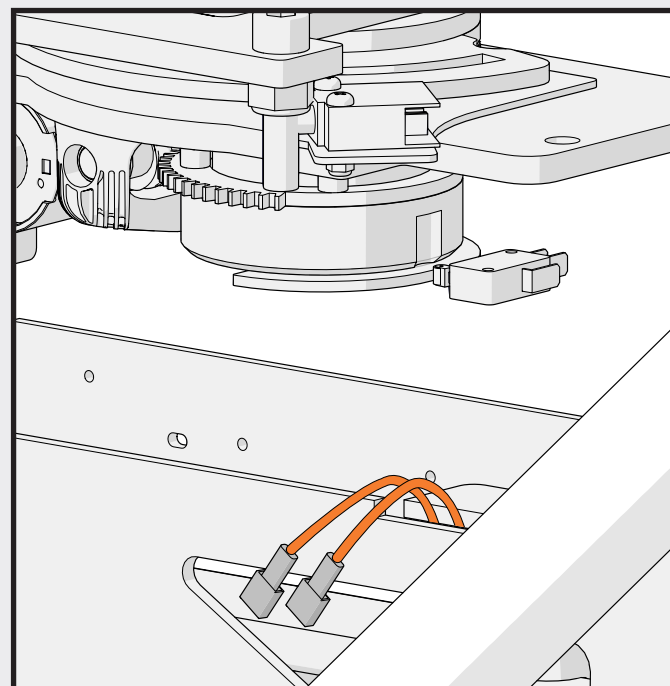
Identify the two orange connecting cables inside the Carriage.

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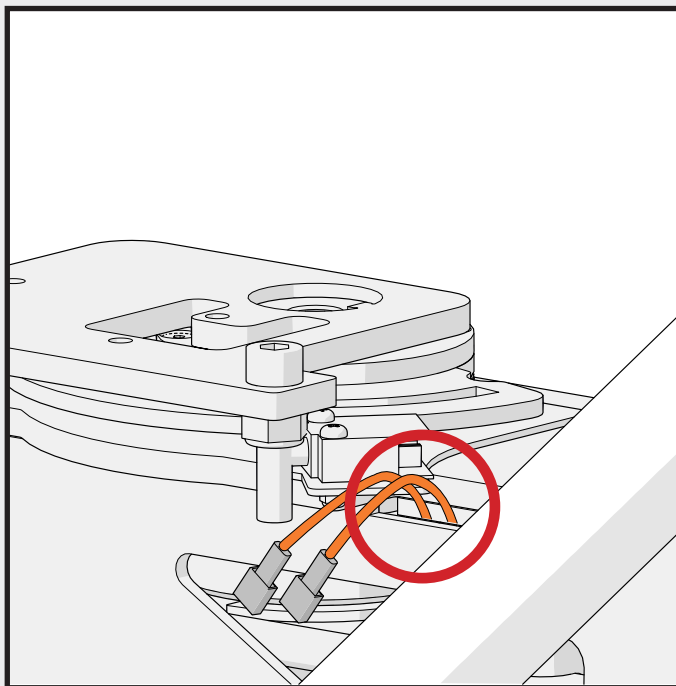
4.5 Assembling the Powered Swivel

Lower the powered swivel unit onto the chassis frame. Carefully position the orange cables in the chassis cut-away to avoid damage.

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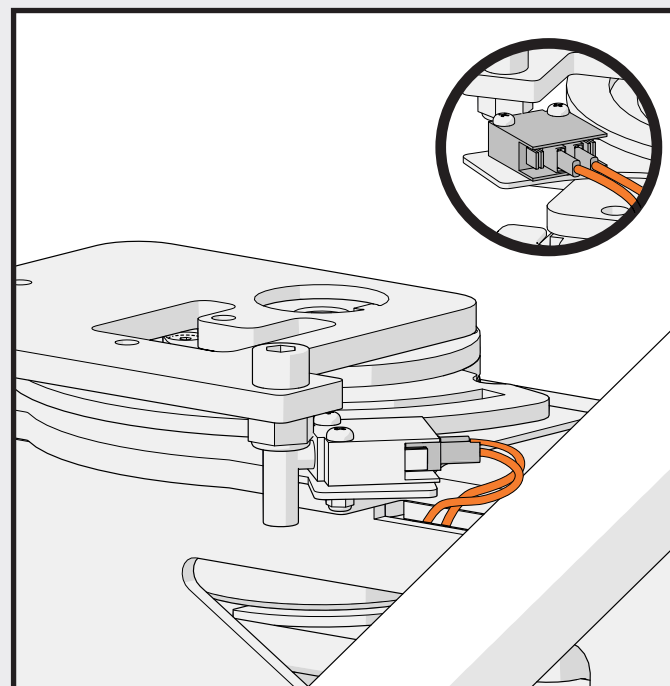
Now connect the orange cables to the powered swivel unit.

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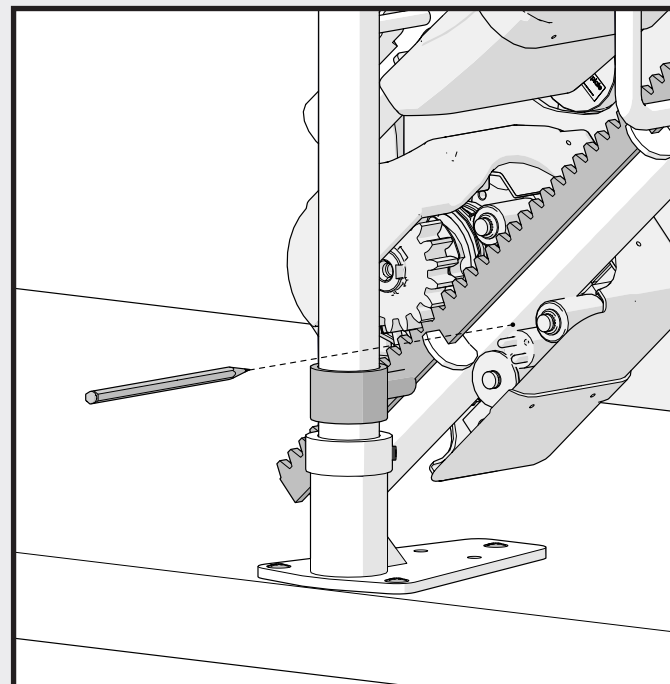
To locate the bottom charge point position use the remote control to drive the carriage to the lower part of the rail. Switch off the carriage.

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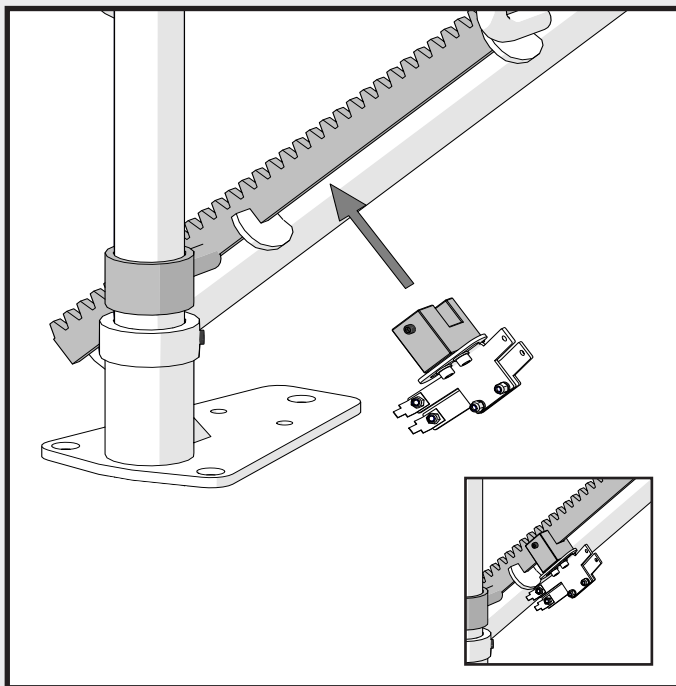
With the carriage in the desired position, stop the carriage and mark the position of the of the charge pins on the bottom skate. Mark this position with a pencil.

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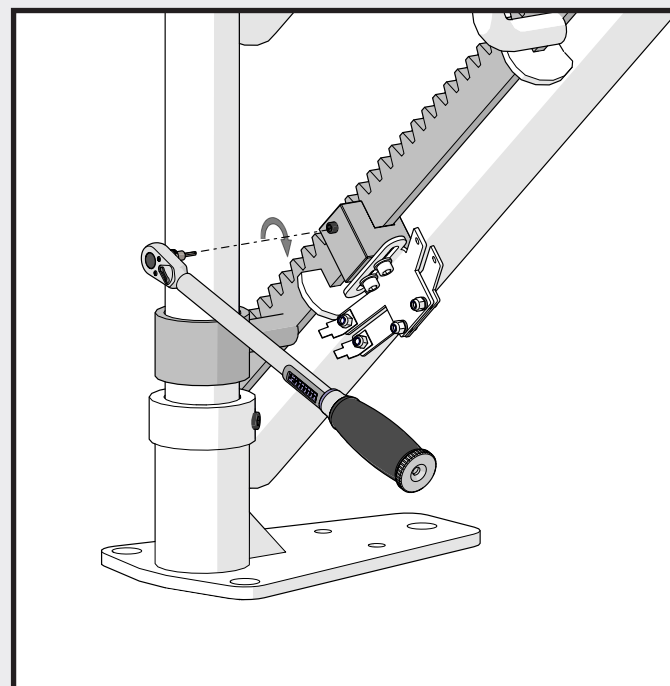
Switch the carriage on and move the carriage up; away from the charge point location. Attach to the racking using the grub screw.

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Tighten the charge point securely at the bottom of the rail to 10Nm.

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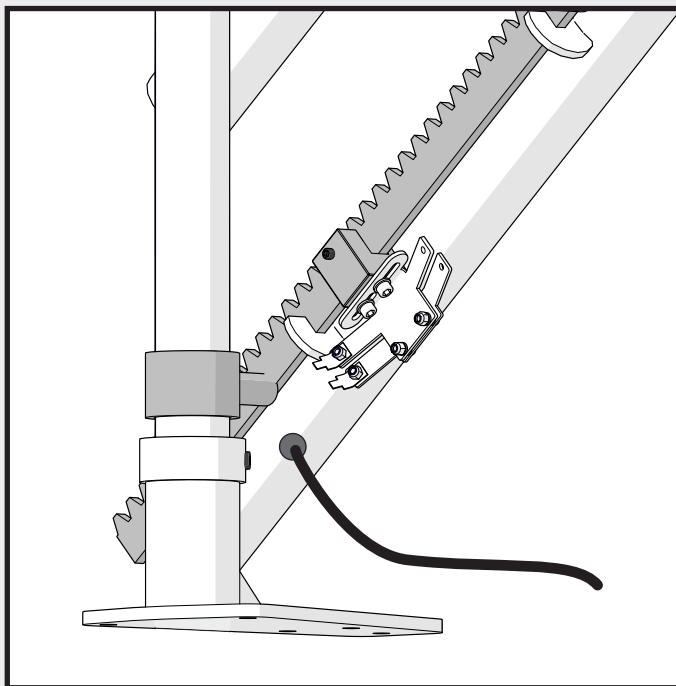
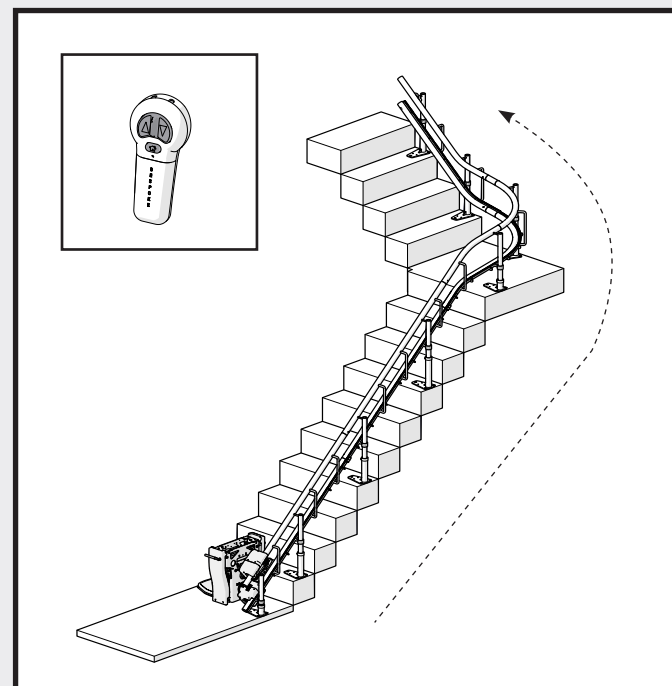
The bottom charge point is now secure. Now feed the cable through the hole located near the bottom of the rail in preparation for connection.

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Move the carriage to the top of the rail to the desired terminal location. Switch off the carriage.

5.1.3

58

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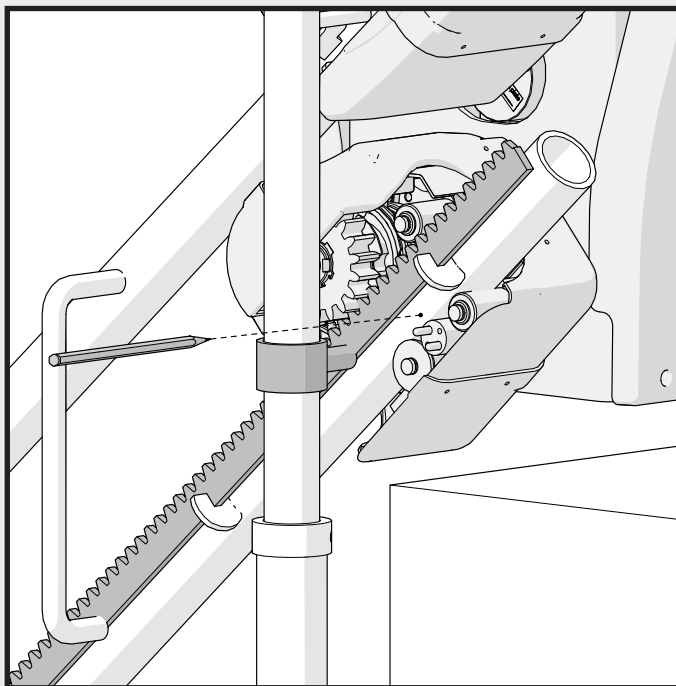
DE Unterer Ladepunkt

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PT Ponto de carga inferior

5.2 Top Charge Point

With the carriage in the desired position mark the position of the charge pins located on the bottom skate. Mark this position with a pencil.

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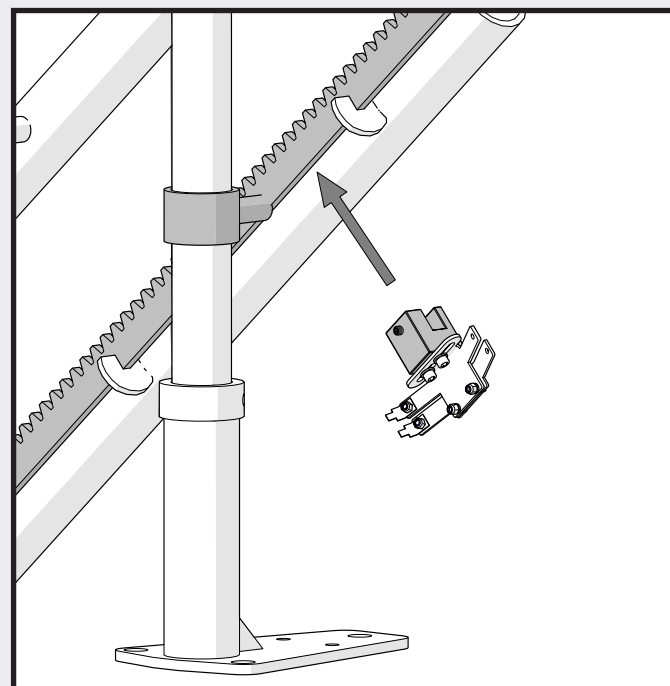
Switch the carriage on and move the carriage down, away from the charge point location. Attach to the racking using the grub screw.

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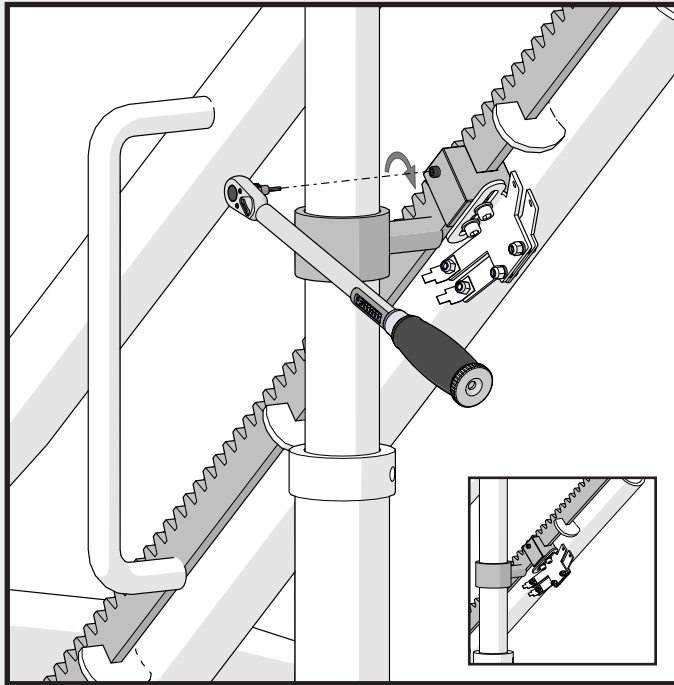
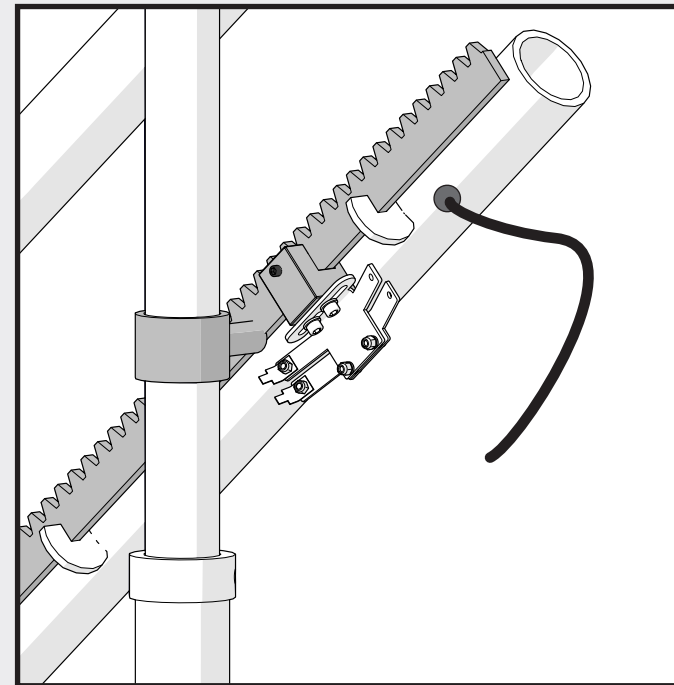
Tighten the charge point securely to the bottom rail, to 10Nm.

- FR** Le point de charge supérieur est maintenant sécurisé. Alimentez maintenant le câble à travers le trou situé près du haut du rail en préparation pour la connexion.
- ES** El punto de carga superior ya está asegurado. Ahora pase el cable a través del agujero ubicado cerca de la parte superior del riel en preparación para la conexión.
- DE** Der obere Ladepunkt ist nun gesichert. Führen Sie nun das Kabel durch das Loch, das sich nahe dem oberen Ende der Schiene befindet, zur Vorbereitung der Verbindung.
- IT** Il punto di carica superiore è ora sicuro. Ora inserire il cavo attraverso il foro situato vicino alla parte superiore della guida in preparazione per la connessione.
- PT** O ponto de carga superior está agora seguro. Agora, passe o cabo pelo buraco localizado perto da parte superior do trilho, em preparação para a conexão.

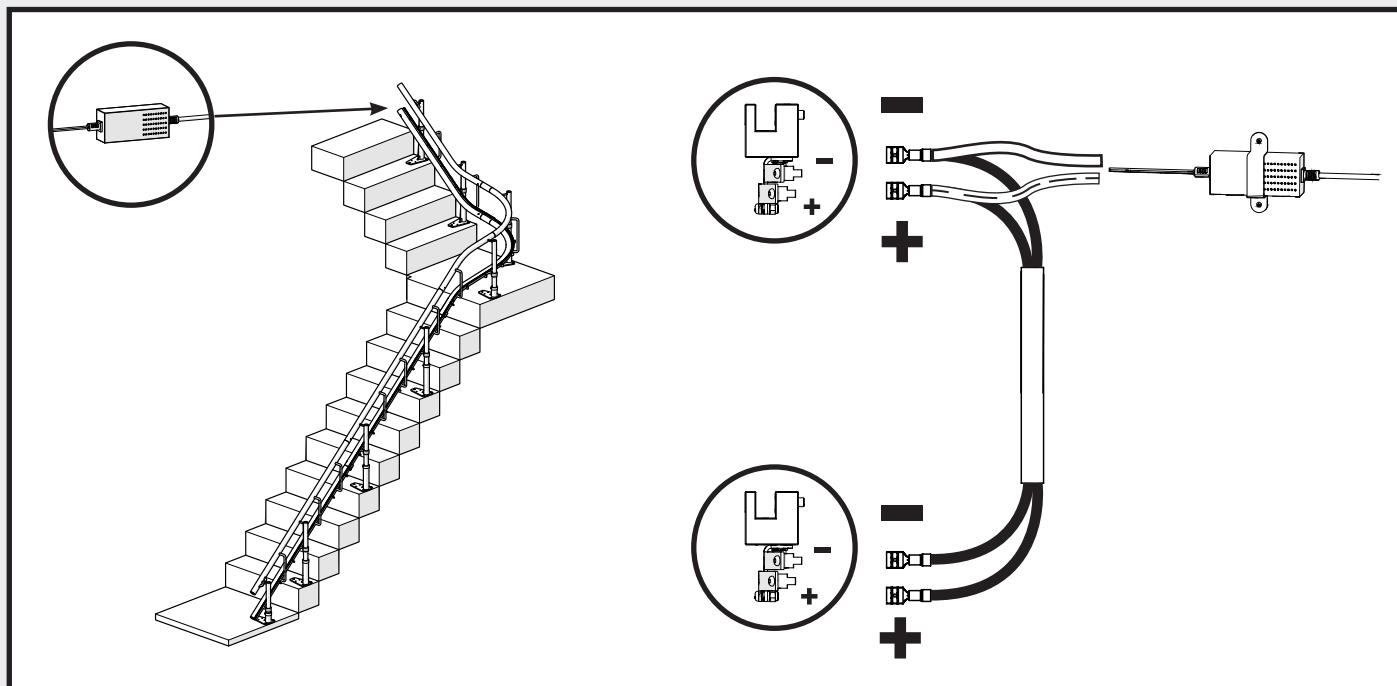
The top charge point is now secure. Now feed the cable through the hole in located near the top of the rail in preparation for connection.

5.2.2

60

FR Point de charge supérieur
ES Punto de carga superior
DE Oberer Ladepunkt
IT Punto di carica superiore
PT Ponto de carga superior

5.3 Top Charger Wiring

If the charger is connected to a power supply on the upper level of the stairlift use the diagram to correctly connect the power charger to the charge points.

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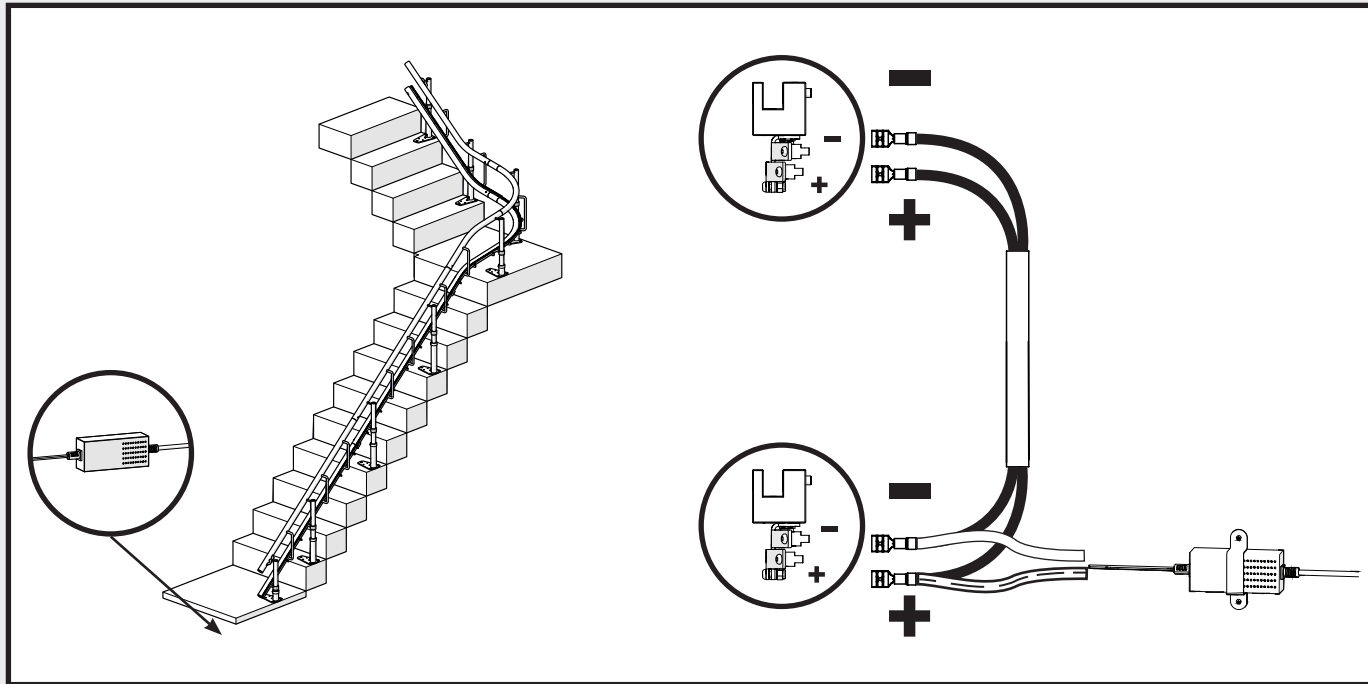
5.4 Bottom Charger Wiring

If the charger is connected to a power supply on the lower level of the stairlift use the diagram to correctly connect the power charger to the charge points.

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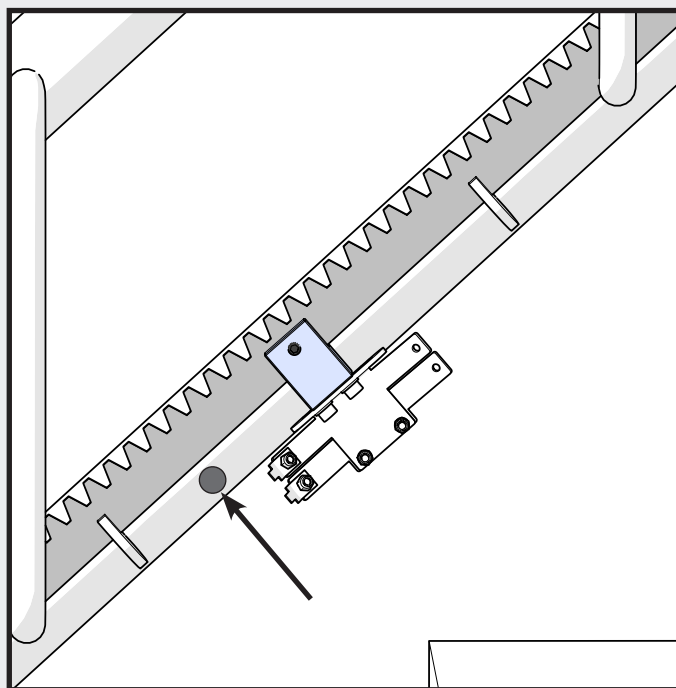
5.5 Intermediate Charge Point

If the rail has an intermediate charge point to park the Stairlift in a position other than the top or bottom, locate the pre-drilled hole at the location specified on the installation drawing. Attach the charge point using the provided grub screw.

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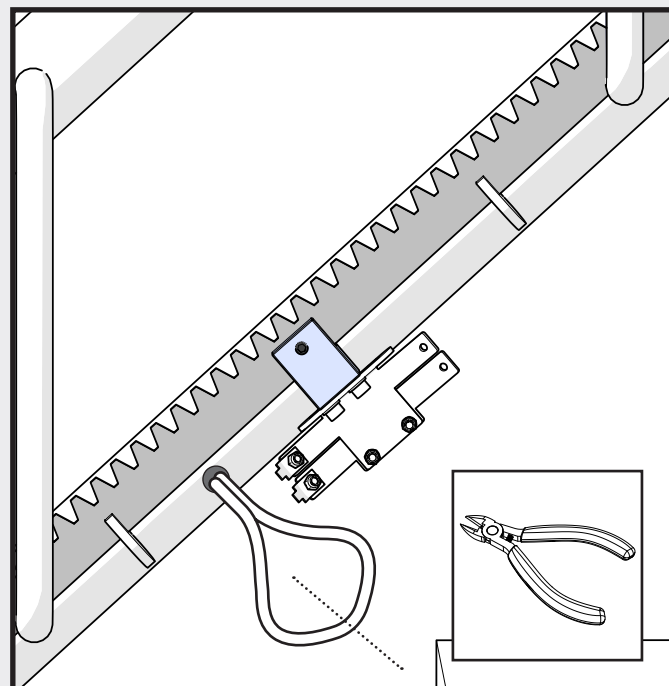
Feed the cable through the hole, ensuring enough cable is available to connect to the charge point.

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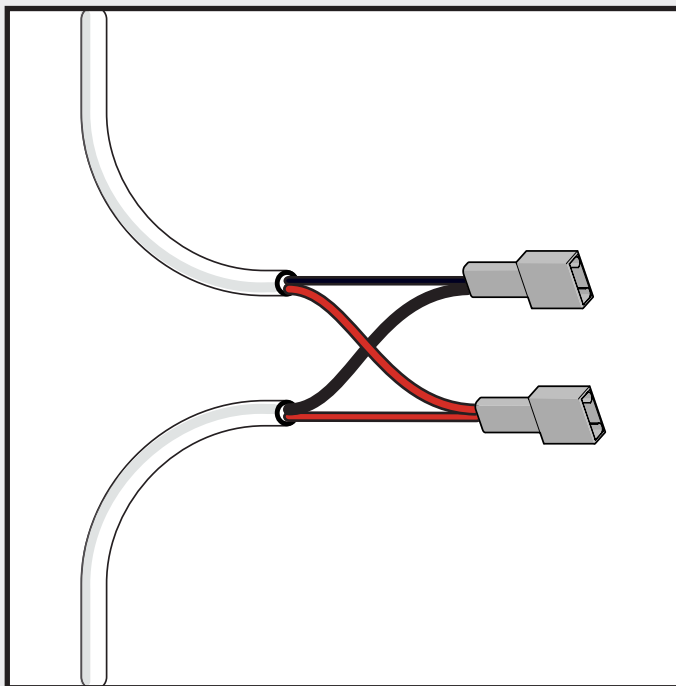
5.5 Intermediate Charge Point

Connect the cables together as shown and crimp into the female spade terminals.

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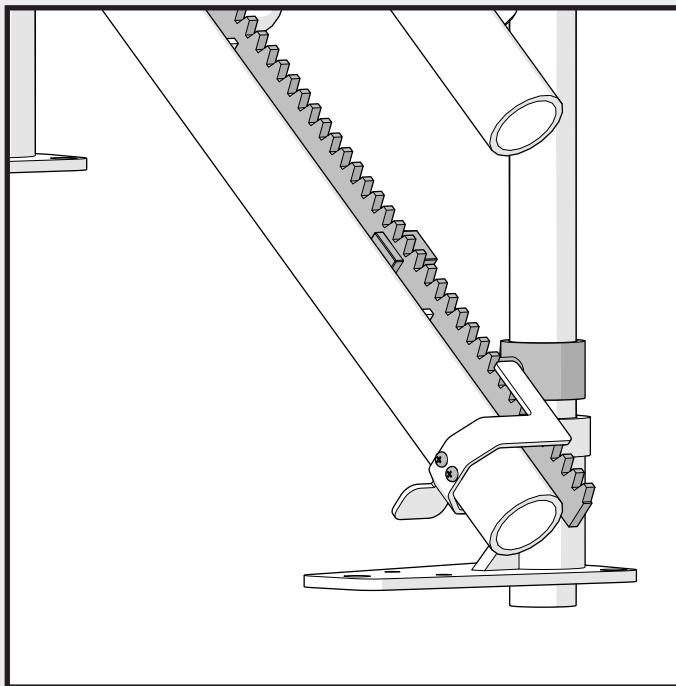
5.6 Bottom End Stop

The bottom end stop fitting is now complete.

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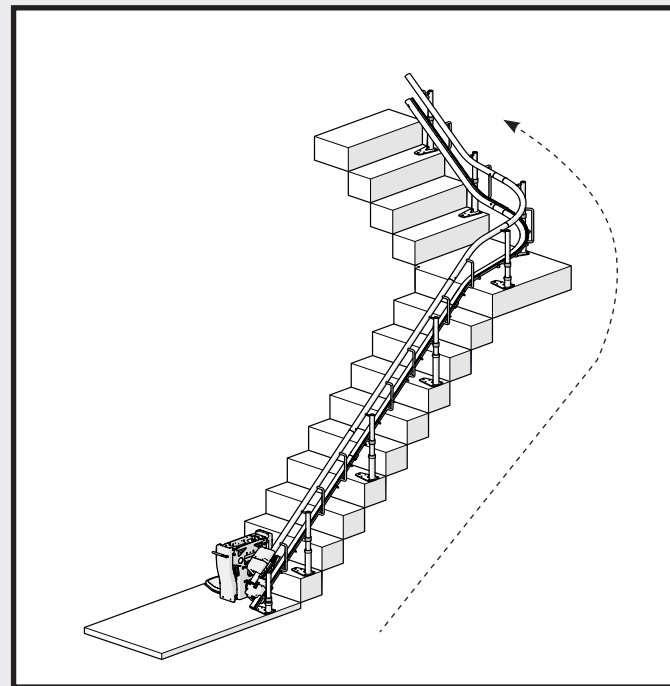
Now run the carriage to the top section of the rail, ready for installing the top end stop.

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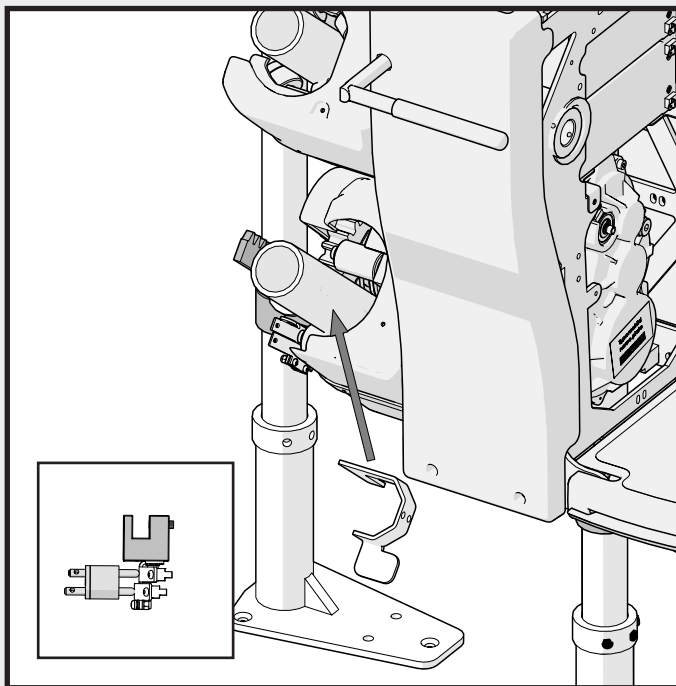
5.6.3
 67

5.7 Top End Stop

Run the carriage upward, so that the charge pins are engaged with the charge point at the top of the rail. Place the end stop in position.

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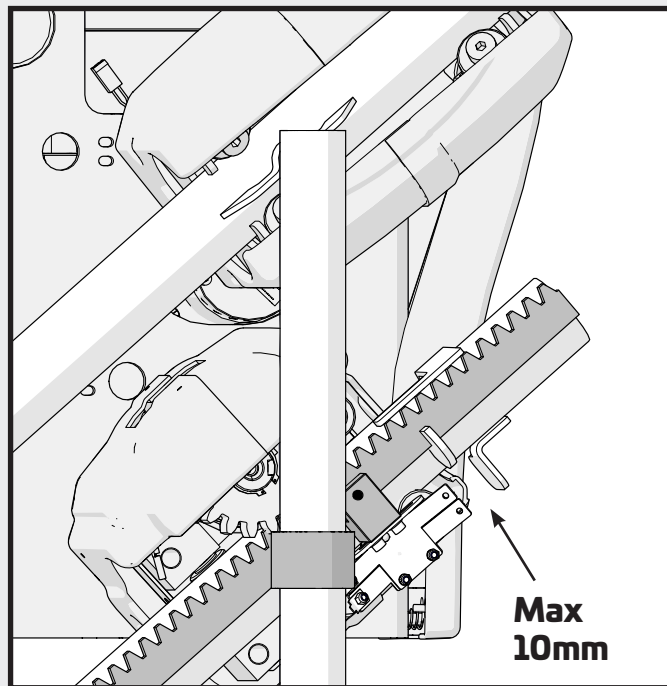
As with the lower end stop, a gap between the end stop and bottom skate covers should be a maximum of 10mm. This is to ensure the carriage will still charge should the carriage slightly overrun.

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
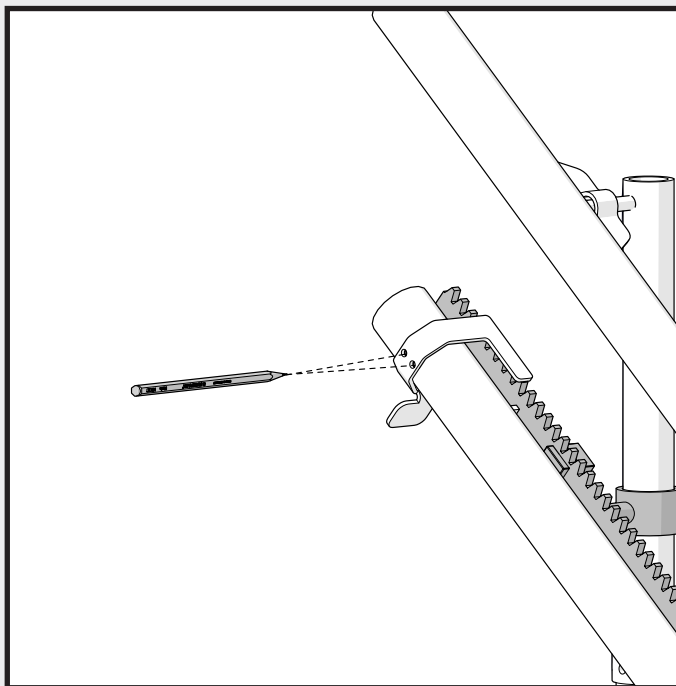
 **BESPOKE**
 STAIRLIFTS

5.7 Top End Stop

Mark the the position of the upper end stop in the correct location.

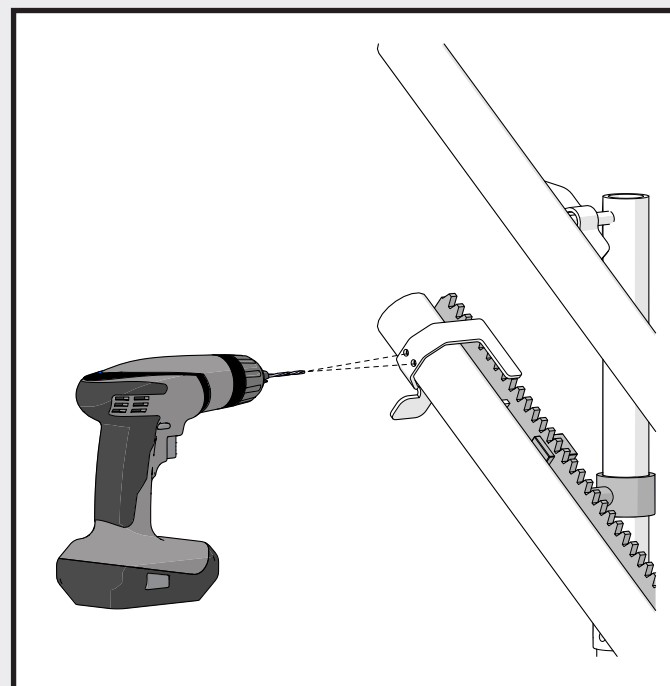- FR** Marquez la position de la butée de fin de course supérieure à l'emplacement correct.
- ES** Marque la posición del tope final superior en la ubicación correcta.
- DE** Markieren Sie die Position des oberen Endanschlags an der richtigen Stelle.
- IT** Segna la posizione dell'arresto di fine corsa superiore nel punto corretto.
- PT** Marque a posição do batente final superior no local correto.

Using a 4mm drill, create a pilot hole for the fixing screws.

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
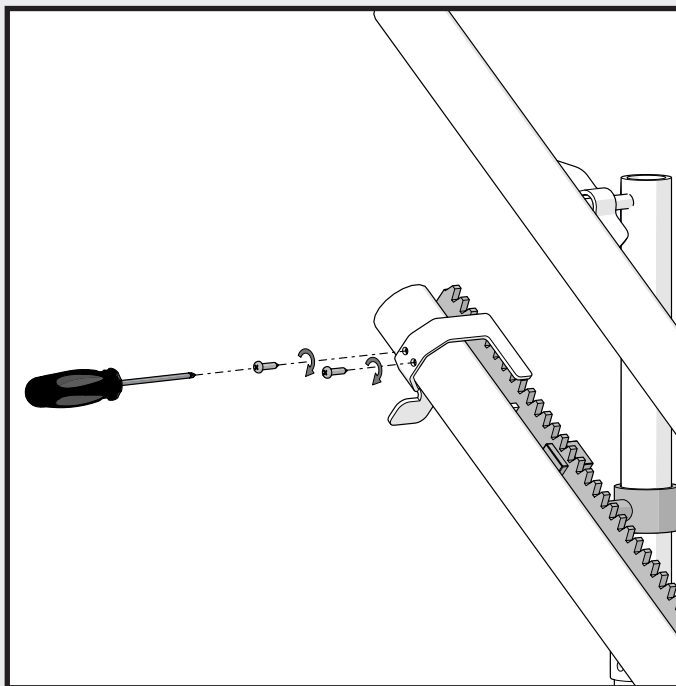
69

5.8 Top End Stop

Attach the end stop using the self tapping screws provided.

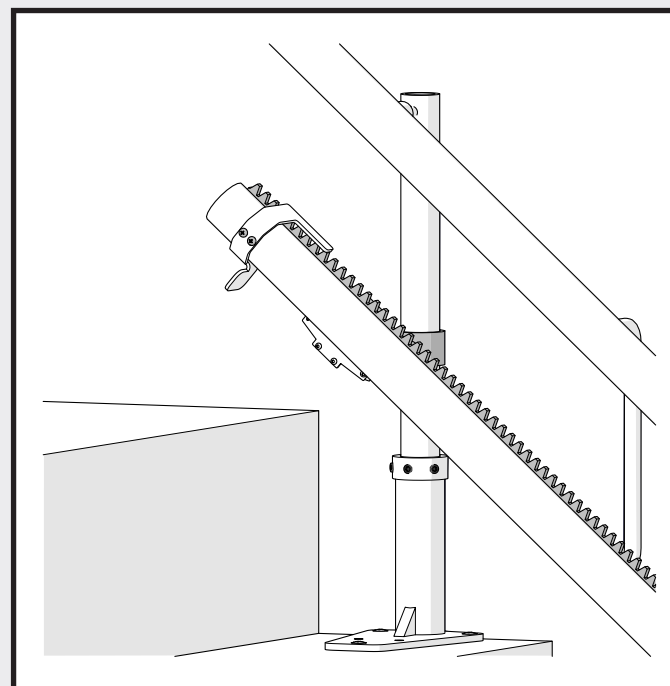- FR** Fixez la butée de fin de course à l'aide des vis autotaraudeuses fournies.
- ES** Fije el tope final usando los tornillos auto perforantes proporcionados.
- DE** Befestigen Sie den Endanschlag mit den mitgelieferten selbstschneidenden Schrauben.
- IT** Fissare l'arresto di fine corsa utilizzando le viti autofilettanti fornite.
- PT** Prenda o batente final usando os parafusos autoatarraxantes fornecidos.

The top end stop fitting is now complete.

- FR** L'installation de la butée de fin de course supérieure est maintenant terminée.
- ES** La instalación del tope final superior está ahora completa.
- DE** Die Montage des oberen Endanschlags ist nun abgeschlossen.
- IT** Il montaggio dell'arresto di fine corsa superiore è ora completato.
- PT** A instalação do batente final superior está agora concluída.

5.8.1

70

- FR** Butée supérieure
- ES** Tope superior
- DE** Oberer Endanschlag
- IT** Fermo superiore
- PT** Pino de fim superior

SECTION 6.0

Programming

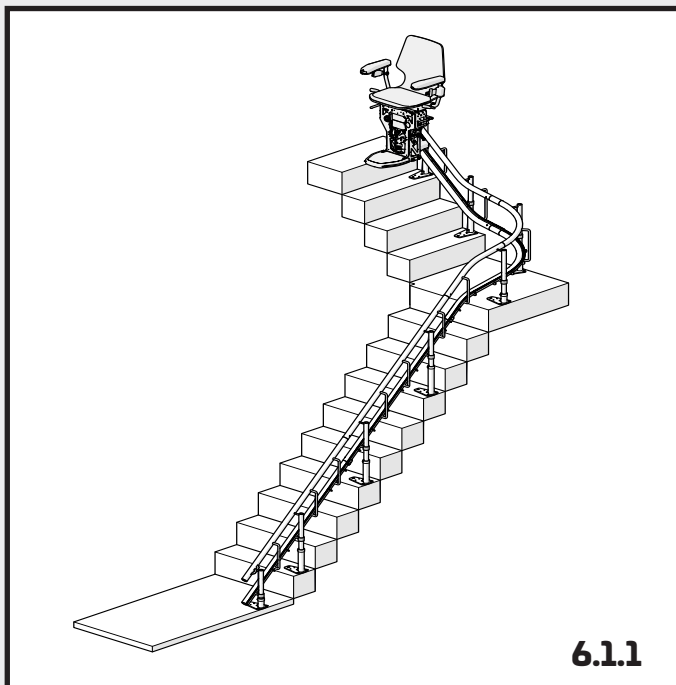
Programmation **FR**
Programación **ES**
Programmierung **DE**
Programmazione **IT**
Programação **PT**

6.0

71

6.1 Programming the Stairlift

6.1.1

With the stairlift at the top, locate the main board on the right side of the carriage section of the lift.

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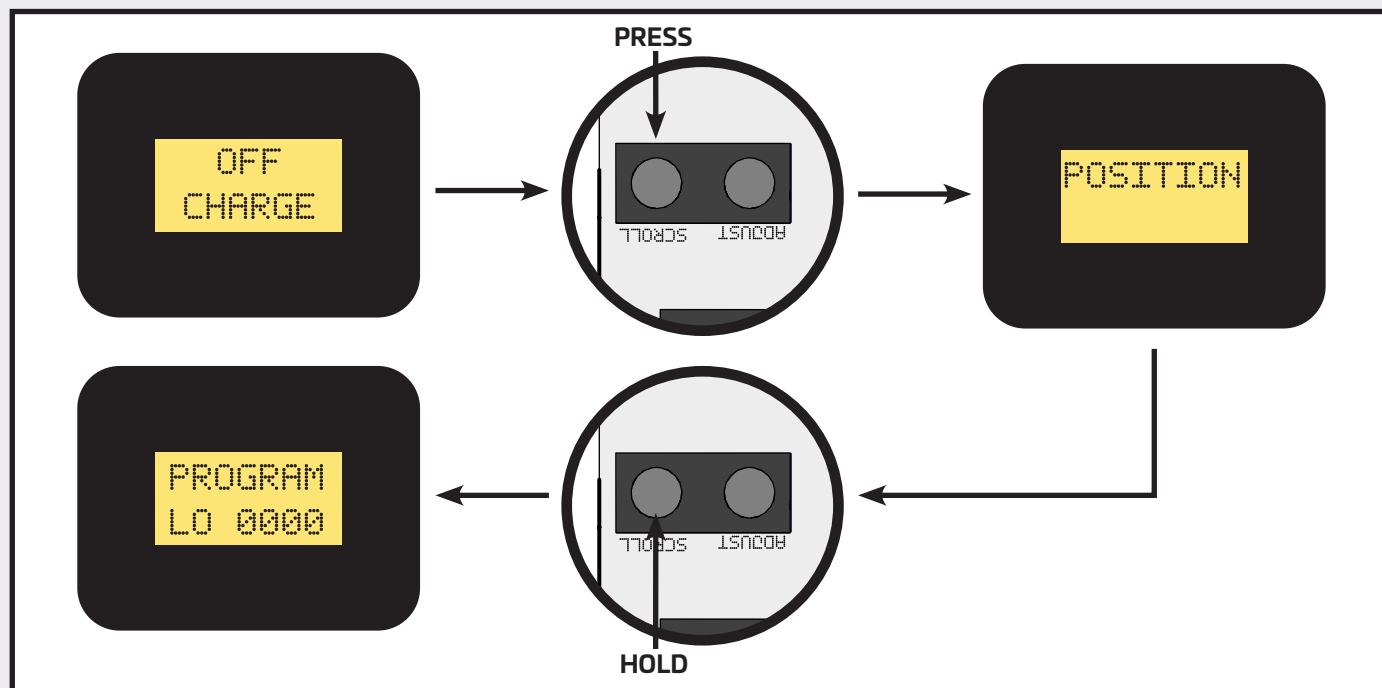
6.1 Programming the Stairlift

The display will read 'OFF CHARGE'.
Press SCROLL until the display reads 'POSITION'.
Press and hold the 'SCROLL' button.
The display should read 'PROGRAM LO 0000'
(The numbers will be different on each carriage).

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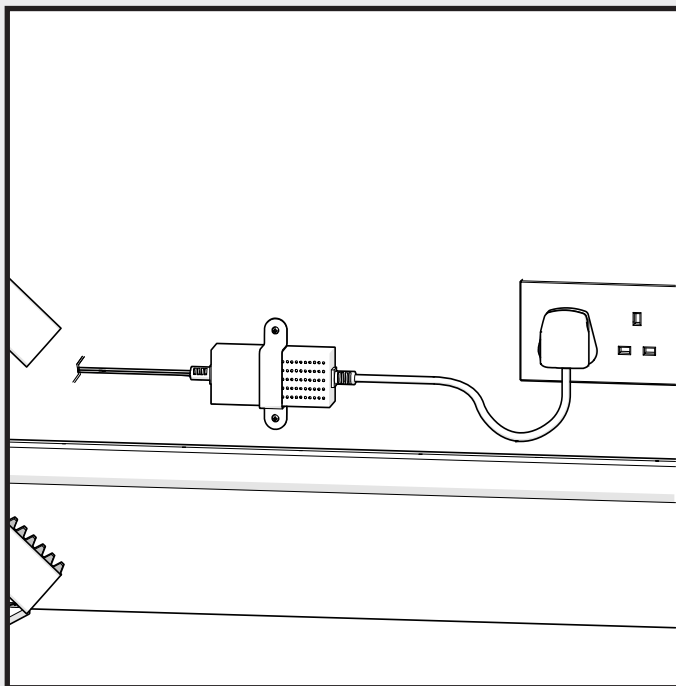
6.2 Standard Programming

Turn 'ON' the charger at the mains. (If an intermediate charge point has been installed, move the carriage down the rail until it is past this point prior to turning on the charger).

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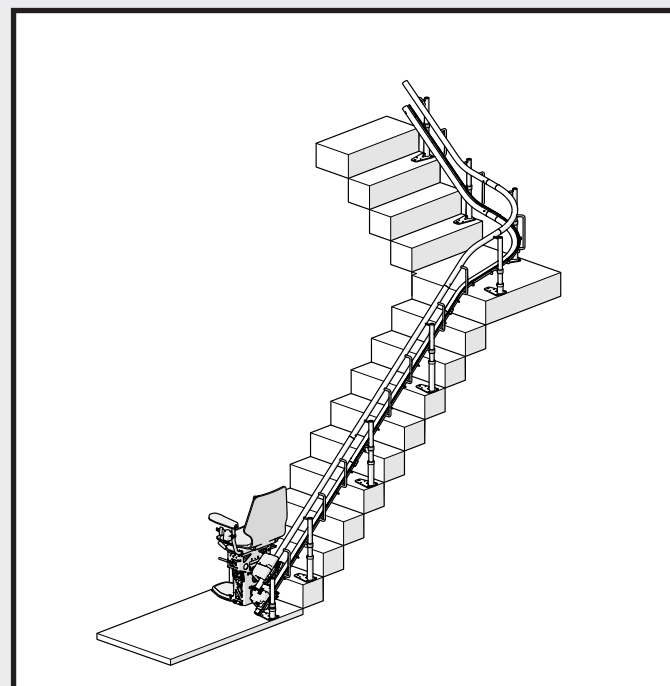
The lift should be positioned on the bottom charge point.

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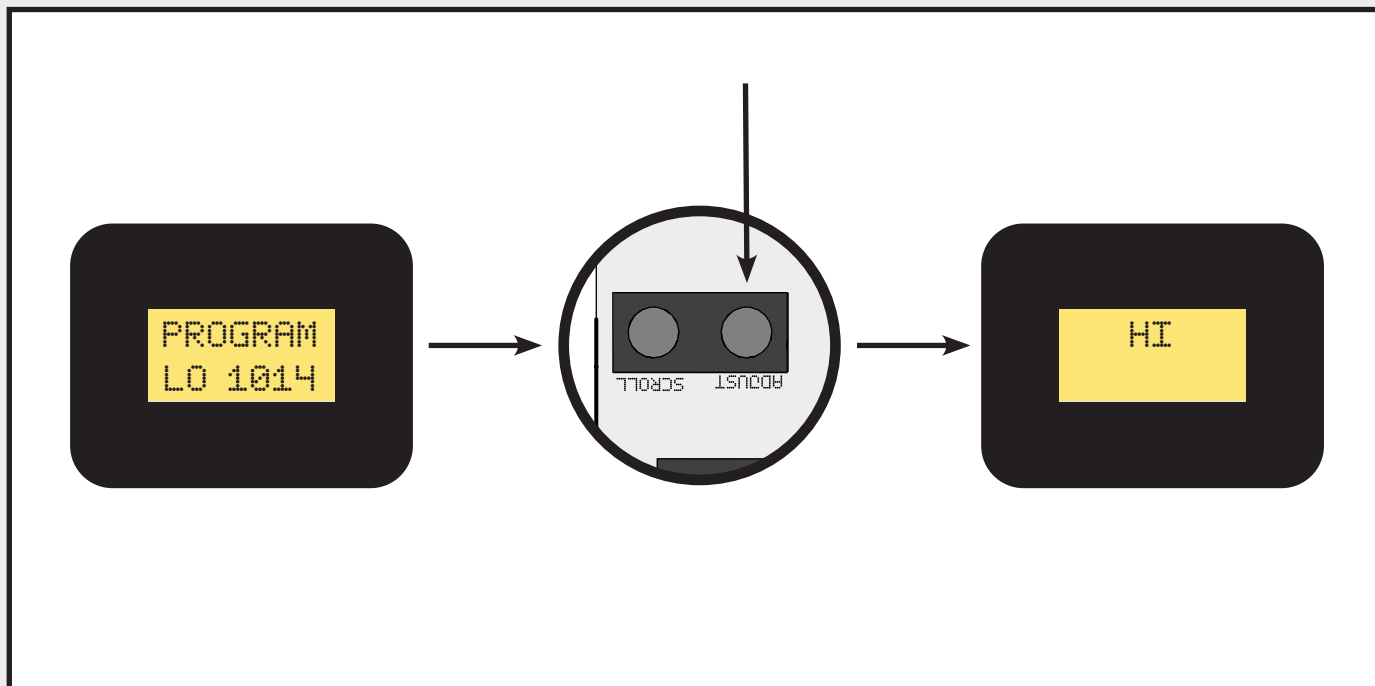
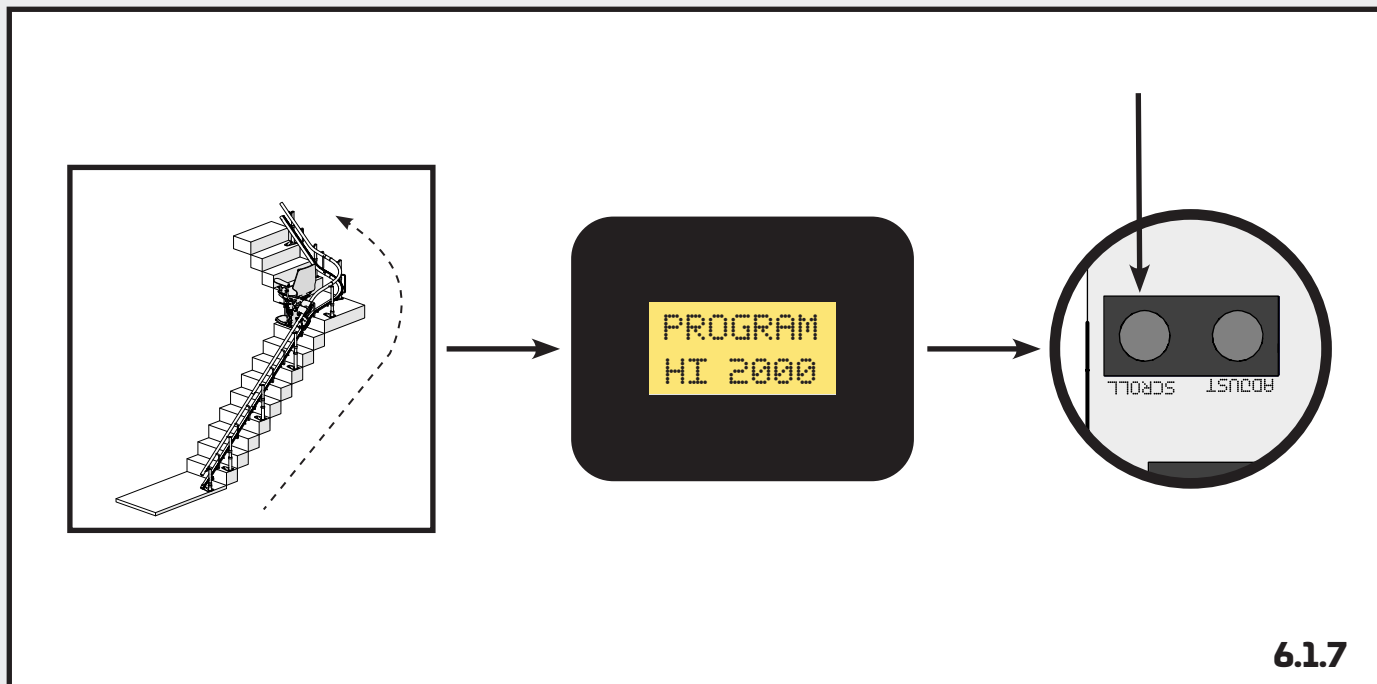
6.2 Standard Programming

Using the remote control, move the stairlift up the rail until it reaches the top charge point. The display will read a higher number as the stairlift travels. Press and hold the 'SCROLL' button.

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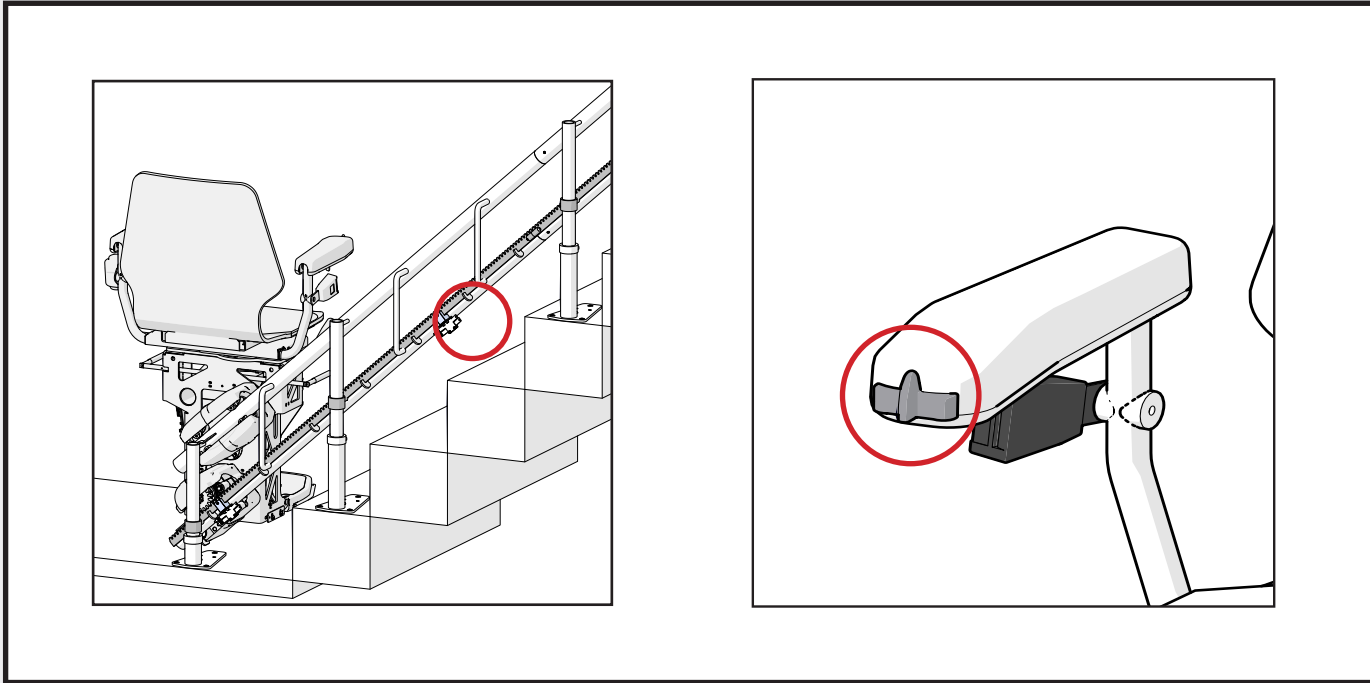
6.3 Programming an Intermediate Charge Point

If your stairlift has an intermediate charge point, when running the stairlift upwards the stairlift will stop on this charge point. Let go of the controller, then press up once again and continue to the top charge point. Press the scroll button.

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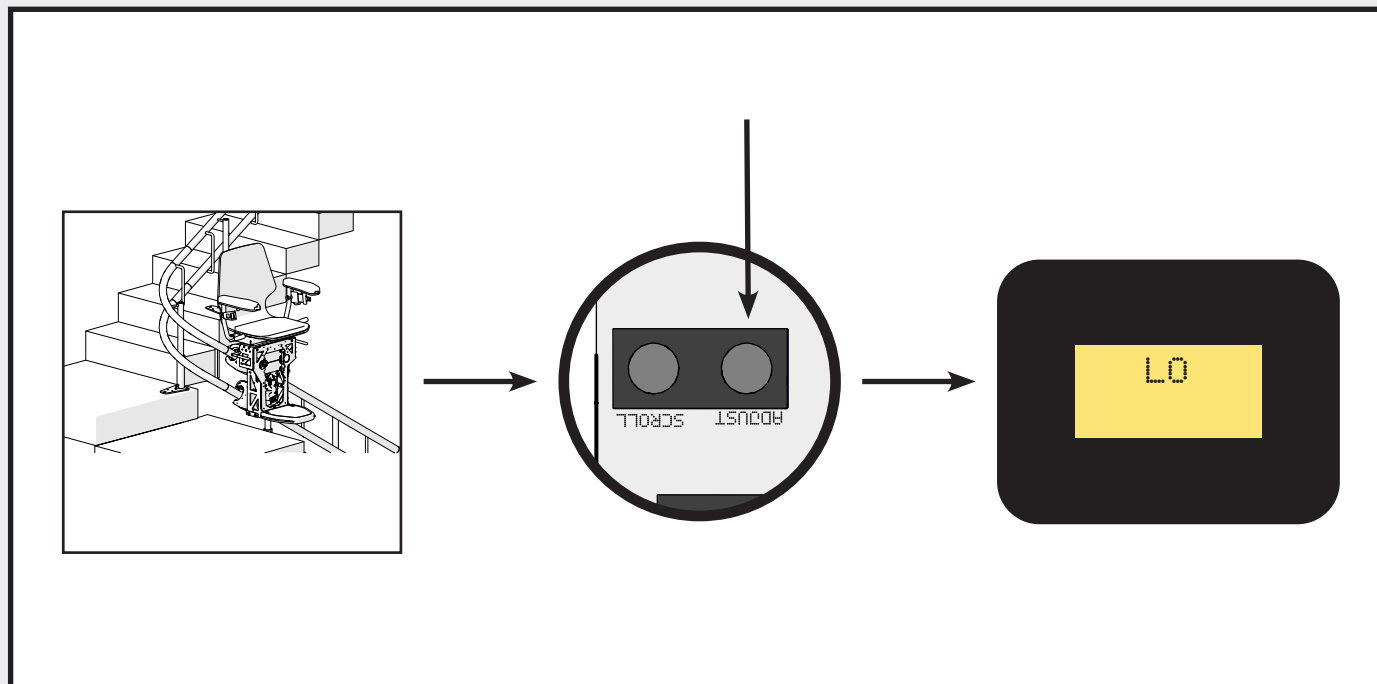
6.5 Setting the Bend Speed

To program the stairlift with internal bends follow steps 6.1.1 to 6.1.6 to start the programming process. Once the rail is on the bottom charge point, move the stairlift to a position just before the rail begins to bend.

Press the 'ADJUST' button and the LCD will display 'LO'
Move carriage in the up direction.

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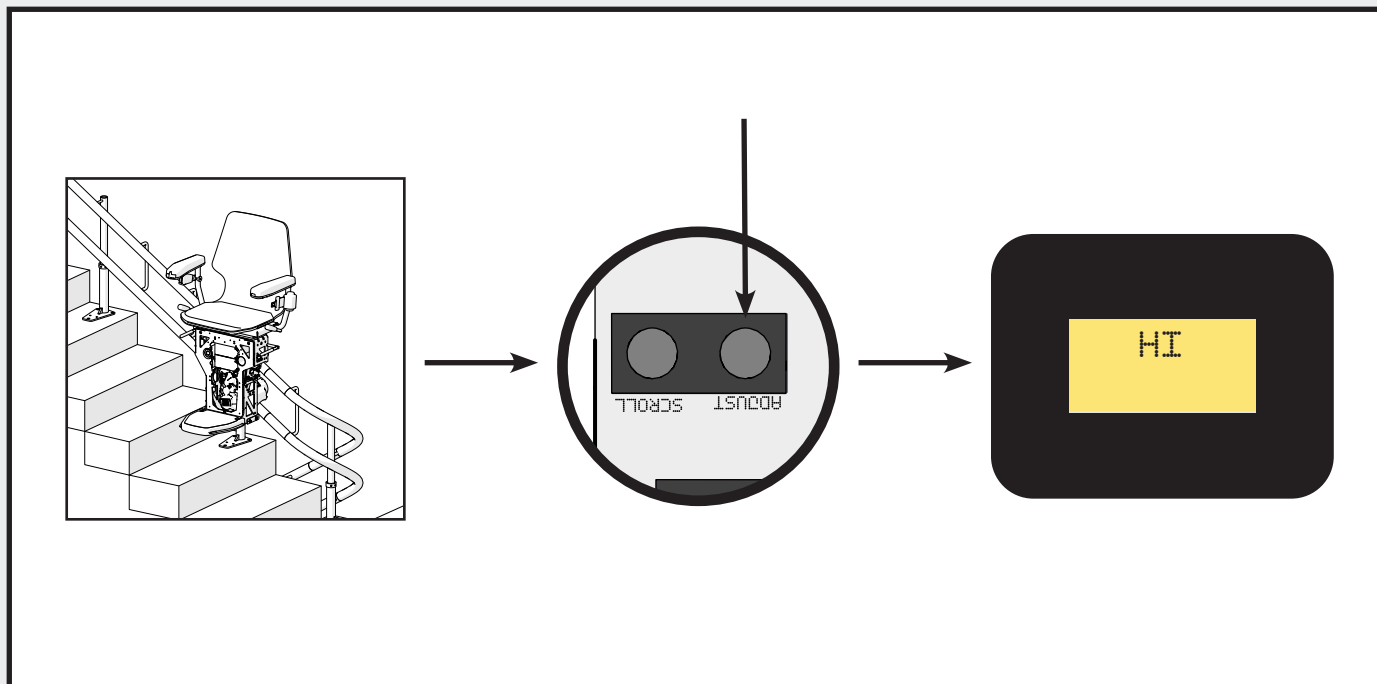
6.5 Setting the Bend Speed

Once the carriage is fully out of the bend, stop the stairlift, press the 'ADJUST' button and the LCD will display 'HI'. The carriage is now in high speed. Repeat the process if there is more than one internal bend.

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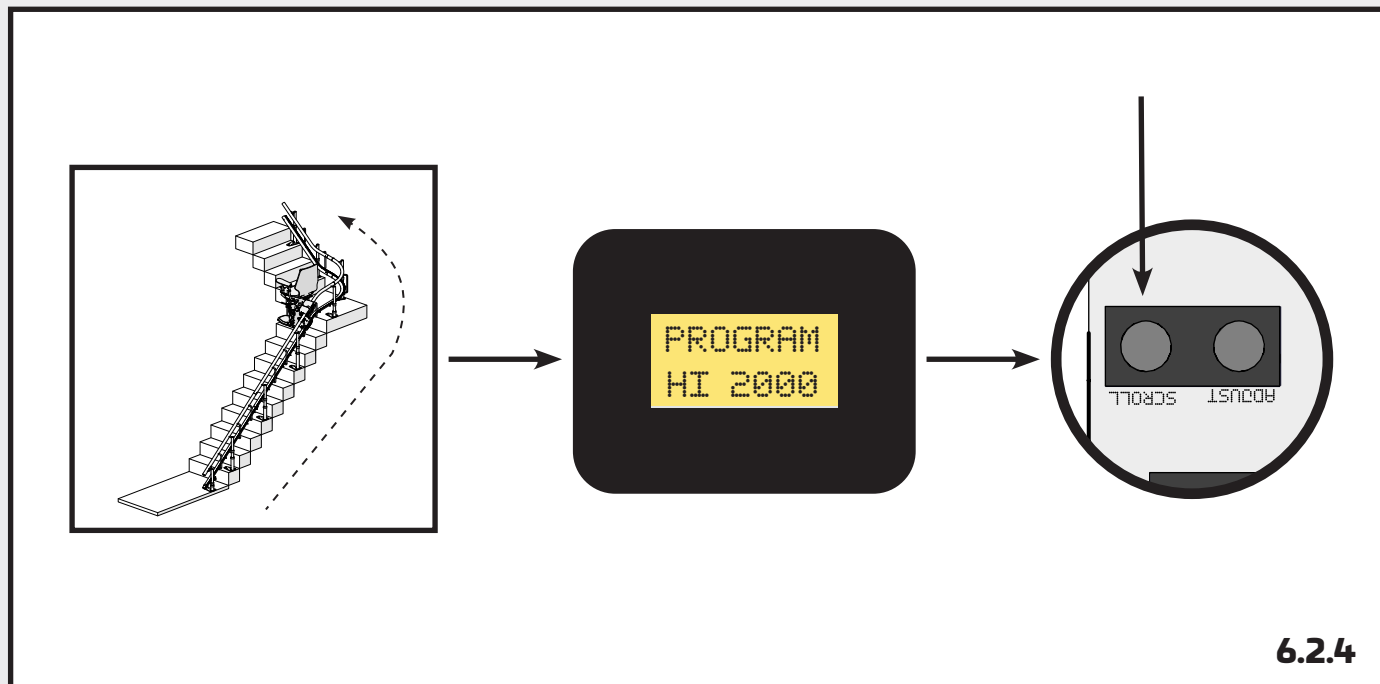
6.5 Setting the Bend Speed

Once the bend(s) speed is programmed, using the remote control, move the stairlift up the rail until it reaches the top charge point. The display will read a higher number as the stairlift travels. Press and hold the 'SCROLL' button.

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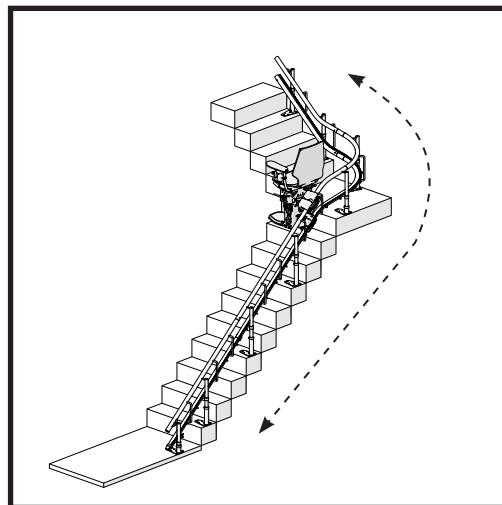
6.5 Setting the Bend Speed

6.2.5

The programming is now complete and the display will read 'INFINTY READY'. Test the rail by running the stairlift back down to the bottom charge point and then the top. The lift is now programmed.

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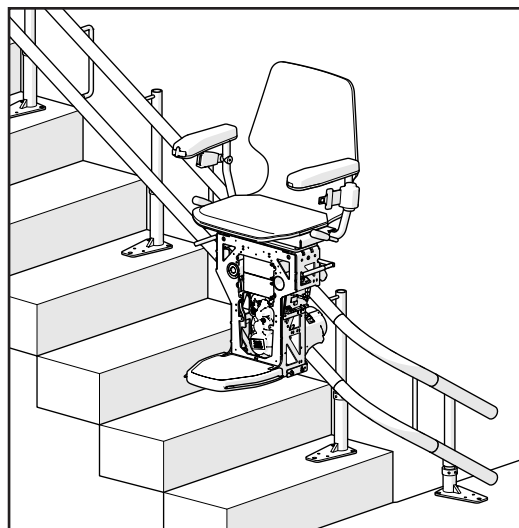
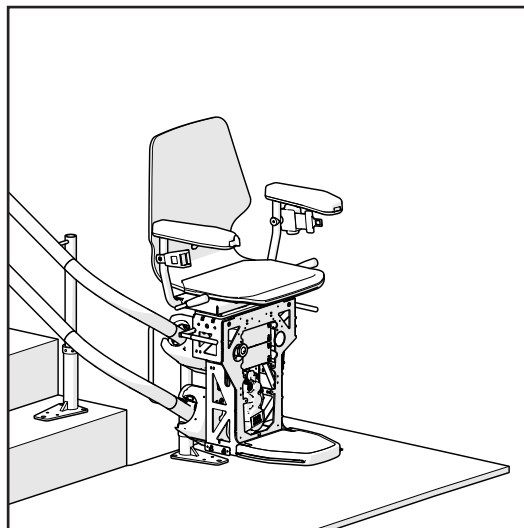
83

6.5 Setting the Bend Speed

For internal wrap starts follow the same process as sections 6.2.2 to 6.2.5 but begin the process in 'LO' speed.

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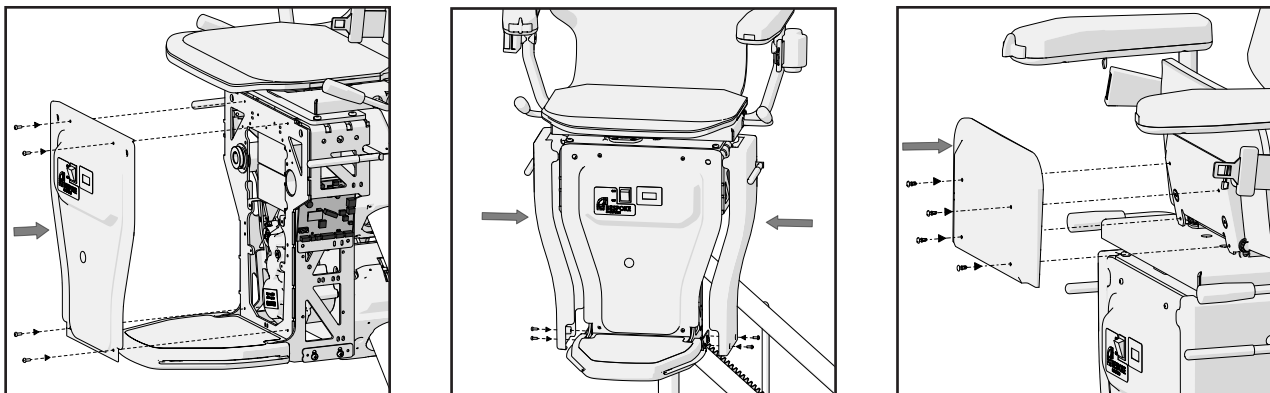
6.6 Refitting the Panels

Replace the front and side panels using the screws removed earlier. Finally, place the seat base cover using the fixings provided.

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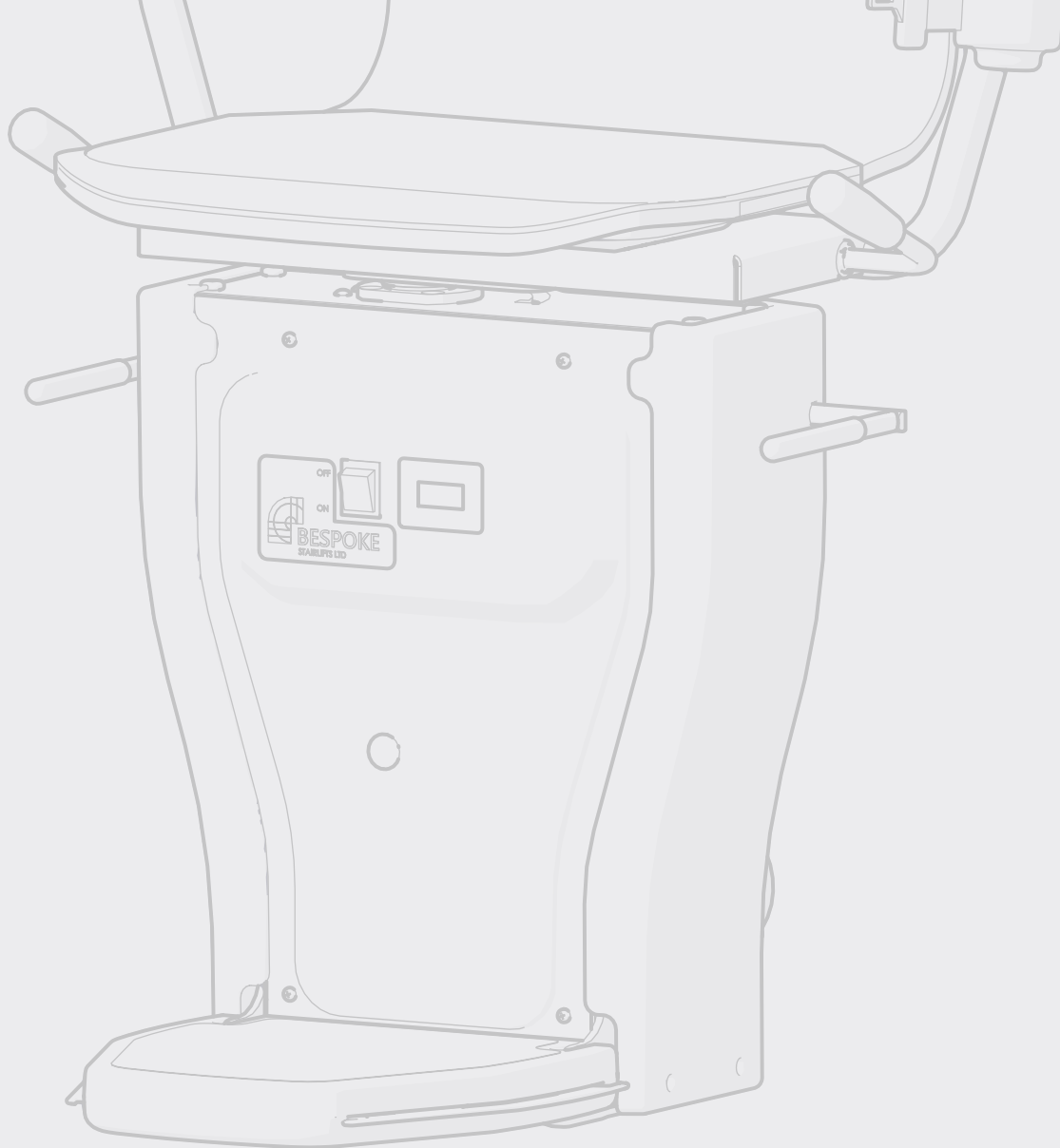
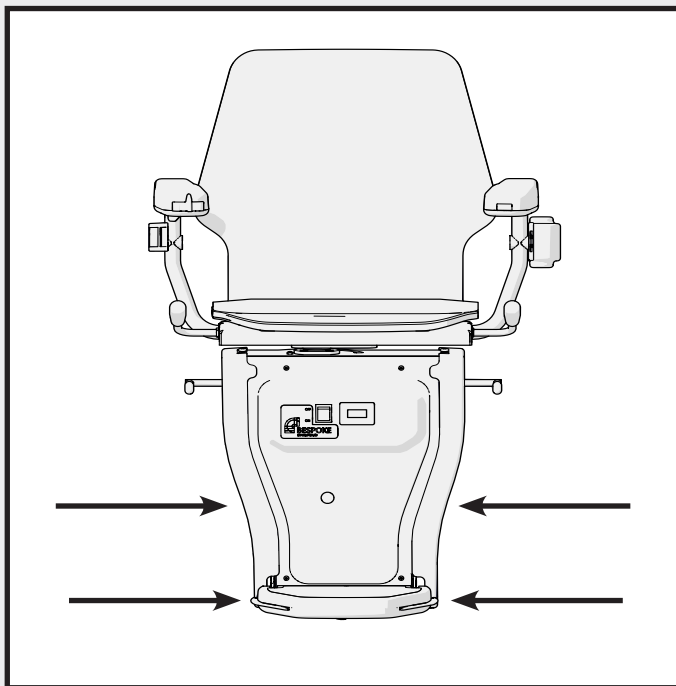
7.1 Front Safety Edges

To test the function of the front and side panel safety edges, run the carriage in either direction and press each safety edge shown in the diagrams.

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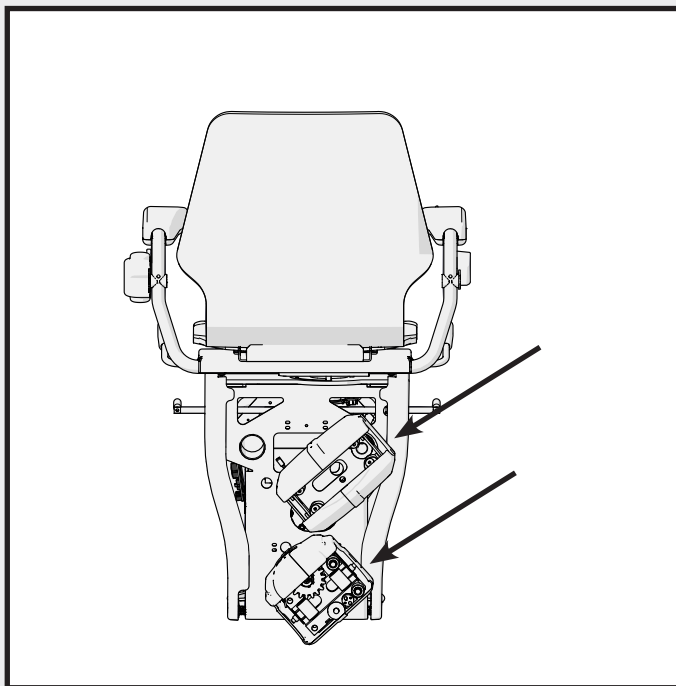
7.2 Rear Safety Edges

Continue the safety checks on the top and bottom skate safety edges as shown in the diagram.

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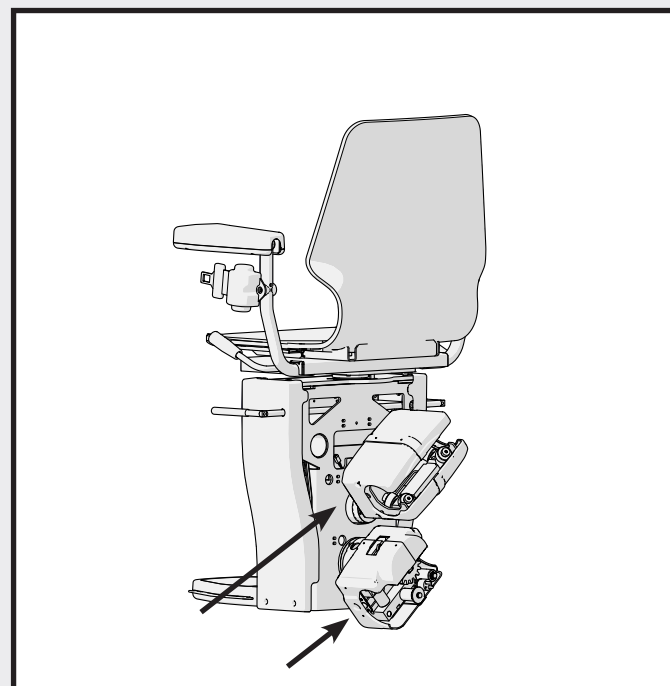
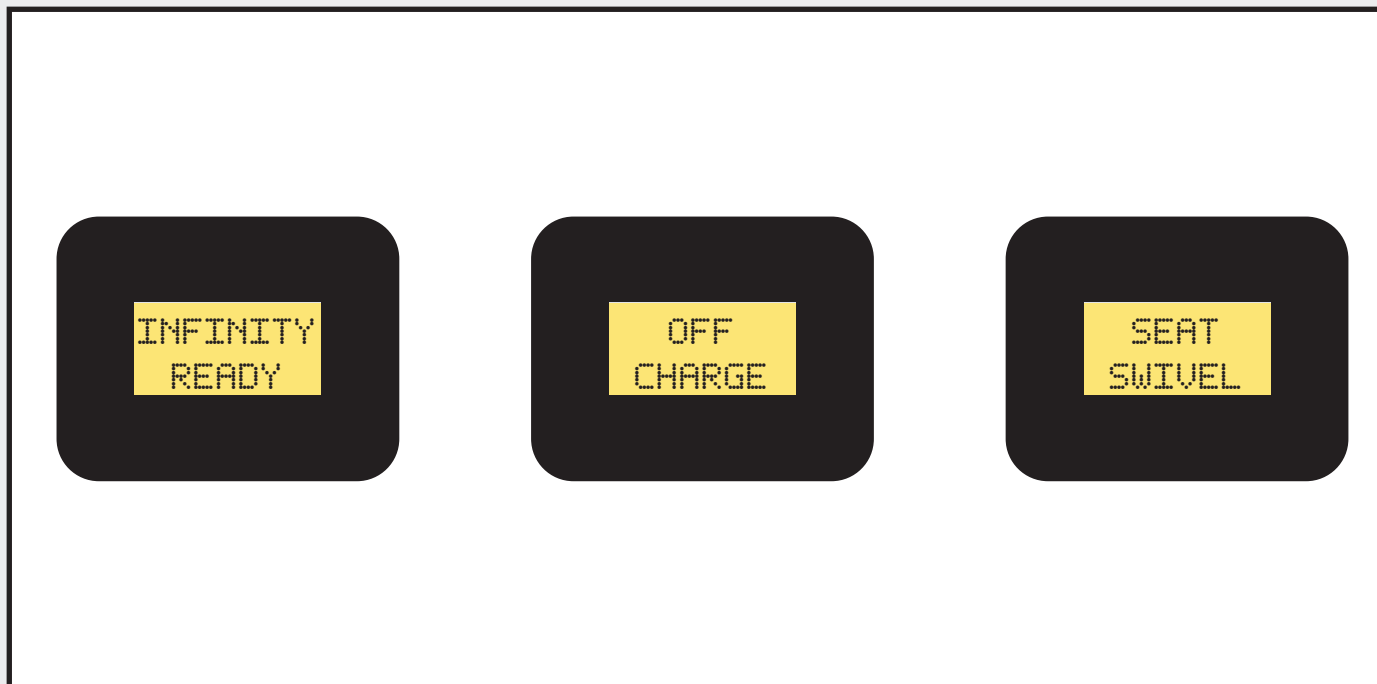
8.1 Diagnostics

Infinity Ready

This will show when the stairlift is on charge and ready to use.

Off Charge

This will show when the stairlift is off charge and will need moving to a charge point.

Seat swivel

Make sure the seat is swivelled in the correct position. Also check the switch and the wiring circuit.

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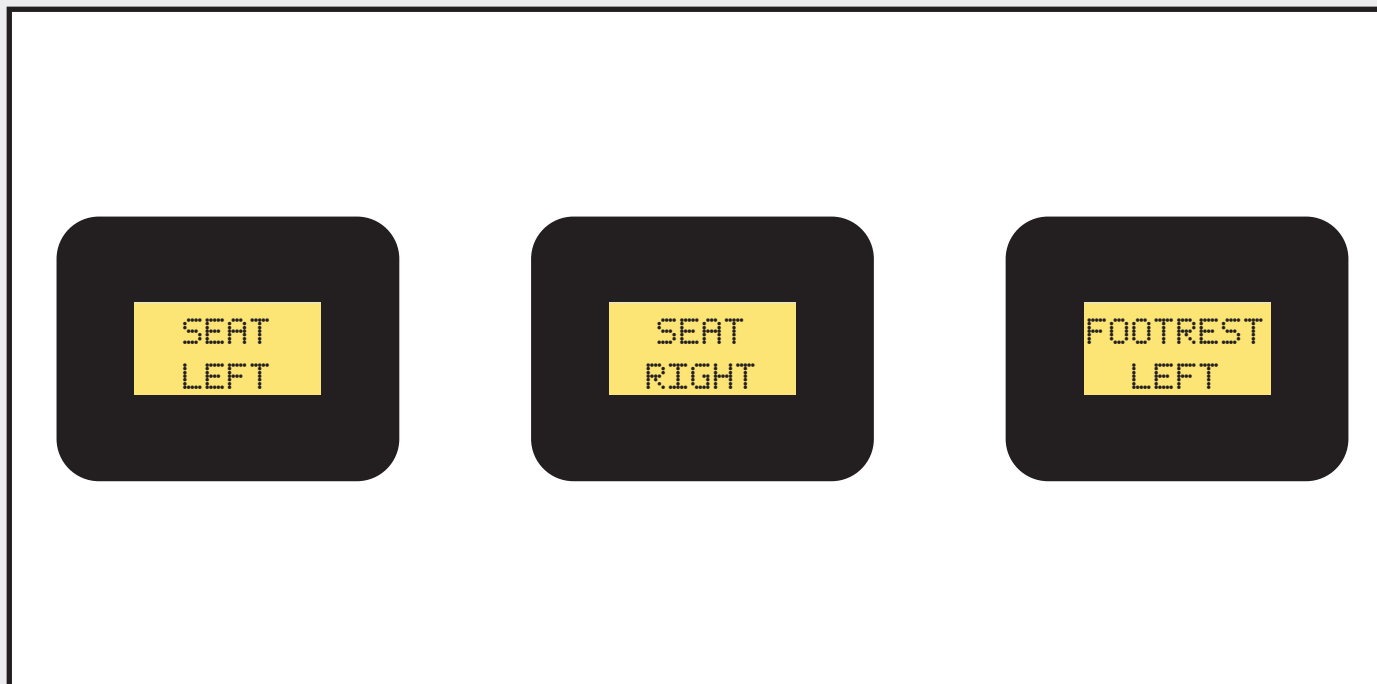
8.1 Diagnostics

Seat Left Check the chassis side plastic covers are free from obstruction. Also check the switch/wiring circuit.

Seat Right Check the chassis side plastic covers are free from obstruction. Also check the switch/wiring circuit.

Footrest Left Check the footrest sides are free from obstruction. Also check the switch/wiring circuit.

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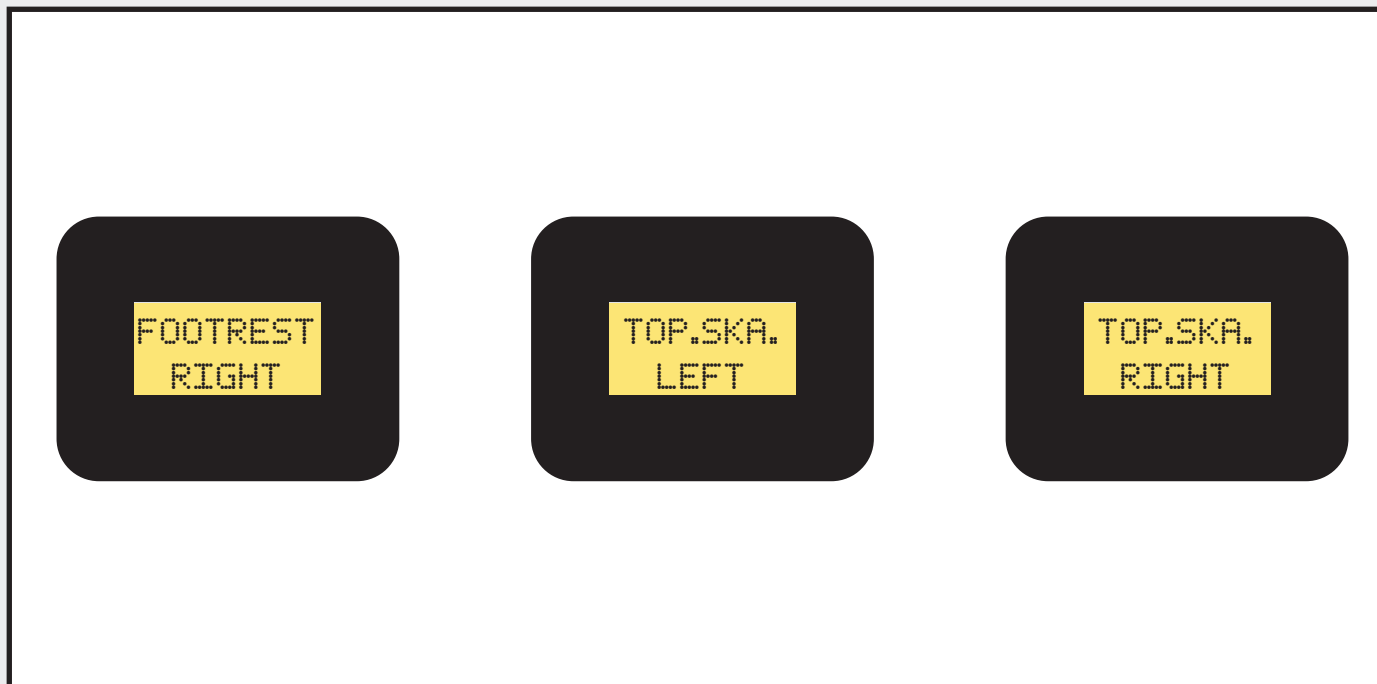
8.1 Diagnostics

Footrest Left Check the footrest sides are free from obstruction. Also check the switch/wiring circuit.

Top Ska Left Check the top skate plastic side covers are free from obstruction. Also check the switch/wiring circuits.

Top Ska Right Check the top skate plastic side covers are free from obstruction. Also check the switch/wiring circuits.

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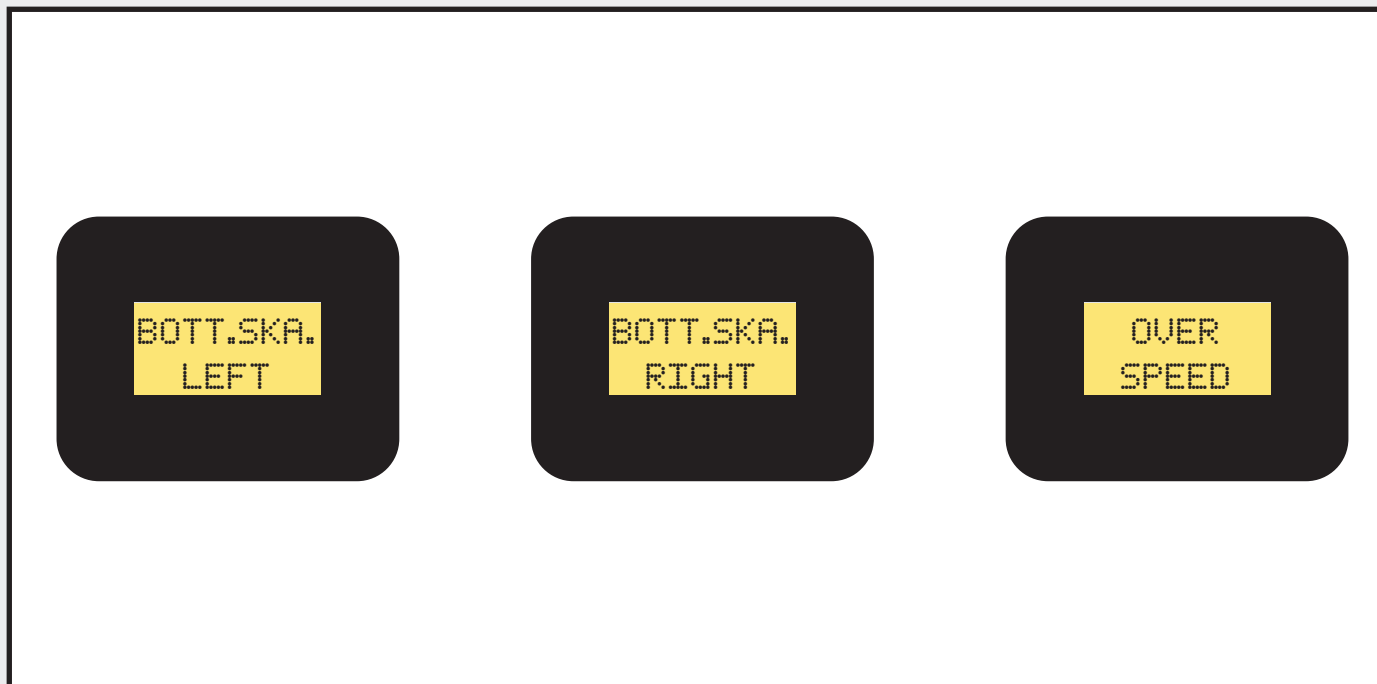
8.1 Diagnostics

Bottom Ska Left Check the bottom skate plastic side covers are free from obstruction. Also check the switch/wiring circuit.

Bottom Ska Right Check the bottom skate plastic side covers are free from obstruction. Also check the switch/wiring circuit.

Over Speed Fault Wind the stairlift up the rail for 5cm or until the fault code no longer shows. Always test in the up direction before testing in the down direction.

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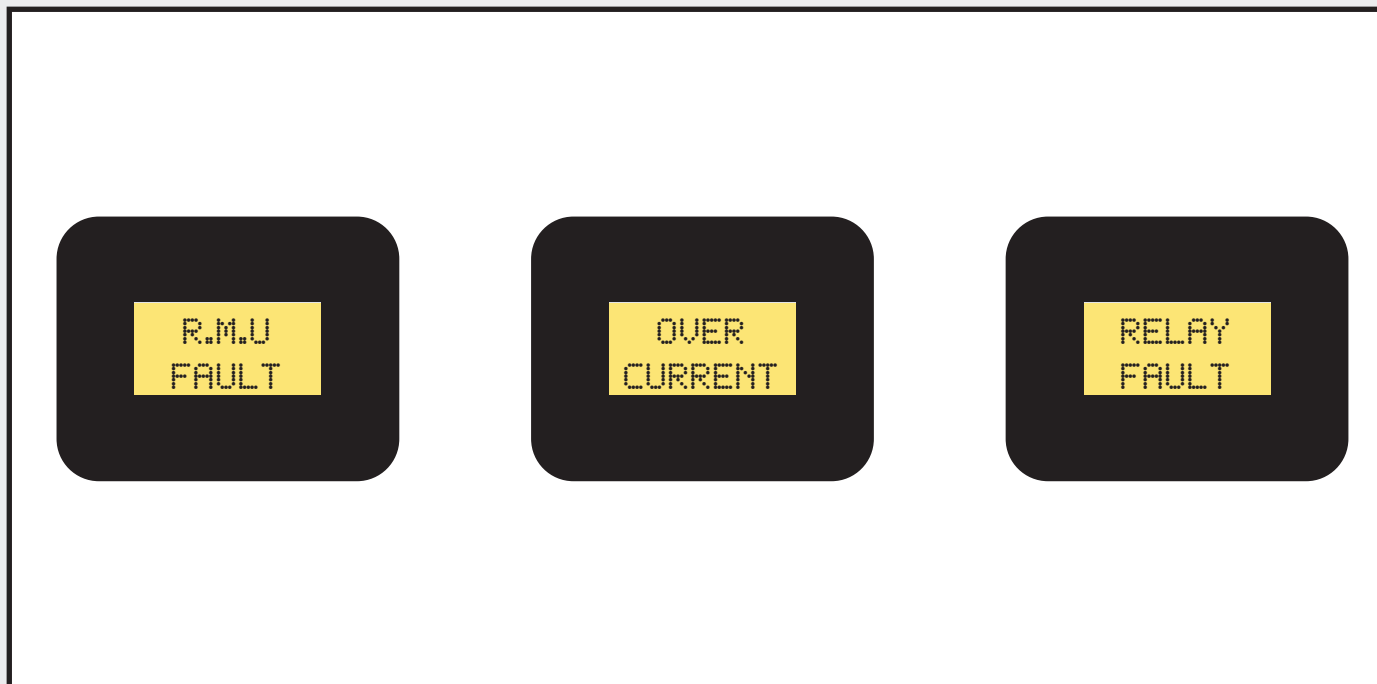
8.1 Diagnostics

Rmu Fault Check the roller on the motor shaft is rotating. Also check the switch/wiring circuit.

Over Current Check the racking/gear is free from obstruction. Check the customer doesn't exceed the weight limit.

Relay Fault Check the voltage from the batteries. Also try gently tapping the relay on the PCB.

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MULTIPLE
CALLS

BATTERY
CRITICAL

Lost Position Try reprogramming the Stairlift as per rail configuration.

Multiple Calls Check for a remote pressed in a drawer. Also check for a short circuit in the arm loom.

Battery Critical (Short Circuit) Check for two red LED'S on the PCB. Unplug each of the connectors from the PCB until the second light goes off. Then from that connector check each cable to earth until the short circuit is located.

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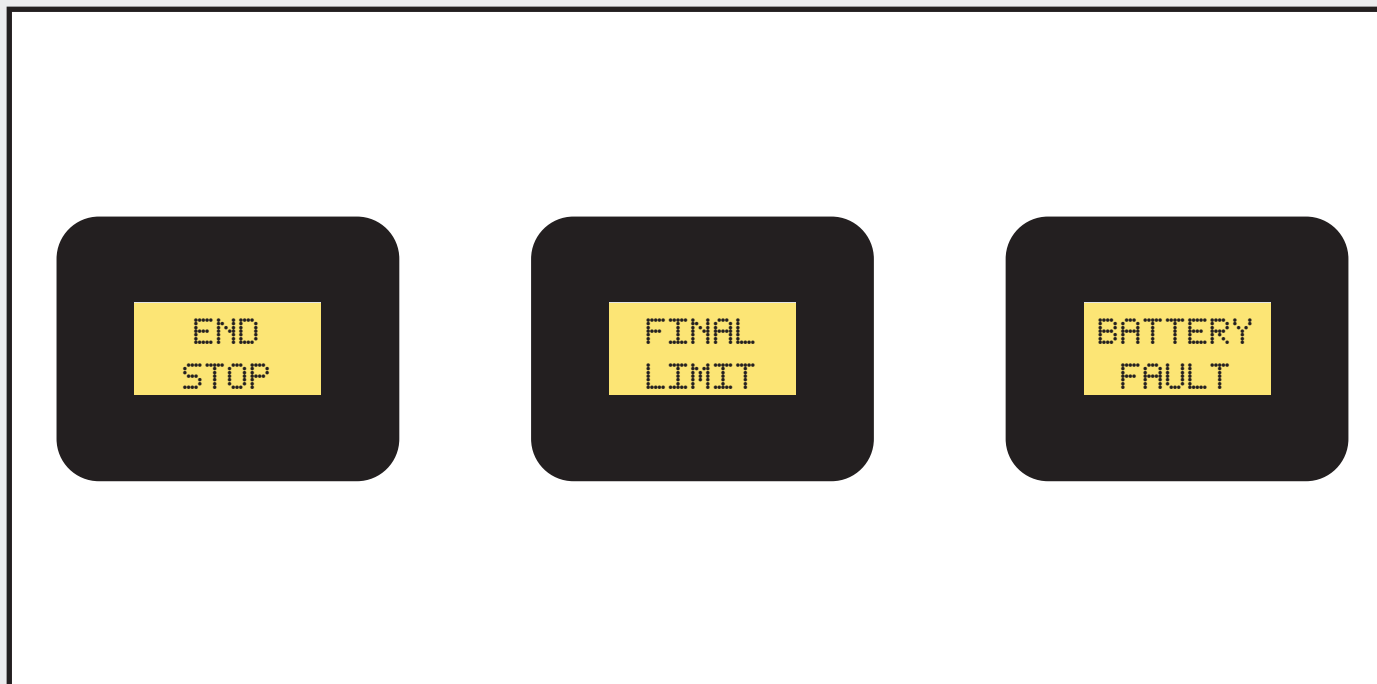
8.1 Diagnostics

End Stop Check the bottom skate covers are moving freely. Check the stairlift isn't running into the end stops. Try reprogramming.

Final Limit The stairlift is in straight mode in the PCB settings.

Battery Fault Check the connectors haven't come off the batteries. Check the in-line fuse hasn't blown

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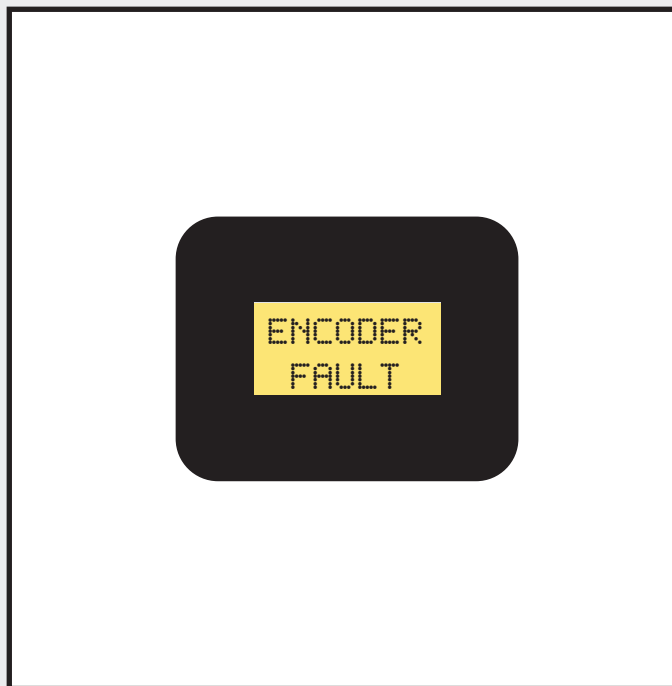
8.1 Diagnostics

Encoder Fault In Tornado motor and not AMT. Low voltage coming from the batteries or charge points. Faulty encoder cable or Motor cable.

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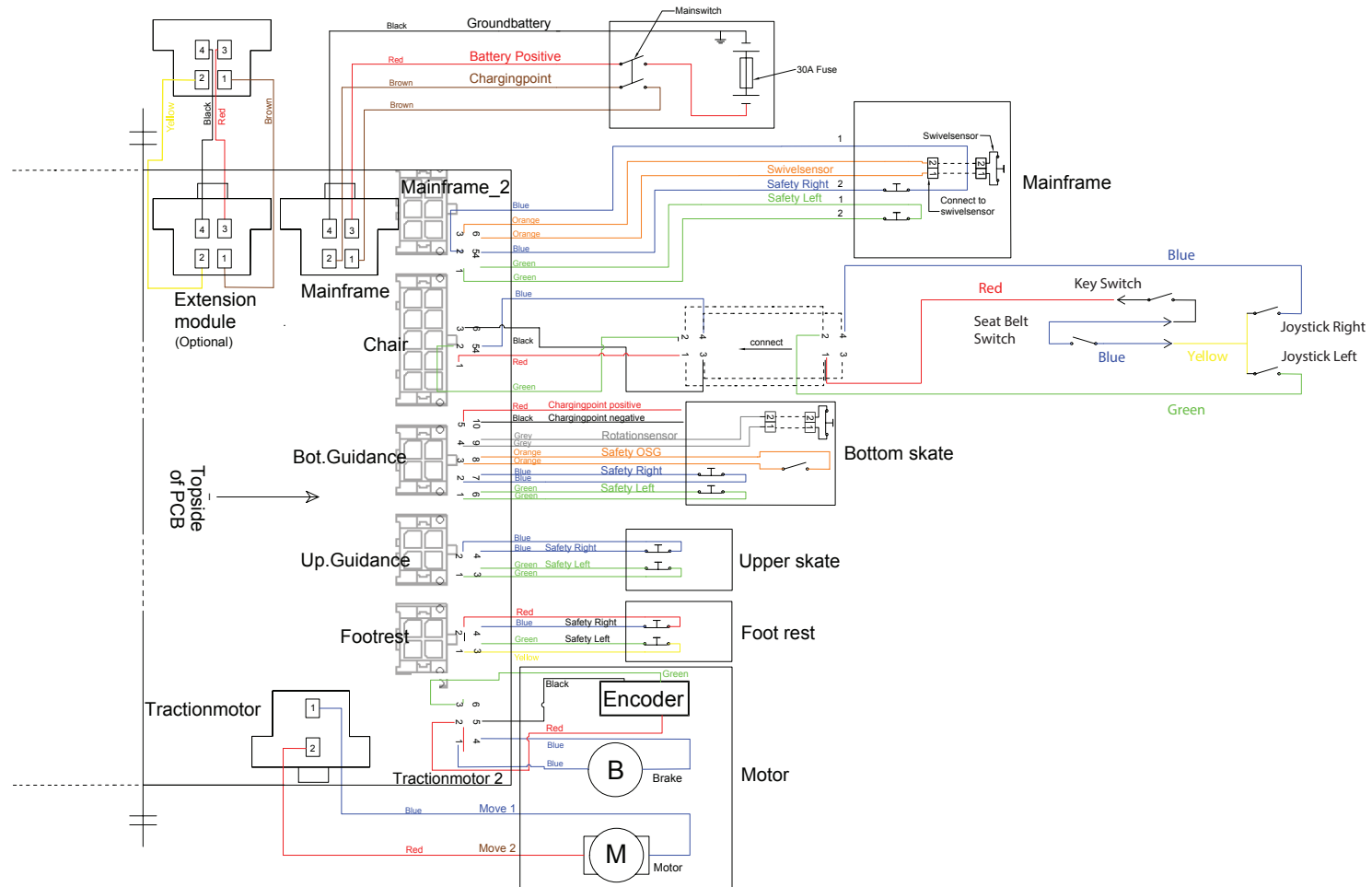
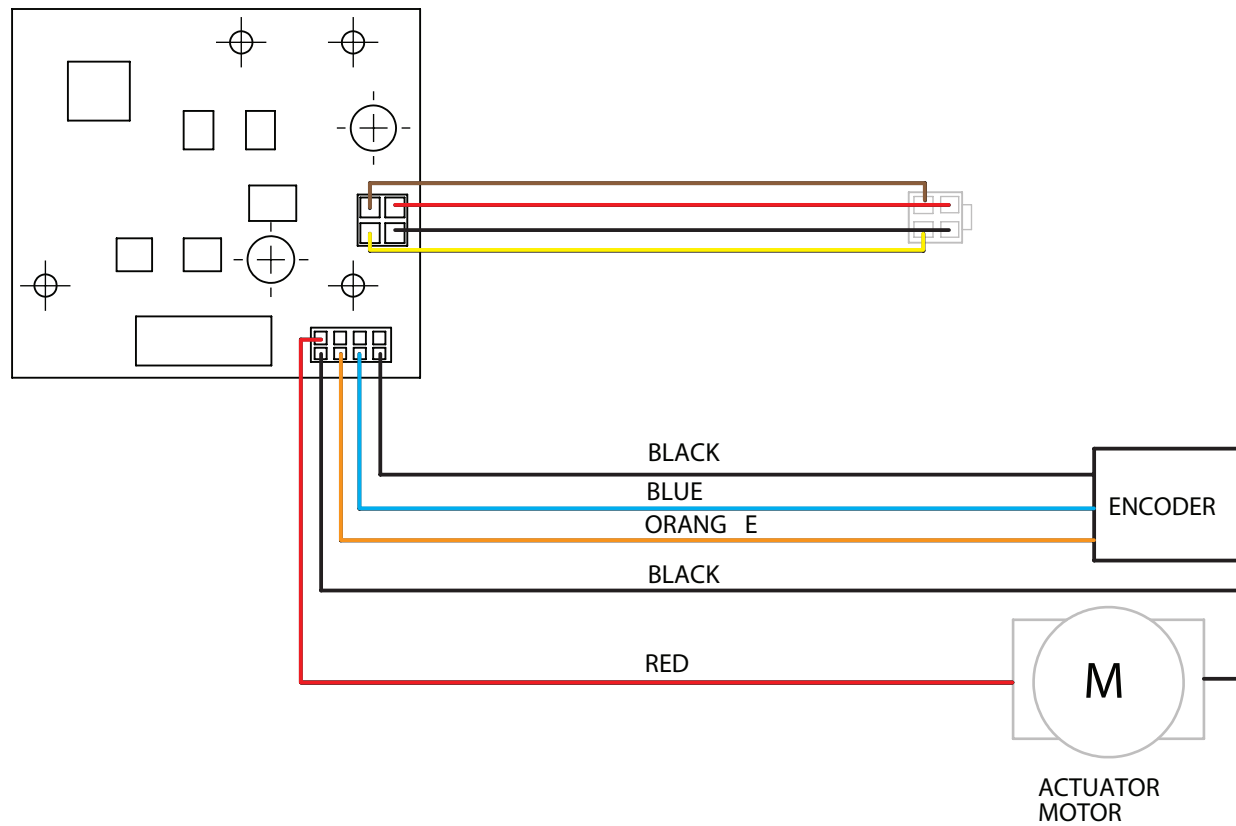
Unit 9, Old Fieldhouse Industrial Estate,
Old Fieldhouse Lane, Huddersfield,
West Yorkshire, HD2 1FA, England.

www.bespokestairlifts.com

2410116/INM/INF/1224